


Basic Recline (4XP)

97-188 2G Storm Series parts catalog for Invacare AT5544.

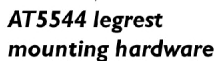

Power Elevating Legrest (4XP)

Genius and AT5544

Basic Recline (4XP)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0239000-NLA	NLA - Actuator Motor, Power Leg, 4XP (4" Stroke)	1	1
2		0053600-NLA	NLA - Pin, Roll (1/8 x 1-1/8")	1	1
3		1106223	Screw, Socket Head (3/8-16 x 2-1/2")	1	1
4		0054106-NLA	NLA - Locknut, High Profile (3/8-16)	1	1
5		0053900-NLA	NLA - Washer, Nylon (3/8 x 13/16 x 1/16")	1	2
6		1077509	Actuator Motor w/ Hardware, Power Leg, 4XP (4" Stroke)	1	1
7		0236102-NLA	NLA - Brace, Cross, Power Leg - Rear (4XP) (16" W)	1	1
7		0236202-NLA	NLA - Brace, Cross, Power Leg - Rear (4XP) (18" W)	1	1
8		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	4
9		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	10
10		7930333	Locknut (1/4-20)	1	6
11	C	0238206-NLA	NLA - Screw, Hex Head (1/2-20 x 2") (For 16" Seat Depth)	1	1
11	D	0238306-NLA	NLA - Screw, Hex Head (1/2-20 x 4") (For 18" Seat Depth)	1	1
11	E	0238406	Screw, Hex Head (1/2-20 x 6") (For 20" Seat Depth)	1	1
12	D	0235622-NLA	NLA - Actuator Extension, 2" (18" Seat Depth)	1	1
12	E	0235722-NLA	NLA - Actuator Extension, 4" (20" Seat Depth)	1	1
13		0235802-NLA	NLA - Bar, Push, Power Leg (4XP) (16" W)	1	1
13		0235902-NLA	NLA - Push Bar, Power Leg (4XP) 18" Wide	1	1
14		1071237	Plug Button, Square, Black (1")	1	2
15		0132206-NLA	NLA - Screw, Allen Head (1/4-20 x 1-3/4")	1	2
16		0026400-NLA	NLA - Receptacle, Power Legrest	1	2
17		0025306-NLA	NLA - Push Pin, AT5544	1	1
18		0236602	Bracket, Push - Right	1	1
18		0236502	Bracket, Push - Left	1	1
19		0055206-NLA	NLA - Screw, Cap, Button Head, Socket (5/16-18 x 1-3/4")	1	4
20		0055900-NLA	NLA - U-Clamp (7/8")	1	2
21		0054806-NLA	NLA - Washer, Flat (11/32 x 11/16 x 1/16")	1	10
22		0054906	Locknut, High Profile (5/16-18)	1	4
23		0054106-NLA	NLA - Locknut, High Profile (3/8-16)	1	2
24	A	1084181	Hardware, Legrest Mounting, AT5554 - Right	1	1
24	A	1084182	Hardware, Legrest Mounting, AT5554 - Left	1	1
25		0203906-NLA	NLA - Pin, Push, Inside Mount, Genius	1	2
26	B	1077518	Hardware, Legrest Mounting, Genius	1	2
NOTE: A - Includes items 17-23 B - Includes items 23, 25 C - 16" Seat Depth D - 18" Seat Depth E - 20" Seat Depth					