


Profile Back Assembly

Sliding, VSR

Tilt/Recline & Recline Only

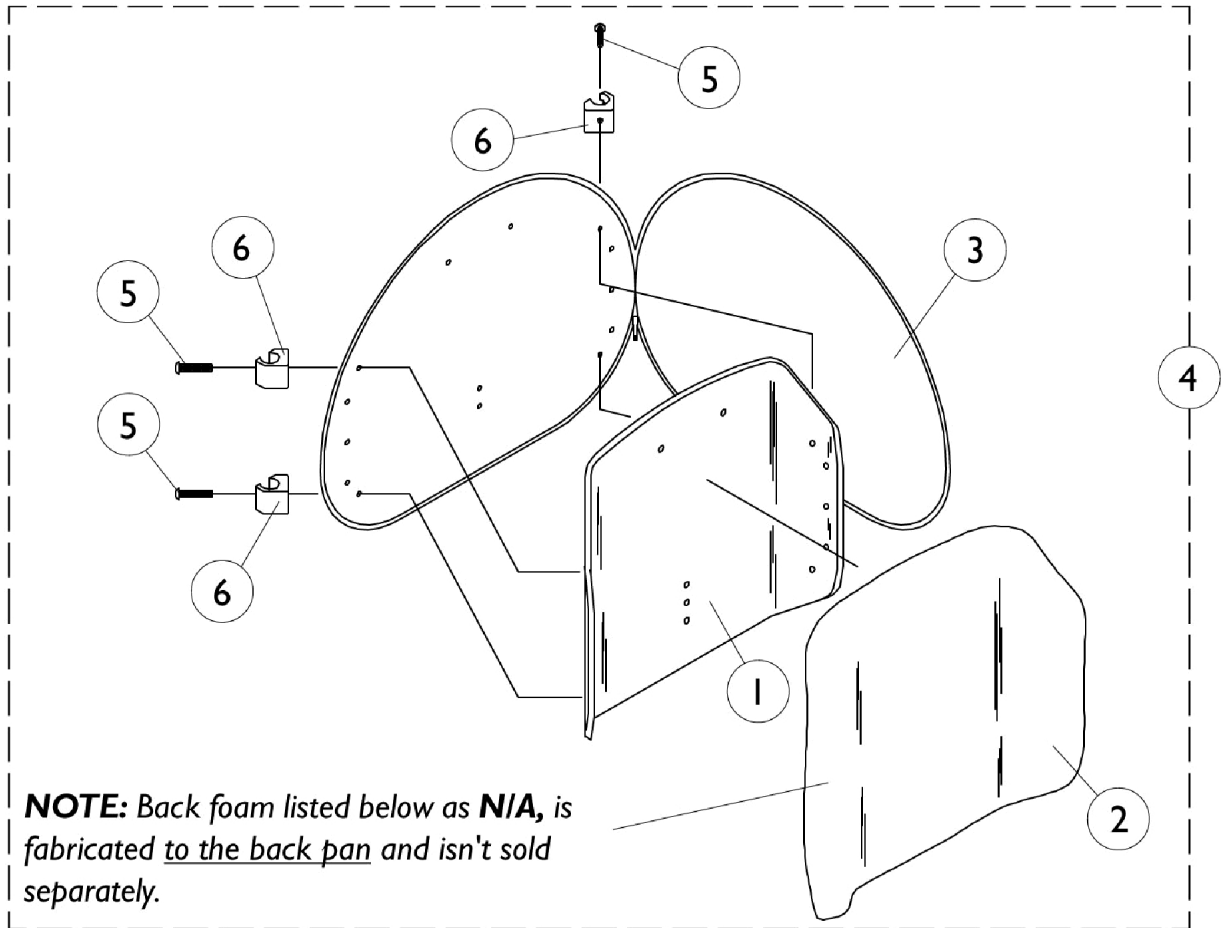

Profile Back Assembly

Sliding, VSR

Tilt/Recline & Recline Only

Material Back Width x Height	Overall Back Height	Item #1 Back Pan	Item #2 Back Foam	Item #3 Back Cover	Item #4 Back Assembly w/Sliding Hdwr
16" x 16"	23" x 24"H	0133019-NLA	0259220-NLA	0276800-NLA	0323200-NLA
17" x 16"	23" x 24"H	0133019-NLA	0259220-NLA	0276800-NLA	1072908-NLA
18" x 16"	23" x 24"H	0133119-NLA	0259120-NLA	0276900-NLA	0323300-NLA
19" x 16"	23" x 24"H	0133119-NLA	0259120-NLA	0276900-NLA	1072909-NLA
20" x 16"	23" x 24"H	0133219-NLA	N/A	0277000-NLA	0323400-NLA
16" x 18"	25"	0133319-NLA	N/A	0277100	0323500-NLA
17" x 18"	25"	0133319-NLA	N/A	0277100	1072910-NLA
18" x 18"	25"	0133419-NLA	0259020-NLA	0277200	0323600-NLA
19" x 18"	25"	0133419-NLA	0259020-NLA	0277200	1072911-NLA
20" x 18"	25"	0133519	N/A	0277300-NLA	0323700-NLA

Profile Back Assembly

Sliding, VSR

Tilt/Recline & Recline Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-4		See Chart	#'s 1-4 Profile Back Assembly (Sliding, VSR)	1	1
5	B	0052906-NLA	NLA - Screw, Socket Button Head (1/4-20 x 3/4")	1	4
5	C	0051206-NLA	NLA - Screw, Machine, Hex Socket, Button Head (1/4-20 x 1-1/4)	1	4
6		0121200	Block, Glide (16"/18"/20" W)	1	4
6		0186400-NLA	NLA - Block, Glide (17"/19" W)	1	4
	A,D	0143900-NLA	NLA - Flap, Back Pan - Tarsys	1	1
	A,E	0144000-NLA	NLA - Flap, Back Pan - Tarsys	1	1
	A,F	0144100-NLA	NLA - Flap, Back Pan - Tarsys	1	1
NOTE: A - Not Shown B - 16"/18"/20" Wide C - 17"/19" Wide D - 16"-17" Wide E - 18"-19" Wide F - 20" Wide					