
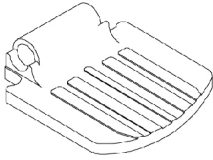
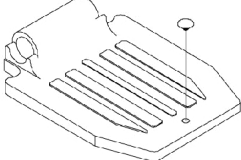
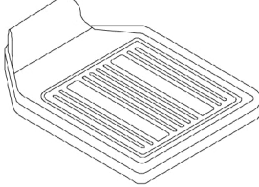
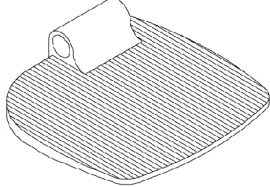


# Pivot/ Slide Tube and STD. (1650), Deluxe (1450) and X-Large (1350) Aluminum Footplates

			
<p><b>14" Wide</b> <b>1650</b> <b>Standard Alum.</b></p>	<p><b>16"-22" Wide</b> <b>1650</b> <b>Standard Alum.</b></p>	<p><b>16"-22" Wide</b> <b>1450</b> <b>Deluxe Alum.</b></p>	<p><b>14"-22" Wide</b> <b>1350</b> <b>X-Large Alum.</b></p>

For complete pivot/slide tube and footplates see Front Riggings Reference Chart page.

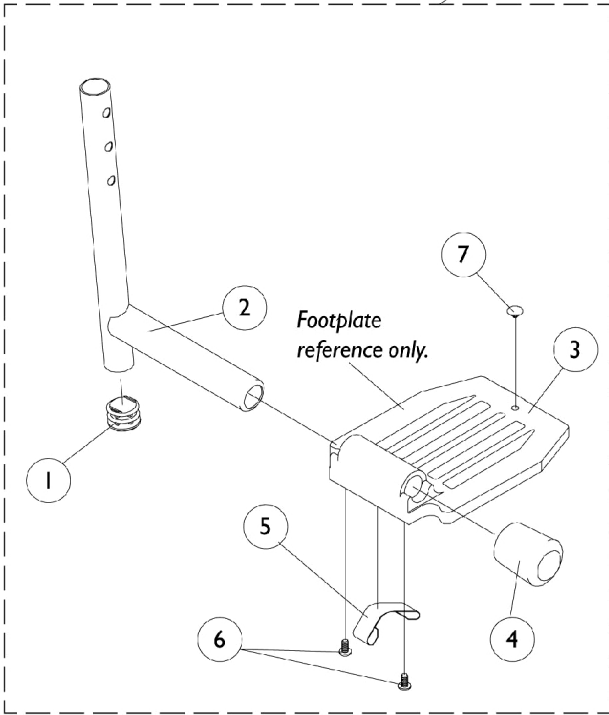

## Pivot/ Slide Tube and STD. (1650), Deluxe (1450) and X-Large (1350) Aluminum Footplates

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	A,F	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A	1086481-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	A,F	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	B,F	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C,F	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	F	1086479-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	F	1086480-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	C,E	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
2	D	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	D	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	D,E,F	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D,G	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
3		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3		1002343	Footplate, Aluminum, Extra Large, Black - Right (7" W x 8-1/2" L)	1	1
3		1002344	Footplate, Aluminum, Extra Large, Black - Left (7" W x 8-1/2" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	E	1016346	Cap, End, Tube, Black (7/8")	1	1
4	F	1027095	Plug Button, Black (7/8")	1	1
5	E,H	1006981	Spring, Tension	1	2
5	G,H	1036120-NLA	NLA - Spring	1	2
5	F,J	1036123	Spring, Leaf	1	1
6	I	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	1	2
7		1070168	Plug, Push, Footplate	1	1

# Pivot/ Slide Tube and STD. (1650), Deluxe (1450) and X-Large (1350) Aluminum Footplates

NOTE: A - 60MFX, 60HD, 70NHD, PW93, PW93E  
B - PAL4  
C - P93M  
D - P904, AT5544  
E - Standard Aluminium Footplate  
F - Extra Large Footplate  
G - Deluxe  
H - Takes (2) per side  
I - Takes (1) per side  
J - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.