
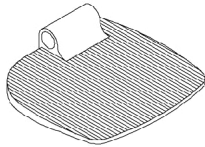
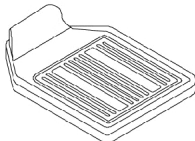
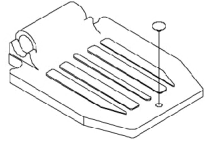
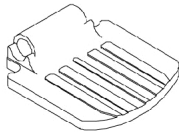


Pivot/Slide Tube and 1650, 1450, 1350 Aluminum Footplates

NOTE: Heel loops are not included. See footrest and legrest accessories.

NOTE: For complete pivot/ slide tube and footplate assemblies, see front riggings reference charts.

 <p>1350 Extra Large</p>	 <p>1450 Deluxe Aluminum</p>
 <p>16"-22" Wide 1650 Standard Alum.</p>	 <p>14" Wide 1650 Standard Alum.</p>

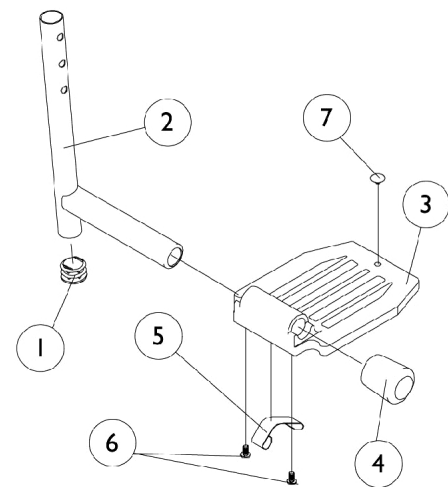

Pivot/Slide Tube and 1650, 1450, 1350 Aluminum Footplates

NOTE: Heel loops are not included. See footrest and legrest accessories.

NOTE: For complete pivot/ slide tube and footplate assemblies, see front riggings reference charts.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	A	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2		1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2		1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D,B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	E	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	E	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2		1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	C	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	D,B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	E,F	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
2	C	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminium, Black	1	1
3	G	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3	H	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	I	1016346	Cap, End, Tube, Black (7/8")	1	1
4	J	1027095	Plug Button, Black (7/8")	1	1
5	I	1006981	Spring, Tension	1	2
5	K,L	1036120-NLA	NLA - Spring	1	2
5	J,N	1036123	Spring, Leaf	1	1
6	M	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
7		1070168	Plug, Push, Footplate	1	1

Pivot/Slide Tube and 1650, 1450, 1350 Aluminum Footplates

NOTE: Heel loops are not included. See footrest and legrest accessories.

NOTE: For complete pivot/ slide tube and footplate assemblies, see front riggings reference charts.

NOTE: A - 60MFX, AT5544, 60, 70, PW93, PW93E

B - PAL4

C - P93M

D - PAS4A

E - P904

F - AT5544

G - 14"-15" Wide

H - 16" and Wider

I - Standard Aluminium Footplate

J - Extra Large Footplate

K - Deluxe

L - Takes (2) per side

M - Deluxe takes (1)

N - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.