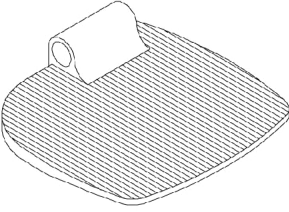
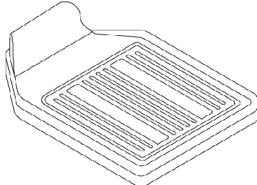
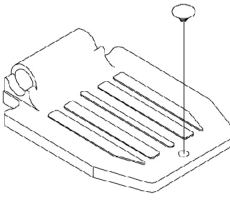
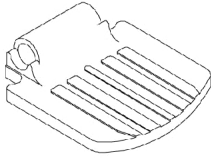



Pivot & Slide Tube w/ Footplate

P93M, PW93, and PW93E


Note: Heel Loops are not included. See footrest and legrest accessories.

Note: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

			
1350 Extra large	1450 Deluxe	16 to 20" Wide 1650 Standard Alum.	14" Wide 1650 Standard Alum.

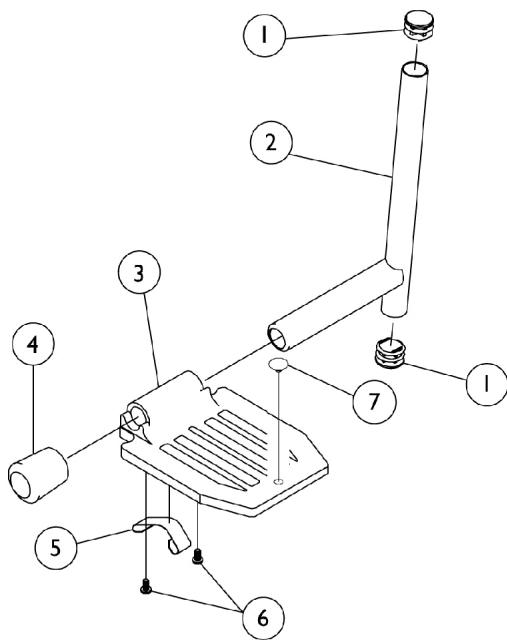

P93M

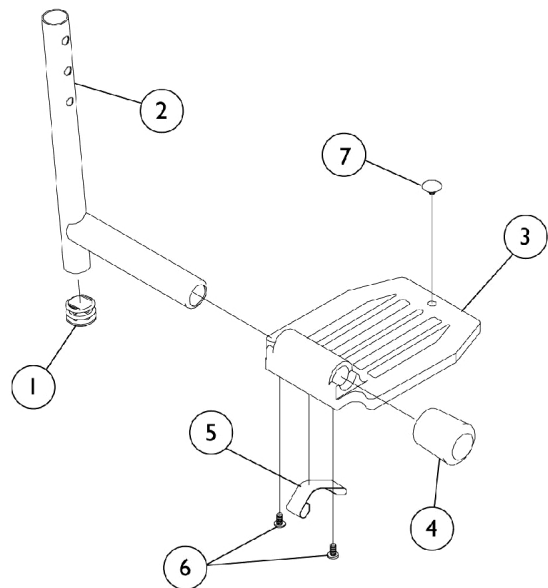

PW93 and PW93E

Pivot & Slide Tube w/ Footplate

P93M, PW93, and PW93E

Note: Heel Loops are not included. See footrest and legrest accessories.

Note: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	A	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	B	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	C	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
2	D	1070762-NLA	NLA - Tube, Pivot/Slide w/ Plug Button, Deluxe Aluminum, Black	1	1
3	E	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3	F	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	F	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3	F	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	E	1002343	Footplate, Aluminum, Extra Large, Black - Right (7" W x 8-1/2" L)	1	1
3	E	1002344	Footplate, Aluminum, Extra Large, Black - Left (7" W x 8-1/2" L)	1	1
3	F	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3	F	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	G	1016346	Cap, End, Tube, Black (7/8")	1	1
4	H	1027095	Plug Button, Black (7/8")	1	1
5	G	1006981	Spring, Tension	1	2
5		1036120-NLA	NLA - Spring	1	2
5	H,I	1036123	Spring, Leaf	1	1
6		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Standard Aluminum (1650) and Extra Large Aluminum (1350) footplates for P93M
 B - Standard Aluminum (1650) and Extra Large Aluminum (1350) footplates for PW93 and PW93E
 C - Deluxe Aluminum (1450) footplate for P93M
 D - Deluxe Aluminum (1450) footplate for PW93 and PW93E
 E - 14" Wide
 F - 16-20" Wide
 G - Standard Aluminum Footplate
 H - Extra Large Aluminum Footplate
 I - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.