
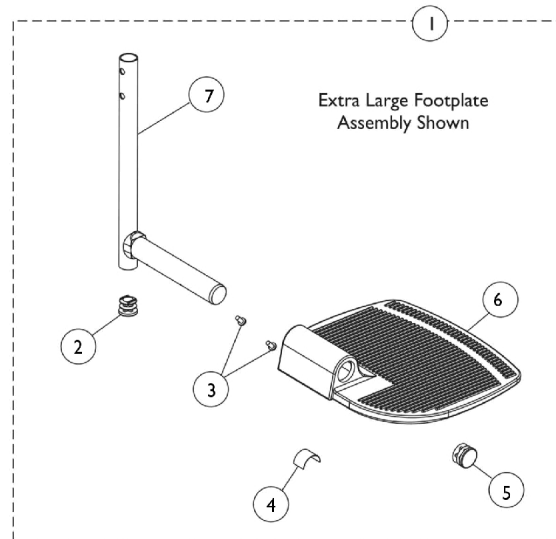


Pivot/Slide Tube w/ Footplate - T93H Series - Veranda Only

NOTE: Pivot/Slide Tube and Footplate is attached to Footrest Support using the spring button or bolt thru hardware. See chart titled, "Pivot Slide Tube Attaching Hardware."

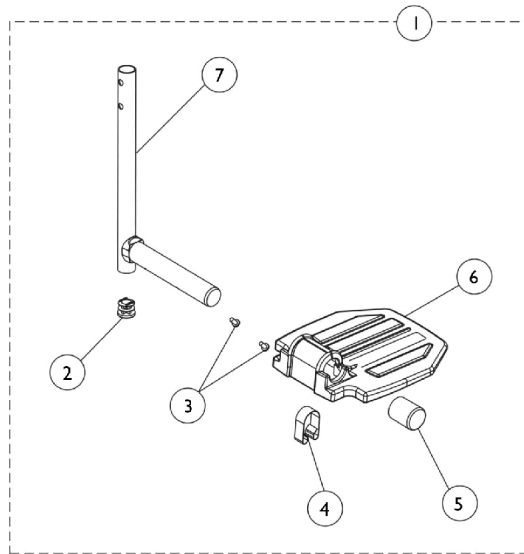

Pivot/Slide Tube w/ Footplate - T93H Series - Veranda Only

NOTE: Pivot/Slide Tube and Footplate is attached to Footrest Support using the spring button or bolt thru hardware. See chart titled, "Pivot Slide Tube Attaching Hardware."

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	Item #1 Pivot Slide Tube with Footplate	1	1
2		1111277	Pivot/Slide Tube w/ Plug Button, Footrest, Black	1	1
3		1027097	Plug Button, Black (3/4")	1	1
4		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
5		1006981	Spring, Tension	1	1
5	B,D	1036123	Spring, Leaf	1	1
6		1016346	Cap, End, Tube, Black (7/8")	1	1
6	B	1027095	Plug Button, Black (7/8")	1	1
7		1015178	Footplate, Composite, Black (6" W x 7-53/64" L)	1	1
7		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
7		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
7	B	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
7	B	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
	C	See Chart	Pivot Slide Tube Attaching Hardware	1	1
NOTE: A - Includes items 2-7. B - For use on extra large footplate. C - Not Shown. D - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.					

Pivot/Slide Tube w/ Footplate - T93H Series - Veranda Only

NOTE: Pivot/Slide Tube and Footplate is attached to Footrest Support using the spring button or bolt thru hardware. See chart titled, "Pivot Slide Tube Attaching Hardware."

Footrest Pivot Slide Tube with Footplate

Model Number	Chair Width	Footplate Style	Frame Side	Standard	2" Longer	4" Longer
T93HSA T93HA - Black	14-15"	Aluminium	Right	1110361	N/A	N/A
T93HSA T93HA - Black	14-15"	Aluminium	Left	1110362	N/A	N/A
T93HA T93HAB - Black	16-22"	Aluminium	Right	1111285	N/A	N/A
T93HA T93HAB - Black	16-22"	Aluminium	Left	1111286	N/A	N/A
T93HC T93HCB - Black	16-22"	Composite	Right	1111289	N/A	N/A
T93HC T93HCB - Black	16-22"	Composite	Left	1111290	N/A	N/A
T93HA - Black	16-22"	Extra Large	Right	1110344	N/A	N/A
T93HA - Black	16-22"	Extra Large	Left	1110345	N/A	N/A

Pivot Slide Tube Attaching Hardware

Style	Part Number	Quantity per Package	Quantity Per Footrest	Quantity Per Legrest
Spring Button Hardware Kit	1188335 See Note(s) A	2	2	4
Bolt Through Hardware Kit	1101776 See Note(s) A	2	2	2
Tool Kit	1058407 See Note(s) B	1	N/A	N/A

NOTE: A - Kit includes 2 ea. Double Spring Buttons.
B - Kit includes allen wrench, key, and open end wrench.