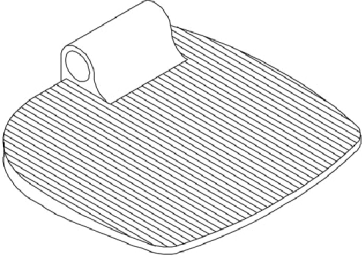
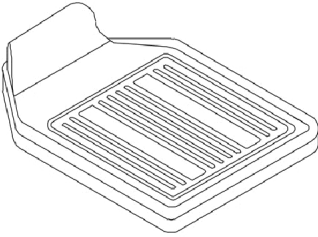
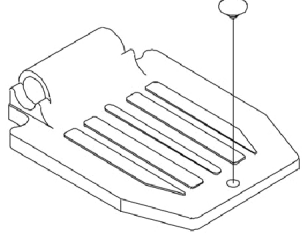



# Pivot Tube w/ Footplate Assemblies

## 1650, 1350, 1450

### PHW93, 70NHD and AT5544

Heel loops are not included. See footrest and legrest accessories.  
For complete pivot/slide tube and footplate assemblies, see front riggings reference chart.

		
1350 Extra large	1450 Deluxe	1650 Standard Aluminum

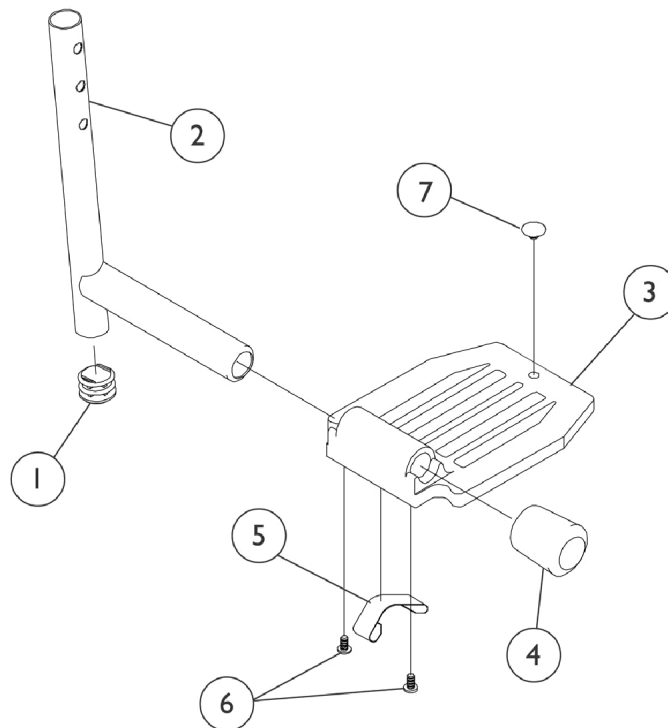

# Pivot Tube w/ Footplate Assemblies

## 1650, 1350, 1450

### PHW93, 70NHD and AT5544

Heel loops are not included. See footrest and legrest accessories.

For complete pivot/slide tube and footplate assemblies, see front riggings reference chart.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	A,C	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A,C	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	A,C	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	B	1070762-NLA	NLA - Tube, Pivot/Slide w/ Plug Button, Deluxe Aluminum, Black	1	1
3	A	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	B	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3	B	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	C	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3	C	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	A	1016346	Cap, End, Tube, Black (7/8")	1	1
4	C	1027095	Plug Button, Black (7/8")	1	1
5	A,D	1006981	Spring, Tension	1	2
5	B,D	1036120-NLA	NLA - Spring	1	2
5	C,F	1036123	Spring, Leaf	1	1
6	E	40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Standard Aluminum (1650)  
 B - Deluxe Aluminum (1450)  
 C - Extra Large Aluminum (1350)  
 D - Two per side  
 E - Deluxe takes one  
 F - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.