
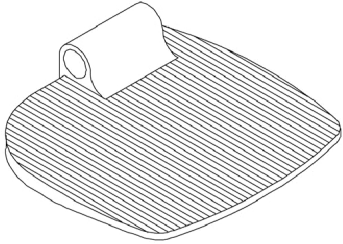
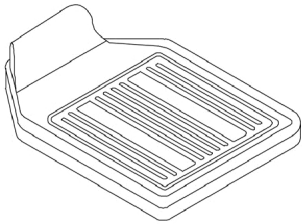
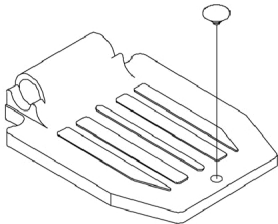


# Pivot Tube w/ Footplate Assemblies - 1650, 1350 and 1450 PH904A and PHAL4A

Heel loops are not included. See footrest and legrest accessories.  
For complete pivot/slide tube and footplate assemblies, see front riggings reference chart.

		
1350 Extra large	1450 Deluxe	1650 Standard Aluminum

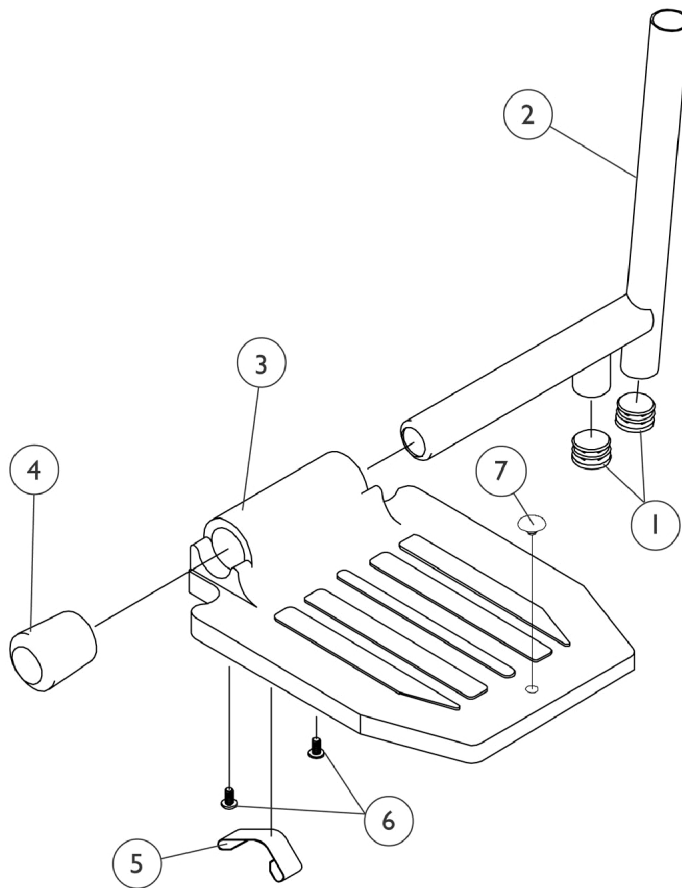

# Pivot Tube w/ Footplate Assemblies - 1650, 1350 and 1450 PH904A and PHAL4A

Heel loops are not included. See footrest and legrest accessories.

For complete pivot/slide tube and footplate assemblies, see front riggings reference chart.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2	D,A,B	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	D,A,B	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	D,A,B	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	E,A,B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C,D	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
2	C,E,H	1023804-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Chrome	1	1
3	A	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	C	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3	C	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	B	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3	B	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	A	1016346	Cap, End, Tube, Black (7/8")	1	1
4	B	1027095	Plug Button, Black (7/8")	1	1
5	A	1006981	Spring, Tension	1	1
5	C,F	1036120-NLA	NLA - Spring	1	1
5	I,B	1036123	Spring, Leaf	1	1
6	G	40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Standard Aluminum (1650)  
 B - Extra Large Aluminum (1350)  
 C - Deluxe Aluminum (1450)  
 D - PH904A  
 E - PHAL4A  
 F - Two per side  
 G - Deluxe takes one  
 H - Part #1023804 is no longer available as a service part individually. See category titled "Front Riggings - Reference Chart - (Hemi) - PHAL4A" and order part #1023924 or 1023925 Pivot/Slide Tube and Footplate Assembly (Deluxe Aluminum - 1450).  
 I - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.