
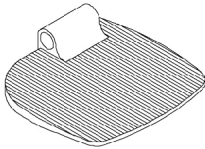
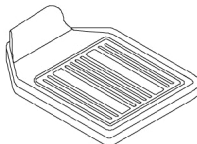
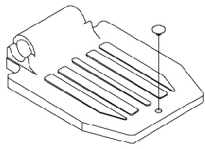
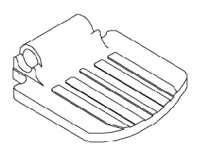


Pivot/Slide Tube and 1650, 1450, 1350 Aluminum Footplates

NOTE: Heel Loops are not included. See footrest and legrest accessories

NOTE: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

 <p>1350 Extra Large</p>	 <p>1450 Deluxe Aluminum</p>
 <p>16"-24" Wide 1650 Standard Alum.</p>	 <p>14" Wide 1650 Standard Alum.</p>

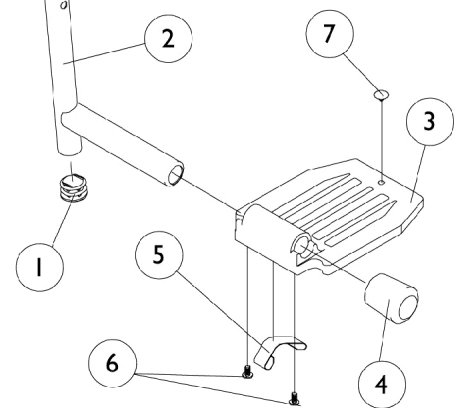

Pivot/Slide Tube and 1650, 1450, 1350 Aluminum Footplates

NOTE: Heel Loops are not included. See footrest and legrest accessories

NOTE: For complete pivot/slide tube and footplate assemblies, see front riggings reference charts.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	A	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	A	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	D	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	D	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	D	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	E	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	F	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	G	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
2	H	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
3		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	I	1016346	Cap, End, Tube, Black (7/8")	1	1
4	J	1027095	Plug Button, Black (7/8")	1	1
5	I	1006981	Spring, Tension	1	2
5	L	1036120-NLA	NLA - Spring	1	2
5	J,M	1036123	Spring, Leaf	1	1
6	K	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Extra Large footplate for 60MFX, AT5544, 60, 70, PW93, PW93E
 B - Extra Large footplate for PAL4A
 C - Extra Large footplate for P93M
 D - Extra Large and Standard Aluminum footplate for P904
 E - Standard Aluminum footplate for P93M
 F - Standard Aluminum footplate for PAL4
 G - Deluxe Aluminum footplate for P904 and AT5544
 H - Deluxe Aluminum footplate for P93M
 I - For Standard Aluminum footplate
 J - For Extra Large footplate
 K - Deluxe footplate takes 1 ea.
 L - Deluxe footplate
 M - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.