


Footplates & Heel Loops w/ Extension - FR75 Legrest Support Assemblies

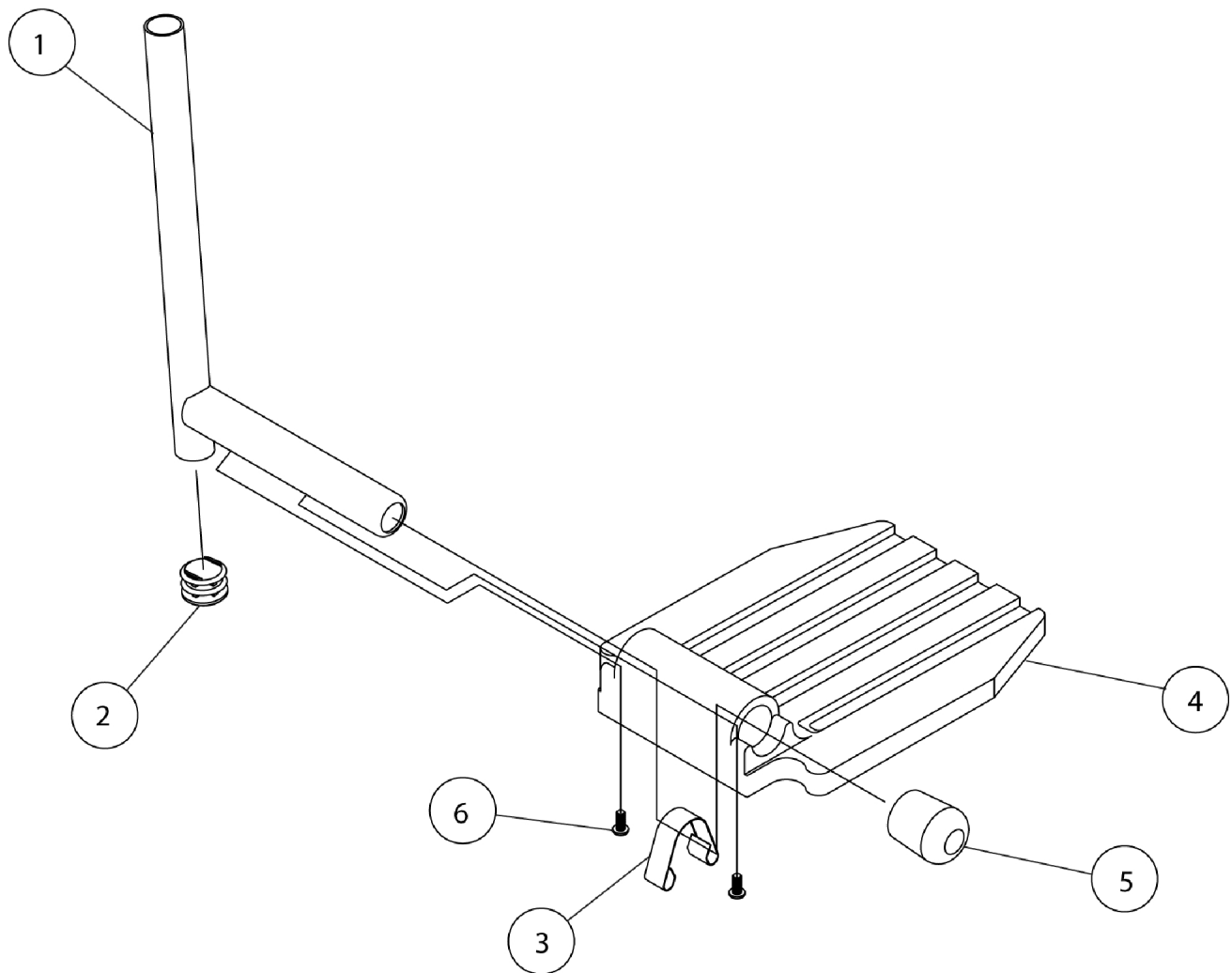

Footplates & Heel Loops w/ Extension - FR75 Legrest Support Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-6	A B,C	7516R	Footplate w/Pivot Slide Tube - RH (16")	--	1
1-6		7516L	Footplate w/Pivot Slide Tube - LH (16")	--	1
1-6		7518R	Footplate w/Pivot Slide Tube - RH (18")	--	1
1-6		7518L	Footplate w/Pivot Slide Tube - LH (18")	--	1
1		1041527	Pivot/Slide Tube w/ Plug Button, Black	1	2
2		1027097	Plug Button, Black (3/4")	1	2
3		1006981	Spring, Tension	1	2
3		1036123	Spring, Leaf	1	2
4		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	2
4		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	2
4		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	2
4		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	2
5		1016346	Cap, End, Tube, Black (7/8")	1	2
6		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	2	4
NOTE: A - Small & Medium B - Extra Large C - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.					