


Front Riggings - Footplates & Heel Loops with Extension - FR77 Legrests Support Assemblies

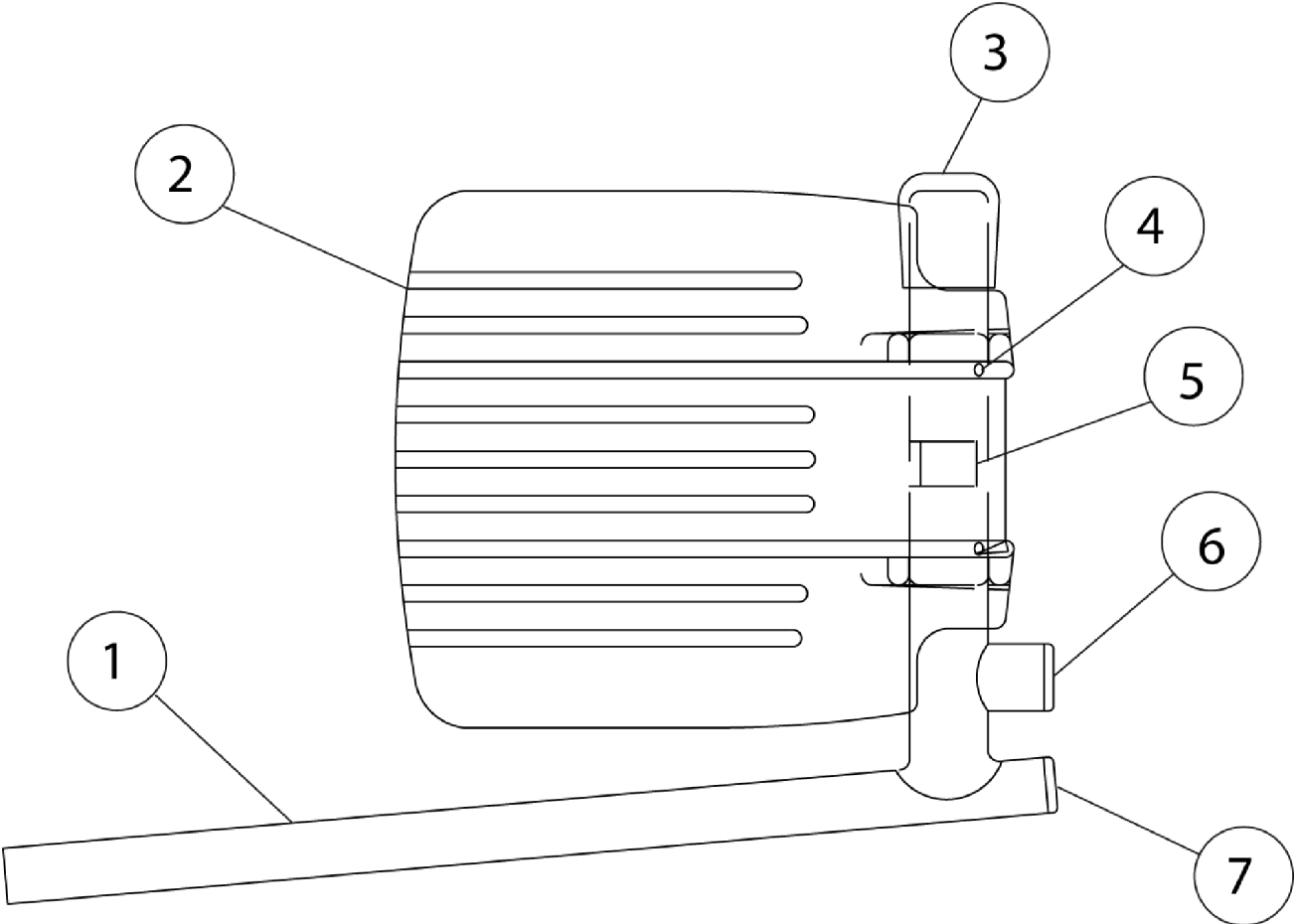

Front Riggings - Footplates & Heel Loops with Extension - FR77 Legrests Support Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-7		7716R	Footplate w/Pivot Slide Tube - RH (16")	--	1
1-7		7716L	Footplate w/Pivot Slide Tube - LH (16")	--	1
1-7		1023907	Footplate w/Pivot Slide Tube - RH (18")	--	1
1-7		1023908	Footplate w/Pivot Slide Tube - LH (18")	--	1
1		1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	--
2		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	--
2		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	--
3		1016346	Cap, End, Tube, Black (7/8")	1	--
4		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
5	B	1036123	Spring, Leaf	1	--
6	A	1027097	Plug Button, Black (3/4")	1	--
7		1018133	Plug Button, Black (5/8")	1	--

NOTE: A - Extra Large
B - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.