


Pivot Slide Tube with Footplate For 94BKR, 96BK, H94, H96 Assemblies (16" to 24" Chairs)

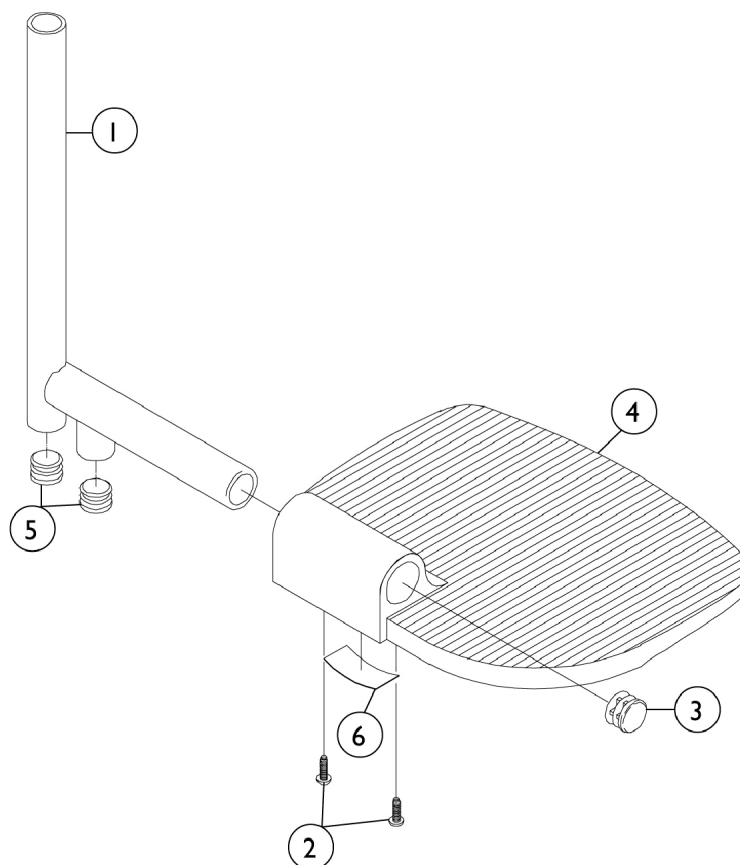

Pivot/Slide Tube and Footplate

Side	Standard	2" Longer	4" Longer
Right	1031934-NLA	1086537-NLA See Note(s) A	1086538-NLA See Note(s) A
Left	1031935-NLA	1086539-NLA See Note(s) A	1086540-NLA See Note(s) A
NOTE: A - Can only be used with Bolt Thru Footrest (T93HAB/HCB) and Elevating Legrest (T94HAB/HCB).			

Pivot Slide Tube with Footplate

For 94BKR, 96BK, H94, H96 Assemblies

(16" to 24" Chairs)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7500085-NLA	NLA - Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
1		1086447-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Chrome (2" Longer)	1	1
1		1086448-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
3		1027095	Plug Button, Black (7/8")	1	1
4	A	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
4	A	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
5		1027097	Plug Button, Black (3/4")	1	2
6	C	1036123	Spring, Leaf	1	1
	B	See Chart	Pivot Slide Tube with Footplate	1	1
NOTE: A - Wrinkle Black Finish B - Not Shown C - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.					