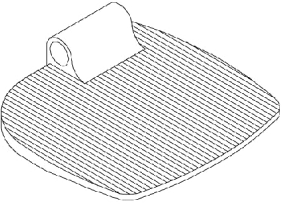
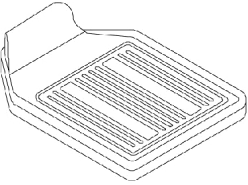
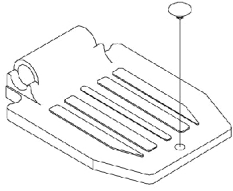
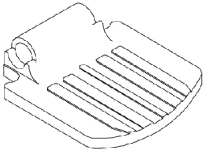



# Pivot/ Slide Tube and Footplate

## P93, P93E, P93M

NOTE: Heel loops are not included. See page titled "Footrest and Legrest Accessories".

NOTE: For complete pivot/slide tube and footplate assemblies, see page titled "Front Riggings Reference Chart".

			
1350 Extra large	1450 Deluxe	16" to 20" Wide	14" Wide

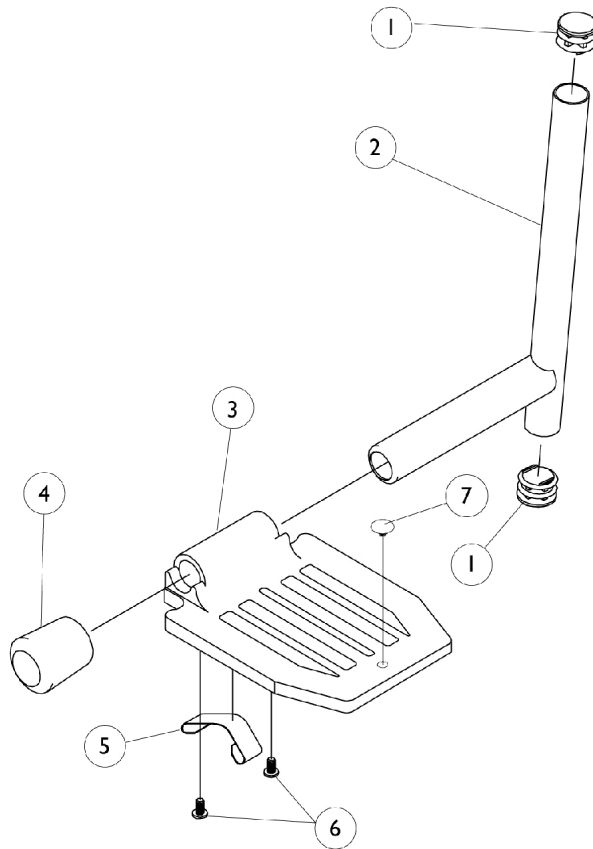

# Pivot/ Slide Tube and Footplate

## P93, P93E, P93M

**NOTE: Heel loops are not included. See page titled "Footrest and Legrest Accessories".**

**NOTE: For complete pivot/slide tube and footplate assemblies, see page titled "Front Riggings Reference Chart".**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	D,E,B	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	E	1086479-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	E	1086480-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2		1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
3	C	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3	A	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	C	1002343	Footplate, Aluminum, Extra Large, Black - Right (7" W x 8-1/2" L)	1	1
3	C	1002344	Footplate, Aluminum, Extra Large, Black - Left (7" W x 8-1/2" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	D	1016346	Cap, End, Tube, Black (7/8")	1	1
4	E	1027095	Plug Button, Black (7/8")	1	1
5	D	1006981	Spring, Tension	1	1
5	B	1036120-NLA	NLA - Spring	1	2
5	E,F	1036123	Spring, Leaf	1	1
6		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
7		1070168	Plug, Push, Footplate	1	1

**NOTE:** A - Aluminum footplates on chairs built before 2/96 had a "Bright aluminum" finish. Footplates after 1/96 now have a black finish. When replacing 1 Bright aluminum footplate, it is suggested that 2 new (black) footplates are ordered.  
 B - Deluxe Aluminium Footplate  
 C - 14" Wide  
 D - Standard Aluminium Footplate  
 E - Extra Large Aluminium Footplate  
 F - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.