
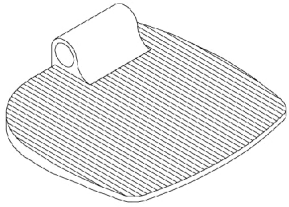
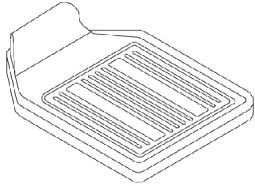
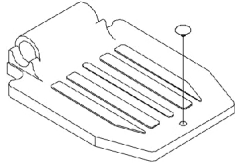


Pivot/ Slide Tube and Standard Aluminum, 1450, 1350 Footplates

		
<p>1350 Extra large</p>	<p>1450 Deluxe Aluminum</p>	<p>16" to 20" W Standard Aluminum</p>

For complete pivot/slide tube and footplates see Front Rigging Reference Chart page.

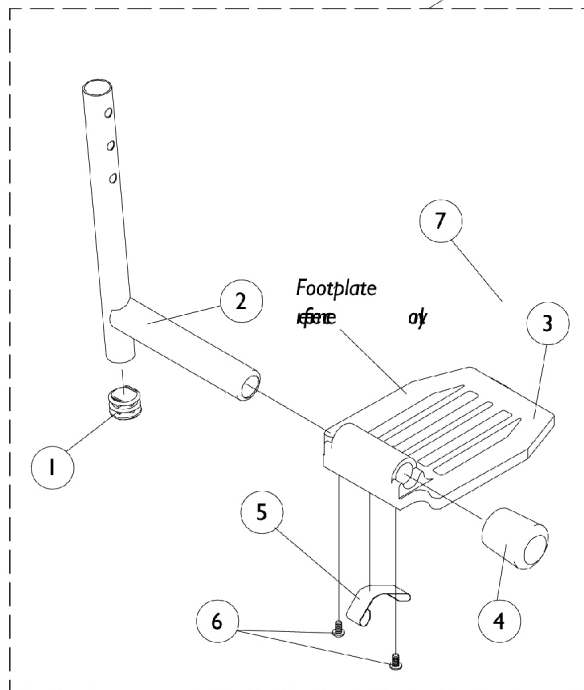

Pivot/ Slide Tube and Standard Aluminum, 1450, 1350 Footplates

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	A,B	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A,B	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	A,B	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	C,A	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	C,A	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	C,A	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D,A	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	E,A	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	F,B	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	F,B	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	F,B	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	C,F	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	C,F	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	C,F	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D,F	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	E,F	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	E,F	1086479-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	E,F	1086480-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	G,B	1070762-NLA	NLA - Tube, Pivot/Slide w/ Plug Button, Deluxe Aluminum, Black	1	1
2	C,G	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
2	E,G	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
3	A	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3	G	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3	G	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3	F	44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3	F	44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	A	1016346	Cap, End, Tube, Black (7/8")	1	1
4	F	1027095	Plug Button, Black (7/8")	1	1
5	H,A	1006981	Spring, Tension	1	2
5	G,H	1036120-NLA	NLA - Spring	1	2
5	F,J	1036123	Spring, Leaf	1	1
6	I	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
7		1070168	Plug, Push, Footplate	1	1

Pivot/ Slide Tube and Standard Aluminum, 1450, 1350 Footplates

NOTE: A - Standard Aluminium Footplate.
B - PW93 and PW93E.
C - P904.
D - PAL4.
E - P93M.
F - Extra Large Aluminium Footplates.
G - Deluxe Aluminium Footplates.
H - Takes 2 Per Side.
I - Deluxe Takes 1 Per Side.
J - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.