
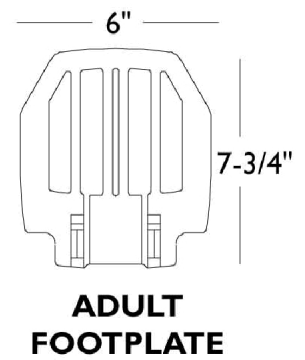


A1562/A1564 - Aluminum Footplate with Pivot Slidetube

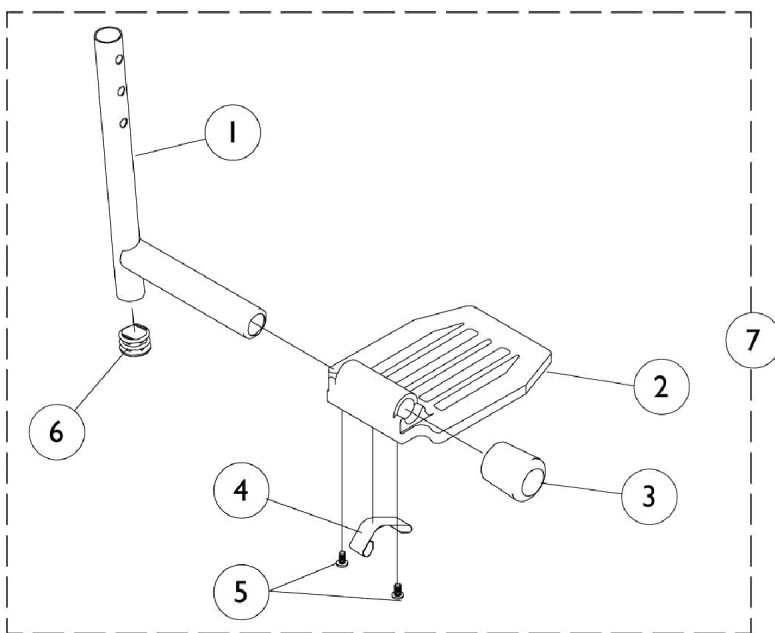

A1562/A1564 - Aluminum Footplate with Pivot Slidetube

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
1	A	1086481-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
1	A	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
2		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
2		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
2		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
3		1027095	Plug Button, Black (7/8")	1	1
4	B	1006981	Spring, Tension	1	1
4	C,D	1036123	Spring, Leaf	1	1
5		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
6		1027097	Plug Button, Black (3/4")	1	1
7		See Chart	60° - Pivot Slide Tube w/ Aluminum Footplate	1	1
7		See Chart	70° - Pivot Slide Tube & Aluminum Footplate	1	1

NOTE: A - Includes Items 3 and 6
 B - Use with Standard Footplates.
 C - Use with Extra Large Footplates.
 D - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.

Pivot Slide Tube w/ Aluminum Footplate - See Chart.

7 - Pivot Slidetube with Aluminum Footplate

60° / Elevating

Chair Width	Small Adult Footplate 6"x6"	Medium Adult Footplate 6"x7"	X-Large Adult Footplate 8"x8-1/2"	X-Large 2" Longer	X-Large 4" Longer
14" Right	1041542-NLA	--	--	--	--
14" Left	1041543-NLA	--	--	--	--
16-20" Right	--	1026401	1026395	1086515	1086516
16-20" Left	--	1026402	1026396	1086517	1086518

70° Tapered

Chair Width / Side	Small Adult Footplate 6 x 6"	Medium Adult Footplate 6 x 7"
14" Right	1041542-NLA	--
14" Left	1041543-NLA	--
16-20" Right	--	1026401
16-20" Left	--	1026402

A1562/A1564 - Aluminum Footplate with Pivot Slidetube