




# Footplate, Pivot Tube and Hardware - A1562/A1564 Aluminum

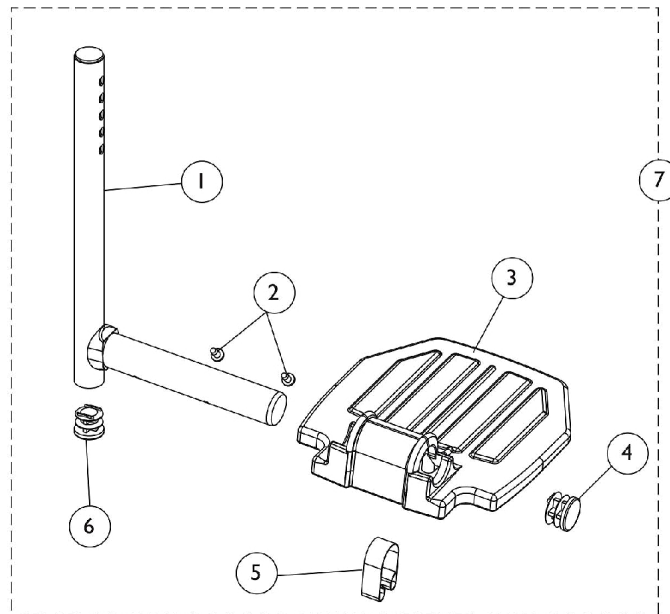

**All Footrests and AT5044 Legrests  
w/ Small / Medium / Extra Large Footplate**

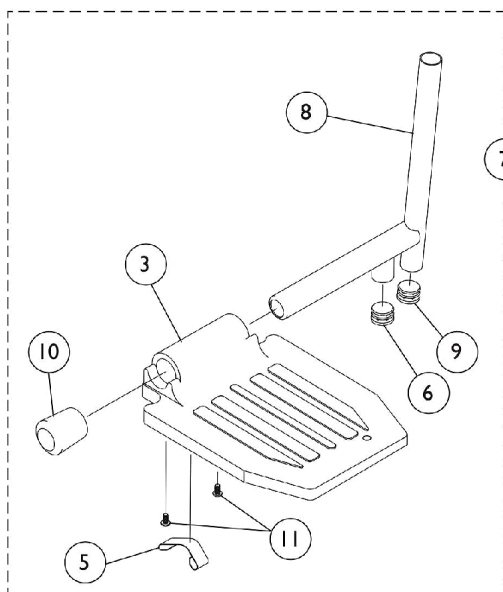

**ATL4A Articulating Legrests  
w/ Small / Medium Footplate**

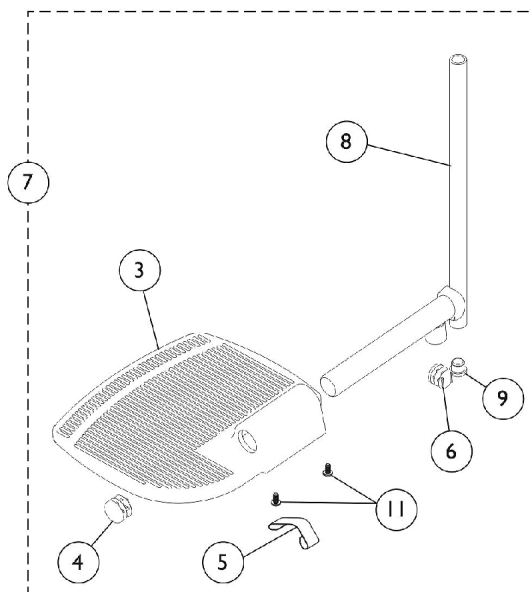

**ATL4A Articulating Legrests  
w/ Extra Large Footplate**

## Footplate, Pivot Tube and Hardware - A1562/A1564 Aluminum

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1120330	Pivot/Slide Tube w/ Plug, Black	1	1
2		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
3		44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
3		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4		1027095	Plug Button, Black (7/8")	1	1
5	A	1006981	Spring, Tension	1	1
5	D,B	1036123	Spring, Leaf	1	1
6		1027097	Plug Button, Black (3/4")	1	1
7		See Chart	#7 Aluminum Footplate w/ Pivot Tube Assembly	1	1
8		1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
9		1018133	Plug Button, Black (5/8")	1	1
10		1016346	Cap, End, Tube, Black (7/8")	1	1
11		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
	C	1074568	Package, Push Plug, Footplate	2	1
NOTE: A - Use with Small / Medium Footplates. B - Use with Extra Large Footplates. C - Not Shown. D - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.					

### #7 Aluminum Footplate w/ Pivot Tube Assembly

Description	Footplate Size	Chair Width	Right	Left
All Footrests	Extra Small	12-13"	1120333	1120334
All Footrests & AT5044 Legrest	Small	14-15"	1120335	1120336
All Footrests & AT5044 Legrest	Medium	16-22"	1120337	1120338
All Footrests & AT5044 Legrest	Extra Large	16-22"	1120339	1120340
ATL4A Articulating Legrest	Small	14-16"	1023905-NLA	1023906-NLA
ATL4A Articulating Legrest	Medium	17-22"	1023907	1023908
ATL4A Articulating Legrest	Extra Large	16-22"	1035456	1035457