
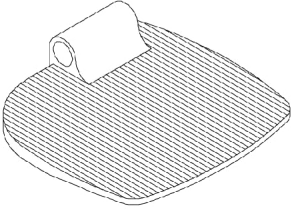
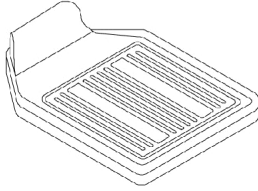
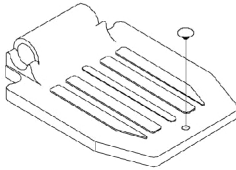
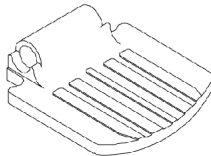


# Pivot/ Slide Tube and Standard Aluminum, 1450, 1350 Footplates

NOTE: Heel loops are not included. See footrest and legrest accessories.

NOTE: For complete pivot/slide tube footplate assemblies, see front riggings reference charts.

			
<p>1350 Extra Large</p>	<p>1450 Deluxe Aluminum</p>	<p>16" to 24" Wide Standard Alum.</p>	<p>14" Wide Standard Alum.</p>

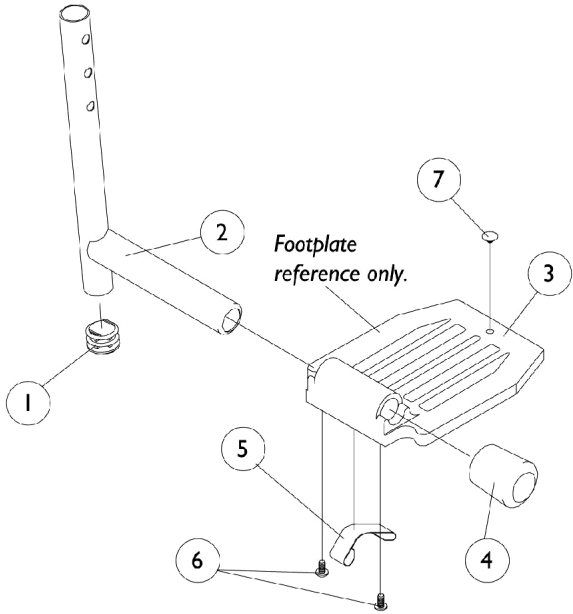

# Pivot/ Slide Tube and Standard Aluminum, 1450, 1350 Footplates

**NOTE: Heel loops are not included. See footrest and legrest accessories.**

**NOTE: For complete pivot/slide tube footplate assemblies, see front riggings reference charts.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	H	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	H	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	H	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	E,I	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	E,J	1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	K	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	K	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	C,E,K	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D,K,L	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
2	J	1041525-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe Aluminum, Black	1	1
3	C,A	44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
3	C,B	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4	C	1016346	Cap, End, Tube, Black (7/8")	1	1
4	E	1027095	Plug Button, Black (7/8")	1	1
5	C,F	1006981	Spring, Tension	1	2
5	D,F	1036120-NLA	NLA - Spring	1	2
5	E,M	1036123	Spring, Leaf	1	1
6	G	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	1	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - 14"-15" wide chairs.  
 B - 16" & wider.  
 C - Use with Standard Aluminium Footplate.  
 D - Use with Deluxe Footplate.  
 E - Use with Extra Large Footplate.  
 F - Deluxe (Takes 2 Per Side)  
 G - Deluxe (Takes 1)  
 H - Use with 60MFX, AT5544, 60, 70NHD, PW93, and PW93E.  
 I - Use with PAL4.  
 J - Use with P93M.  
 K - Use with P904.  
 L - Use with ATT5544.  
 M - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.