


Genius Legrest (Mechanical)

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1109613) when ordering replacement Calf Pads.

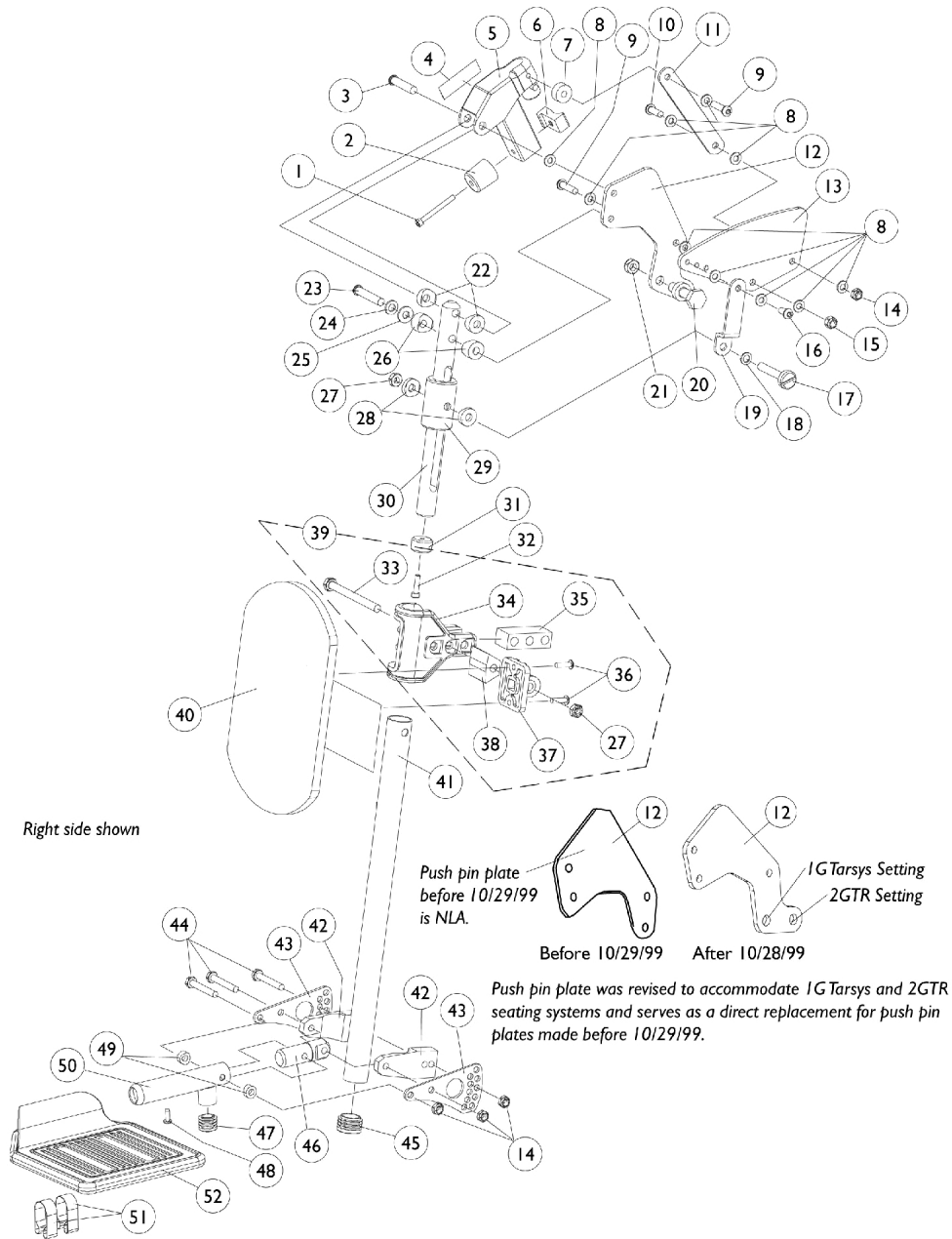

Genius Legrest (Mechanical)

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1109613) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1023966	Screw, Socket Head (#10-32 x 1-3/4")	1	1
2		0060000-NLA	NLA - Bumper, Rubber	1	1
3		0140406-NLA	NLA - Screw, Socket Head, Cap (5/16-18 x 1-1/2")	1	1
4		0061400-NLA	NLA - Decal, Warning Label Tie Down	1	1
5		0289824-NLA	NLA - Legrest Hanger Receptical w/ Dowel Pin Assembly, Black	1	1
6		1096649-NLA	NLA - Guide, Square w/ Brass Barb Insert	1	1
7		0176700-NLA	NLA - Bushing, Coved Latch Legrest Genius	1	1
8		0050100-NLA	NLA - Washer, Nylon Flat (1/4 x 1/2 x 1/16")	1	10
9		0051606-NLA	NLA - Screw, Cap, Button Head, Socket (1/4-20 x 1")	1	2
10		0052906-NLA	NLA - Screw, Socket Button Head (1/4-20 x 3/4")	1	3
11		0177802-NLA	NLA - Link, Black - Rear	1	1
12		1086987-NLA	NLA - Plate, Push Pin	1	1
13		0179202-NLA	NLA - Plate, Link	1	1
14		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
15		7930333	Locknut (1/4-20)	25	1
16		0223506-NLA	NLA - Screw, Socket, Cap, Button Head (1/4-20 x 1/2")	1	1
17		0180506-NLA	NLA - Screw, Slotted, Pan Head (5/16-18 x 1-15/16")	1	1
18		0054700-NLA	NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	2
19		0177702-NLA	NLA - Link, Black - Front	1	1
20		0182406-NLA	NLA - Pin, Push	1	1
21		1028486	Locknut, Low Profile (3/8-16)	1	1
22		0175600	Spacer, Coved (5/16 x 3/4 x 3/8")	1	2
23		0055206-NLA	NLA - Screw, Cap, Button Head, Socket (5/16-18 x 1-3/4")	1	1
24		1072692-NLA	NLA - Lockwasher, Split (5/16")	1	1
25		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
26		0176422-NLA	NLA - Spacer, Coved (5/16 x 3/4 x 3/8")	1	2
27		0055006-NLA	NLA - Locknut, Low Profile (5/16-18)	1	2
28		0176200-NLA	NLA - Bushing, Nylon Guide	1	2
29		0177100-NLA	NLA - Bushing, Legrest Support	1	1
30		0178224-NLA	NLA - Bar, Insert, Black	1	1
31		0175800-NLA	NLA - Bushing, Guide	1	1
32		0224600-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 3/4")	1	1
39	B	1109613	NLA - Kit, Calf Pad Mounting, Retro-Fit	1	1
27			NLA - Locknut, Low Profile (5/16-18)	1	1
33			NLA - Screw, Hex Head (5/16-18 x 3-1/2)	1	1
34			NLA - Clamp, Calf Pad	1	1
35			NLA - Block, Legrest Pad Mount	1	1
36			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
37			Plate, Legrest Pad	1	1
38			NLA - Block, Mounting	1	1
36			Screw, Phillips Pan Head (#10-32 x 1")	1	2

Genius Legrest (Mechanical)

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1109613) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
			Instruction Sheet, Legrest Pad Mtg.	1	1
40	A,F	8881193555	Calf Pad Assembly, Black Base (Pair)	2	1
41		0238124-NLA	NLA - Tube, Legrest (18")	1	1
41		0232424-NLA	NLA - Tube, Legrest (20")	1	1
42		0177022-NLA	NLA - Clamp, Tube Half	1	2
43		0177602-NLA	NLA - Plate, Adjustment	1	2
44		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	3
45		1096664-NLA	NLA - Plug Button, Black (1")	1	1
46		0177222-NLA	NLA - Pivot, Footplate	1	1
47		1027097	Plug Button, Black (3/4")	1	1
48		0194906-NLA	NLA - Screw, Phillips Pan Head Tap (#8-32 x 1/2")	1	1
49	C	0176022-NLA	NLA - Spacer	1	2
49	D	0186000-NLA	NLA - Spacer, Wheel	1	2
50	C	0179824-NLA	NLA - Tube, Footplate, Black	1	1
50	D	0288824-NLA	NLA - Tube, Footplate, Black	1	1
51	C	1036120-NLA	NLA - Spring	1	2
51	D,G	1036123	Spring, Leaf	1	2
52	C	1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
52	C	1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
52		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
52		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
	E	See Chart	Genius Legrest Assembly	1	1

NOTE: A - Item 39 and 40 must be ordered together in pairs for Genius Legrest manufactured before 11/20/01.
 B - Includes Items 27 and 33 - 38 (Per Side).
 C - Standard Footplate with Rubber Edge.
 D - Used with Extra Large Footplate.
 E - Not Shown.
 F - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1109613) when ordering replacement Calf Pads.
 G - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.

Genius Legrest Assembly

Description	Standard Footplate	X-Large Footplate	Standard Footplate	X-Large Footplate
Legrest Tube	18" Long	18" Long	20" Long	20" Long
Width Range	16"-24"	16"-24"	16"-24"	16"-24"
Right Assembly	0196600-NLA	1072595-NLA	0264000-NLA	1072597-NLA
Left Assembly	0196500-NLA	1072594-NLA	0263900-NLA	1072596-NLA
Pair	0138100-NLA	1072904-NLA	0263800-NLA	1072905-NLA