
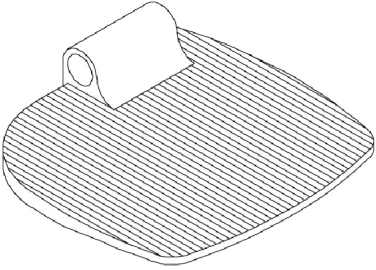
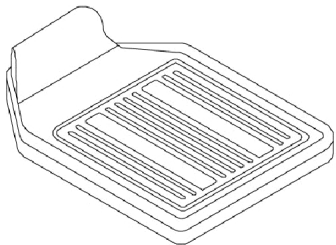
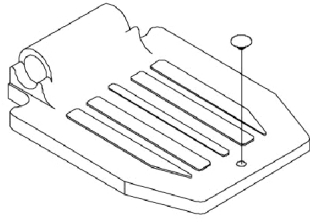


Pivot/ Slide Tube and 1650, 1450, 1350 Footplates

NOTE: Heel Loops are not included. See footreset and legrest accessories.

NOTE: For complete pivot/slide tube and footplate assemblies, see front riggings reference chart.

		
1350 Extra Large	1450 Deluxe Aluminum	1650 Standard Alum.

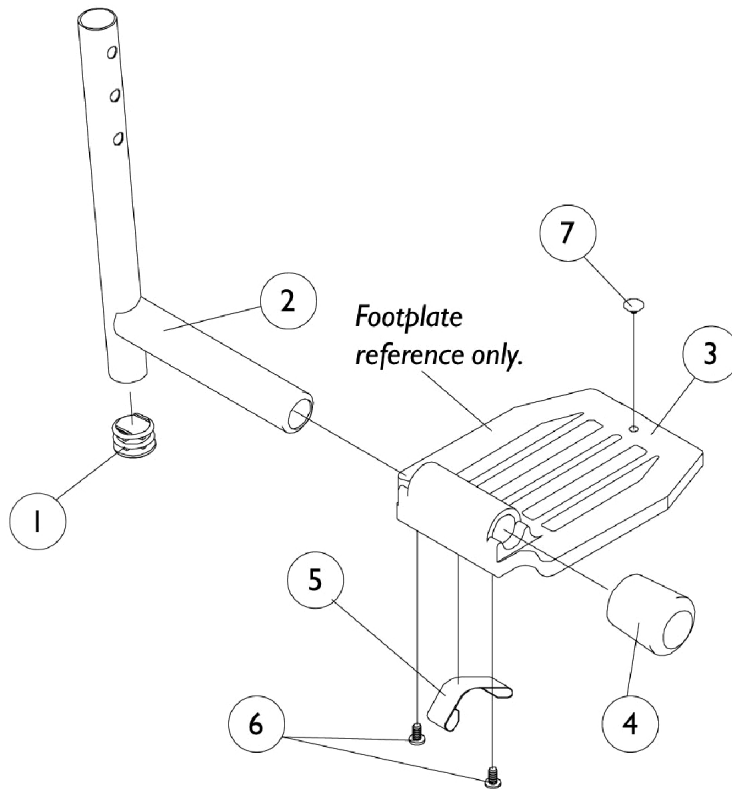

Pivot/ Slide Tube and 1650, 1450, 1350 Footplates

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	1
2	A	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	A	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	A	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	B	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	C	1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2	C	1100569-NLA	NLA - Pivot/Slide Tube Assembly, Extra Large, Standard Aluminium - 2" Longer, Black	1	1
2	C	1100570-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	D	1041526	Pivot/Slide Tube w/ Plug Button Assembly, Chrome	1	1
2	E	1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
2	E	1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
2	E	1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
2	F	1070762-NLA	NLA - Tube, Pivot/Slide w/ Plug Button, Deluxe Aluminum, Black	1	1
2	G	1100571-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Deluxe, Black	1	1
3	H	44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
3		1043659-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Right (8" W x 5-3/4" L)	1	1
3		1043660-NLA	NLA - Footplate, Aluminum, Deluxe, Black - Left (8" W x 5-3/4" L)	1	1
3		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	1
3		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	1
4		1016346	Cap, End, Tube, Black (7/8")	1	1
4		1027095	Plug Button, Black (7/8")	1	1
5	I,K	1006981	Spring, Tension	1	2
5	I,L	1036120-NLA	NLA - Spring	1	2
5	M,N	1036123	Spring, Leaf	1	1
6	J	7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	2
7		1070168	Plug, Push, Footplate	1	1

NOTE: A - Extra Large (1350) footplate for PW93
 B - Extra Large (1350) footplate for PAL4
 C - Extra Large (1350) and Standard Aluminum (1650) footplates for P904
 D - Standard Aluminum (1650) footplate for PAL4
 E - Standard Aluminum (1650) footplate for PW93
 F - Deluxe Aluminum (1450) footplate for PW93
 G - Deluxe Aluminum (1450) footplate for P904
 H - 16" and Wider
 I - Takes two per side
 J - Deluxe footplate takes 1 ea.
 K - Standard Aluminum (1650) footplate
 L - Deluxe Aluminum (1450) footplate
 M - Extra Large (1350) footplate
 N - Use one or more leaf springs per assembly as required such that footplate may be moved by hand to the up-right position and remain in place.