




Van Seat Frame and Mounting Hardware

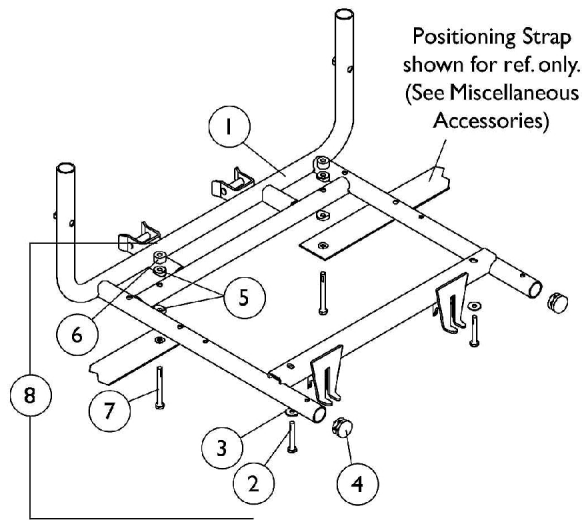

**Van Seat Frame
Before 5/16/02**

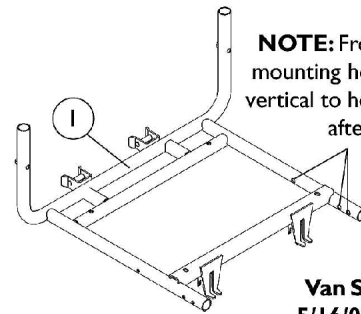


**Van Seat Frame
5/16/02 to 8/31/02**

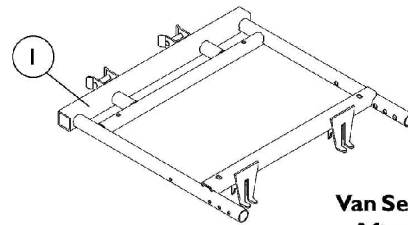

**Van Seat Frame
After 8/31/02**

For Van Seats without Solid Seat Base

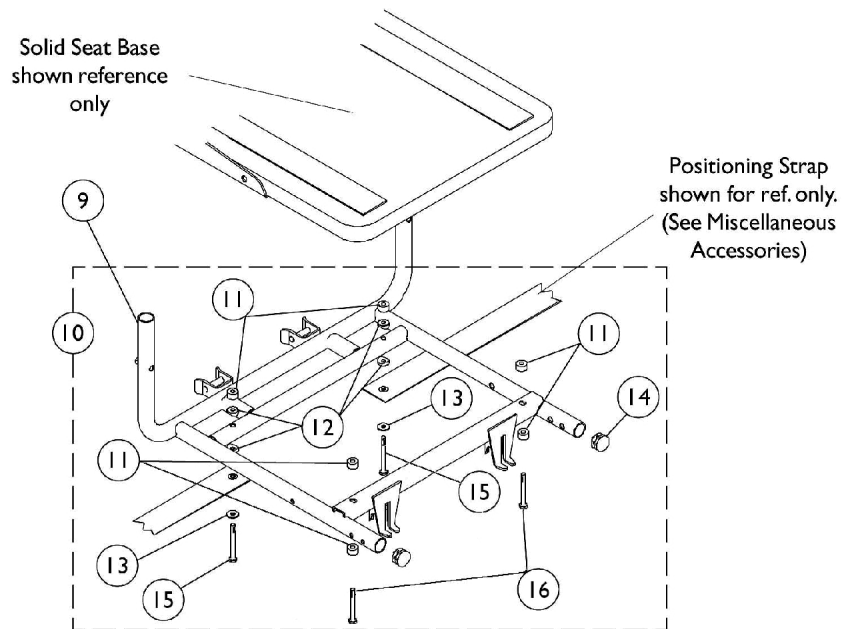

For Van Seats with Solid Seat Base

Van Seat Frame and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1123422-NLA	NLA - Frame, Van Seat, Black (16" W)	1	1
1	A,B	1108466-NLA	NLA - Frame, Van Seat, Black (18" W)	1	1
1	C	1114340-NLA	NLA - Frame, Van Seat, Black (18" W)	1	1
1	A,B	1108467-NLA	NLA - Frame, Van Seat, Black (20" W)	1	1
1	C	1114341-NLA	NLA - Frame, Van Seat, Black (20" W)	1	1
2		1088898-NLA	NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-3/4")	1	2
3		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
4		1029501	Plug Button, Black (1")	1	2
5		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
6		1108411-NLA	NLA - Spacer (1/4 x 5/8 x 3/8")	1	2
7		1109472-NLA	NLA - Screw, Hex Head w/Patch (1/4-20 x 2-1/2")	1	2
8	D	1109195-NLA	NLA - Kit, Van Seat Mounting Hardware	1	1
2			NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-3/4")	1	2
3			Washer (1/4 x 23/32 x 1/16")	1	4
4			Plug Button, Black (1")	1	2
5			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
6			NLA - Spacer (1/4 x 5/8 x 3/8")	1	2
7			NLA - Screw, Hex Head w/Patch (1/4-20 x 2-1/2")	1	2
9		1123422-NLA	NLA - Frame, Van Seat, Black (16" W)	1	1
9		1114340-NLA	NLA - Frame, Van Seat, Black (18" W)	1	1
9		1114341-NLA	NLA - Frame, Van Seat, Black (20" W)	1	1
10	E	1129720-NLA	NLA - Kit, Van Seat Mounting Hardware	1	1
11			NLA - Spacer (1/4 x 5/8 x 3/8")	1	6
12			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
13			Washer (1/4 x 23/32 x 1/16")	1	2
14			Plug Button, Black (1")	1	2
15			NLA - Screw, Hex Head w/Patch (1/4-20 x 2-1/2")	1	2
16			Screw, Hex Head Cap w/ Patch (1/4-20 x 2-1/4")	1	2
NOTE: A - When a new Van Seat Frame is ordered for chairs manufactured before 5/16/02 and use Front Riggings, you must order new Front Rigging Hanger Assemblies made 5/16/02 thru 8/31/02 as well. See Front Rigging Hanger Assembly page. B - Before 9/1/02 C - After 8/31/02 D - For Van Seats without Solid Base Seats. Includes items 2-7. E - For Van Seats with Solid Base Seats. Includes items 11-16.					
NOTE: Seat Height Adjustment Slide Tubes can be found with the Side Frames and Hardware.					