


Includes Labels, Shrouds, Battery, Controller, Actuator and Scale

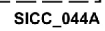

Model 1900S - Mast Assembly

Includes Labels, Shrouds, Battery, Controller, Actuator and Scale

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1147299-NLA	NLA - Shroud, Upper Left w/ Scale	1	1
2		1147298-NLA	NLA - Shroud, Upper Right w/o Scale	1	1
3		1147275	Bar, Lift Support, Mast	1	1
4		1078284	Bracket, Battery - Electric Only	1	1
5		1147293-NLA	NLA - Bracket, Bar, Cross	1	1
6		1078276	Battery Pack, Linak	1	2
7		1078275	Controller - Electric Only	1	1
8		1147306	Actuator, Mast	1	1
9		1147296	Roller, Mast, Outer	1	2
10		1090147-NLA	NLA - Decal, Invacare Medallion - Large	1	2
11		PAR9990-NLA	NLA - Label, Serial and Model Number	1	1
12		1111029-NLA	NLA - Decal, Capacity, Weight 350 lb. Canadian	1	1
13	A	1033800-NLA	NLA - Hook, 2" Adhesive Back	1	1
14	A	1026082-NLA	NLA - Fastener, Dual Lock	1	1
15		1147365-NLA	NLA - Screw, Hex Head (3/8-16 x 1-1/2")	1	2
16		1147364-NLA	NLA - Screw, Hex Head (3/8-16 x 7/8")	1	2
17		40363X000	Hex Nut (3/8"-16)	1	4
18		1078980	Screw, Phillips Pan Head (#10-24 x 1/4")	1	3
19		IHPAR9911	Screw, Phillips, Flat Head (#10-24 x 3/4")	1	6
20		60105597	Washer, #10 Countersunk Finishing	1	6
21		1147292	Plug, Threaded - Plastic	1	6
22		1147363	Screw, Shoulder, Socket (1/8 x 2-1/2")	1	2
23		1154071-NLA	NLA - Washer, Bellville, Separated (17/32 x 3/4 x 1/32")	1	2
24		1089061-NLA	NLA - Locknut (3/8-16)	1	2
25		1154073-NLA	NLA - Bracket, L Shaped, Actuator	1	1
26		1147369-NLA	NLA - Locknut, Hex (M4 x .7)	1	1
27		1147297-NLA	NLA - Strapping, Actuator	1	1
28	B,C	1159620	Kit, Digital Scale, 1900S	1	1
			NLA - Digital Scale Assembly, 1900S	1	1
			Instruction Sheet, Digital Scale	1	1
			NLA - Hook, 2" Adhesive Back	1	1
		1159621	Kit, Load Cell and Connector	1	1
			Load Cell and Connector Assembly	1	1
			Instruction Sheet, Digital Scale	1	1
29		1177604	Pendant, Two Channel w/ Hook	1	1
30		1079133	Cord, Power, USA	1	1
31	D	1078274	Charger, Electric Only w/o Power Cord	1	1

NOTE: A - Pieces must be cut from roll into 2" by 2" Squares (2 each).
B - Includes Digital Scale, Instruction Sheet, and Hook.
C - If Lift produced prior to 7/1/07 then order part number 1159621 Load Cell and Connector Kit.
D - Power Cord not included with Charger.