


Legrest Calf Pads and Ratchet Assemblies

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

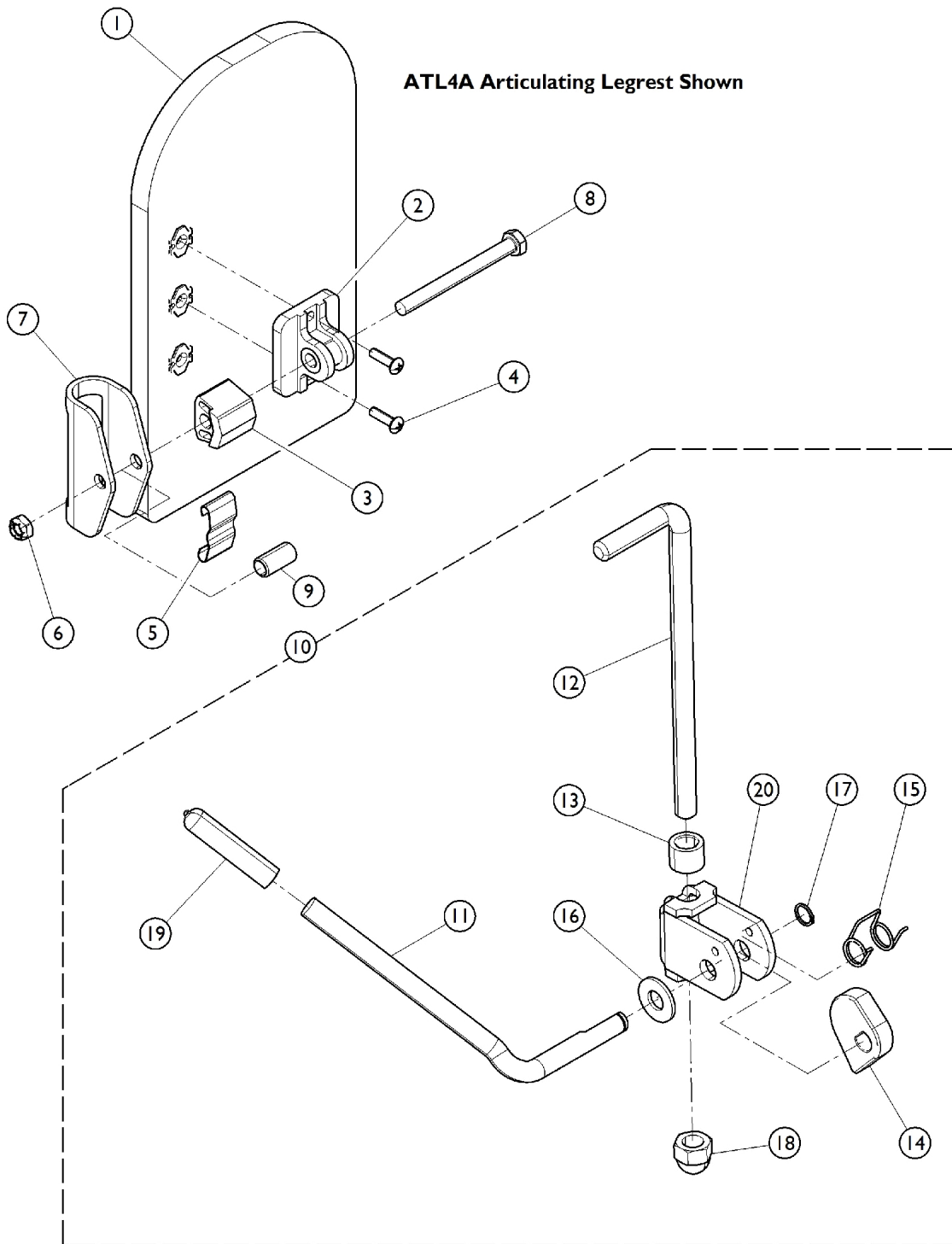

Legrest Calf Pads and Ratchet Assemblies

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	H,A,B	88804529X003	Pad, Legrest, Vinyl - Right	1	1
1	H,A,B	88804529X004	Pad, Legrest, Vinyl - Left	1	1
1	H,A,C	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
1	H,C	1192475	Pad, Legrest, Large - Right	1	1
1	H,C	1192476	Pad, Legrest, Large - Left	1	1
2		04526X019	Plate, Legrest Pad	1	1
3		1025133	Spacer, Legrest Pad, Black	1	1
4	G,I	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
4	G,J	1068649	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
5		48823X008	Spring, Leaf	1	1
6		1025730	Locknut (5/16-24)	1	1
7		1025104	Bracket, Legrest, Chrome	1	1
8		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
9		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
10	D,F	1046329	Kit, Elevating Legrest Ratchet Hardware (Pair)	1	1
11		1037126	Handle, Release	1	1
12		42326X004	Bar, Ratchet - Right/Left	1	1
13		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
14		1037125	Cam, Legrest	1	1
15		1037485	Spring, Release	1	1
16		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	2
17		1040831	Ring, Snap	1	1
18		10261X000	Nut, Acorn (5/16-24)	1	2
19		1040060	Tip, Plastic, Black	1	1
20		04528X002	Bracket, Release, Black	1	1
	E,K	1042798-NLA	NLA - Instruction Sheet, Ratchet Kit	1	1

NOTE: A - Specify upholstery color when ordering.
 B - 14 - 15" wide
 C - 16 - 24" wide
 D - Includes items 11 - 20.
 E - Not Shown
 F - Hardware quantity accommodates the right and left side
 G - Includes Screws and Instruction Sheet.
 H - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.
 I - Used with ATL4A Legrest.
 J - Used with AT5044LIFT and Standard Wedge Style Legrests.
 K - Item is no longer available from the Invcare Service Parts Department. Refer to the Invcare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.