
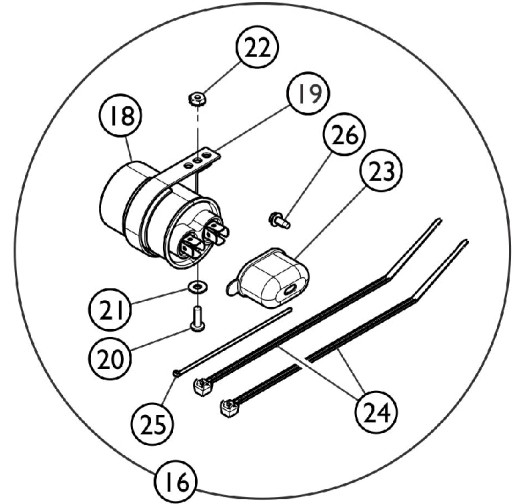



# Compressor and Mounting Hardware

## All Models Except Platinum 5


3LX (After 7/10/97)  
 5LX (After 4/21/97)  
 6LX (All Dates)

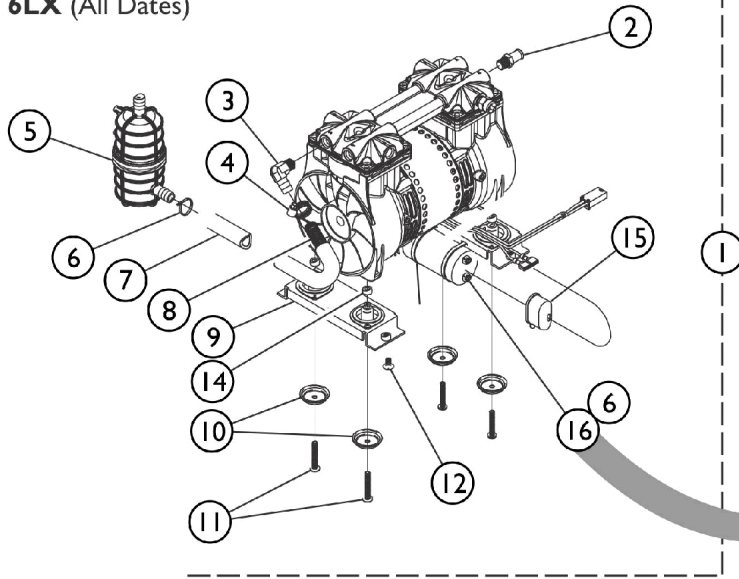

3LX (Before 7/11/97)

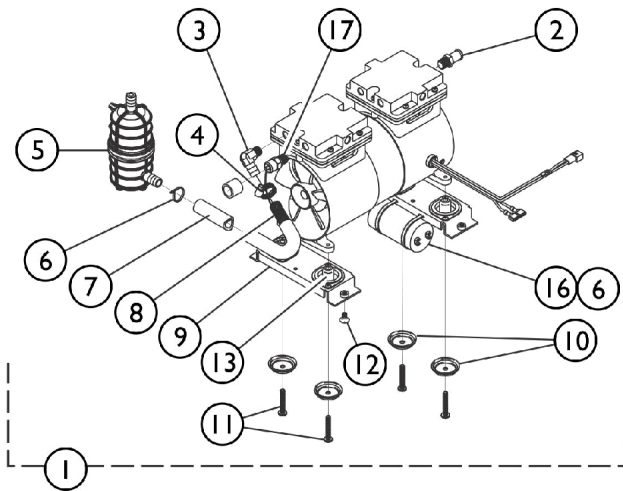

5LX (Before 4/22/97)

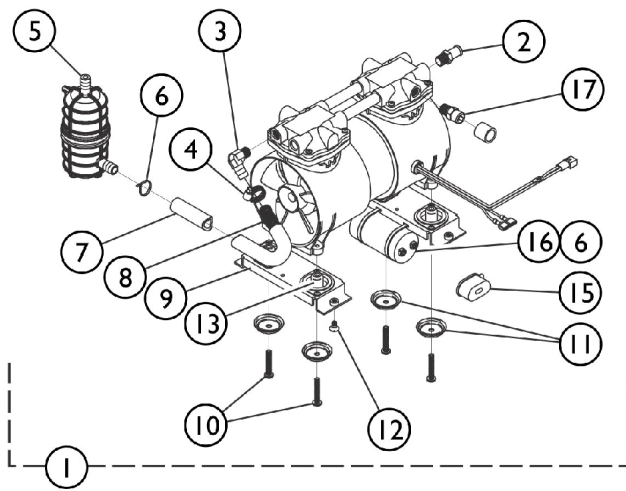

## Compressor and Mounting Hardware All Models Except Platinum 5

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1075071-NLA	NLA - Compressor Assembly - 6LX, 6LX02	1	1
1		1075071E-NLA	NLA - WARR RPLMT - Compressor Assembly (6LX, 6LX02)	1	1
	A,F	2000894-NLA	NLA - Kit, Compressor Rebuild	1	1
			Kit, Rebuild	1	1
			NLA - Instruction Sheet	1	1
	J	1157138	Kit, Compressor Complete Assembly - Thomas	1	1
			Compressor with Mounting Hardware - Thomas	1	1
			NLA - Tech Only - Capacitor, 15 MFD	1	1
			Tie Wrap (8" L)	2	1
		1186230	Kit, Compressor Restraint Bracket w/ Hardware	1	1
			Bracket, Restraint, Compressor	1	1
			NLA - Bumper, Bracket	1	1
			Bumper, Restraint Bracket, Compressor	1	1
			NLA - Screw, Phillips Hex Washer Head w/ Patch (1/4-20 x 3/4")	1	1
			Retainer, Push (1/4")	1	1
		1106690E-NLA	NLA - WARR RPLMT - Compressor Assembly - Thomas (5LX Plat.)	1	1
	A,G,I	1109235-NLA	Kit, Rebuild for Thomas Compressors	1	1
			Kit, Rebuild	1	1
			NLA - Instruction Sheet	1	1
	A,G	1154510	Kit, Compressor Rebuild w/ Trigon Seals & Instruction Sheet - IRC5LX/IRC5LXO2 (Thomas)	1	1
			Compressor Rebuild Kit w/ Trigon Seals - IRC5LX/IRC5LXO2 (Thomas)	1	1
			Instruction Sheet, Thomas Rebuild 2660-2650	1	1
	A,H	2000904-NLA	NLA - Kit, Compressor Rebuild	1	1
			NLA - Kit, Rebuild	1	1
			NLA - Instruction Sheet	1	1
2		1086187-NLA	NLA - Hose, Barbed, Patch Lock (1/2 x 1/4" NPT)	1	1
3	B	2001374-NLA	NLA - Fitting, Resonator	1	1
3	C	1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
4		P0876X026	Clamp, Hose	1	1
5	B	1095537-NLA	NLA - Kit, Resonator Retro-Fit	1	1
			Resonator	1	1
			NLA - Screw, Phillips Pan Head (#10-16 x 1/2")	1	2
			NLA - Tubing, Silicon (1/2 x 5")	1	1
			Tube, Braided (10 x 1/2")	1	1
			Spring, Extension (7/16 x 9-3/4" L)	1	1
			Screw, Socket Button Head (1/4-20 x 3/8")	1	1
			NLA - Bracket, Mounting	1	1
			Clip, Flange	1	1
			Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
			Tie Wrap (8" L)	1	1
			NLA - Instruction Sheet	1	1
5	C	1085873	Resonator	1	1

## Compressor and Mounting Hardware

### All Models Except Platinum 5

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
6		2001682	Tie Wrap (8" L)	1	3
7		2002264	Tubing, Reinforced Braided (1/2 x 26")	1	1
8		1088316	Spring, Extension (7/16 x 9-3/4" L)	1	1
9		2000888-NLA	NLA - Mounting Bracket Assembly	1	2
10		2000876-NLA	NLA - Washer, Cup	1	4
11		1105418-NLA	NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	4
12		1085696	NLA - Screw, Phillips Truss Head (1/4-20 x 1/2")	1	4
13		P0875X510	Mount, Diamond	1	4
14		2002906-NLA	NLA - Spacer (1/4 x 1/2 x 7/32")	1	4
15		P0875X560-NLA	NLA - Boot, Terminal	1	1
16	E	2000770-NLA	NLA - Capacitor - MFD3LX	1	1
16	D	2003019	Kit, Capacitor 15 MFD	1	1
18			NLA - Tech Only - Capacitor, 15 MFD	1	1
19			Clamp, Cap	1	1
20			Screw, Phillips Pan Head (8-32 x 1/2")	1	1
21			Washer, Flat (#8)	1	1
22			NLA - Nut, Keps (#6-32)	1	1
23			NLA - Boot, Terminal	1	1
24			Tie Wrap (8" L)	1	2
25			Tie Wrap (3-29/32" L)	1	1
26			Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	1
17	B	1085697-NLA	NLA - Relief Valve w/ Patch Lock, 35 PSIG	1	1

NOTE: A - Not Shown.  
 B - Before 7/00.  
 C - After 6/00.  
 D - Use with IRC5LX/ IRC6LX After 1/97 and IRC3LX After 6/97.  
 E - Use with IRC5LX/ IRC6LX Before 2/97 and IRC3LX Before 7/97.  
 F - Rebuild Kit is for Thomas Compressor Model #2618 and #2638. Compressor Model numbers are required when ordering the Rebuild Kit.  
 G - Rebuild Kit is for Thomas Compressor Model #2650 and #2660. Compressor Model numbers are required when ordering the Rebuild Kit.  
 H - Rebuild Kit is for Thomas Compressor Model #2619 and #2639. Compressor Model numbers are required when ordering the Rebuild Kit.  
 I - Part #1109235 is No Longer Available. Order part #1154510 as a replacement kit.  
 J - Includes Compressor, Capacitor, Tie Wraps, and 1186230.