


Compressor and Mounting Hardware Platinum 5 Models

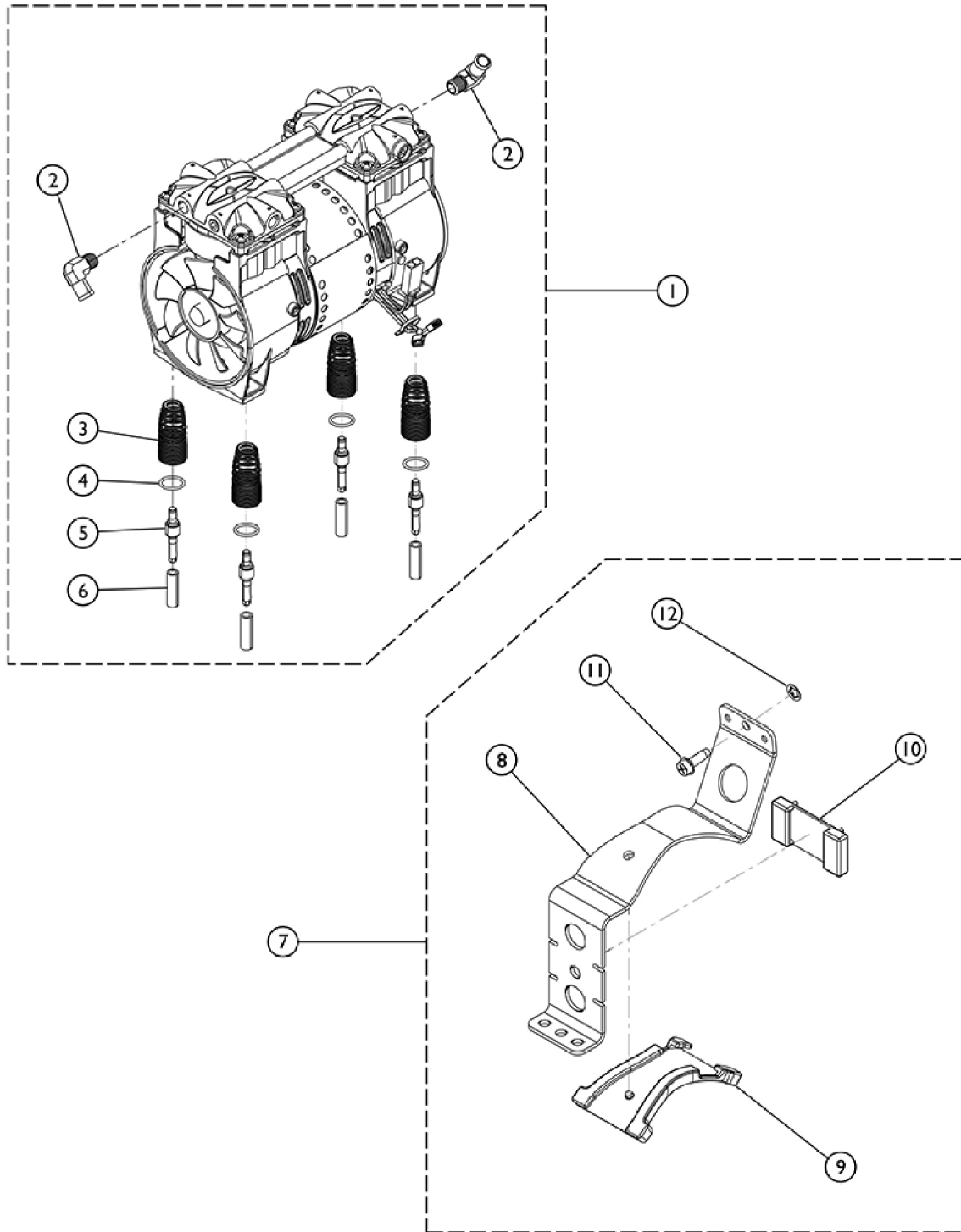

Compressor and Mounting Hardware Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1110825-NLA	NLA - Tech Only - Compressor with Mounting Hardware	1	1
2		1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	2
3		1105504-NLA	NLA - Spring, Compressor Mount	1	4
3		1150581	Spring, Compressor Mount	1	4
4		1106446-NLA	NLA - Pin, Spring Retainer	1	4
4		1150580	Pin, Spring Retainer	1	4
5		1107411	Glide, Pin	1	4
6	B	1186230	Kit, Compressor Restraint Bracket w/ Hardware	1	1
7			Bracket, Restraint, Compressor	1	1
8			NLA - Bumper, Bracket	1	1
9			Bumper, Restraint Bracket, Compressor	1	1
10			NLA - Screw, Phillips Hex Washer Head w/ Patch (1/4-20 x 3/4")	1	1
11			Retainer, Push (1/4")	1	1

NOTE: A - Includes Compressor and items 3 - 5.
B - Includes items 7 - 11.