


Compressor

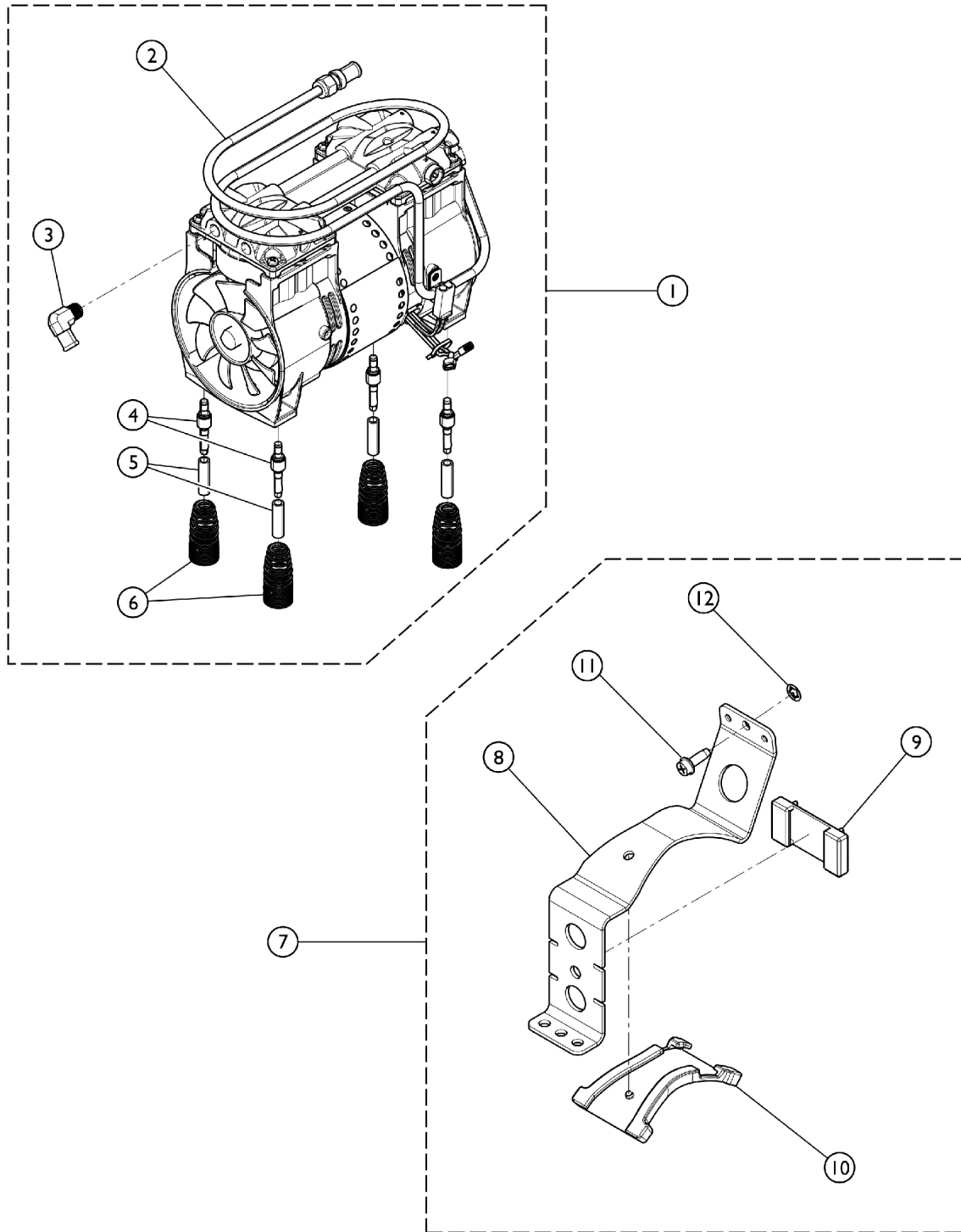

SRE_014A

Compressor

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1136960-NLA	NLA - Tech Only - Compressor with Mounting Hardware	1	1
2		1119590-NLA	NLA - Tech Only - Heat Exchanger Assembly	1	1
3		1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
4		1150580	Pin, Spring Retainer	1	4
4		1106446-NLA	NLA - Pin, Spring Retainer	1	4
5		1107411	Glide, Pin	1	4
6		1105504-NLA	NLA - Spring, Compressor Mount	1	4
6		1150581	Spring, Compressor Mount	1	4
7	C	1186230	Kit, Compressor Restraint Bracket w/ Hardware	1	1
8			Bracket, Restraint, Compressor	1	1
9			NLA - Bumper, Bracket	1	1
10			Bumper, Restraint Bracket, Compressor	1	1
11			NLA - Screw, Phillips Hex Washer Head w/ Patch (1/4-20 x 3/4")	1	1
12			Retainer, Push (1/4")	1	1
	D	1153688	Kit, Compressor Rebuild w/ Instruction Sheet	1	1
			Compressor Rebuild - Thomas	1	1
			Instruction Sheet, Thomas Rebuild 2660-2650	1	1

NOTE: A - Includes Compressor and Items 3 - 6.
 B - Used with IRC5LXO2AWQ.
 C - Includes Items 8 - 12.
 D - Not Shown.