



Fork Assembly - Suspension (Option FLSF)

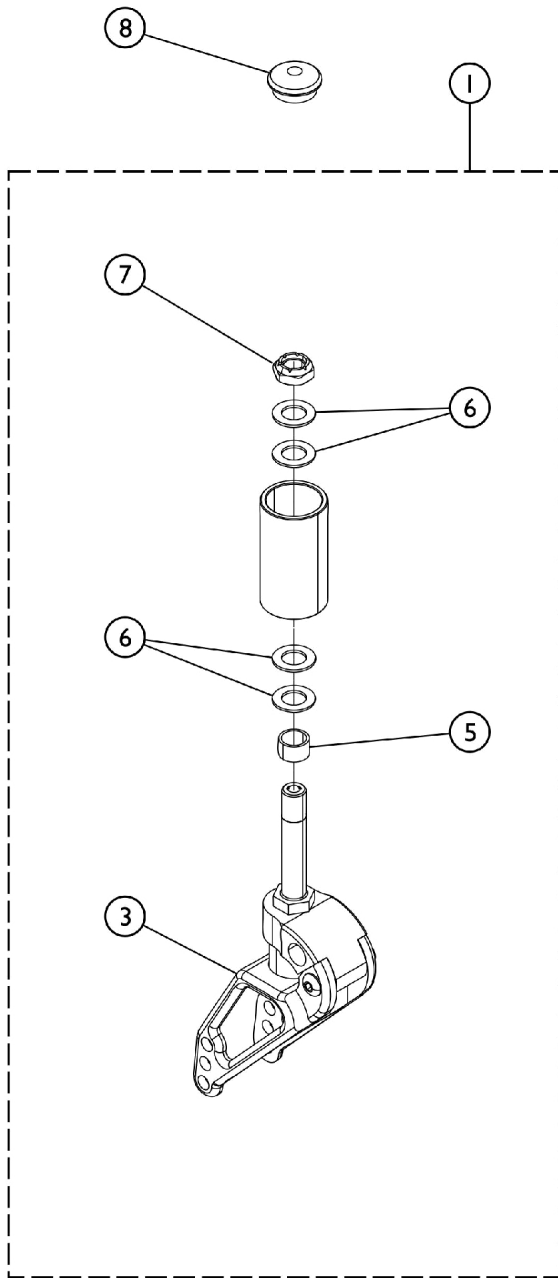

**Suspension Fork for Options
4SPLT, 4ALUM2, 51DC, 5ALUM2
5SPLT, 61DC, 6ALUM2
All Seat to Floor Dimensions**

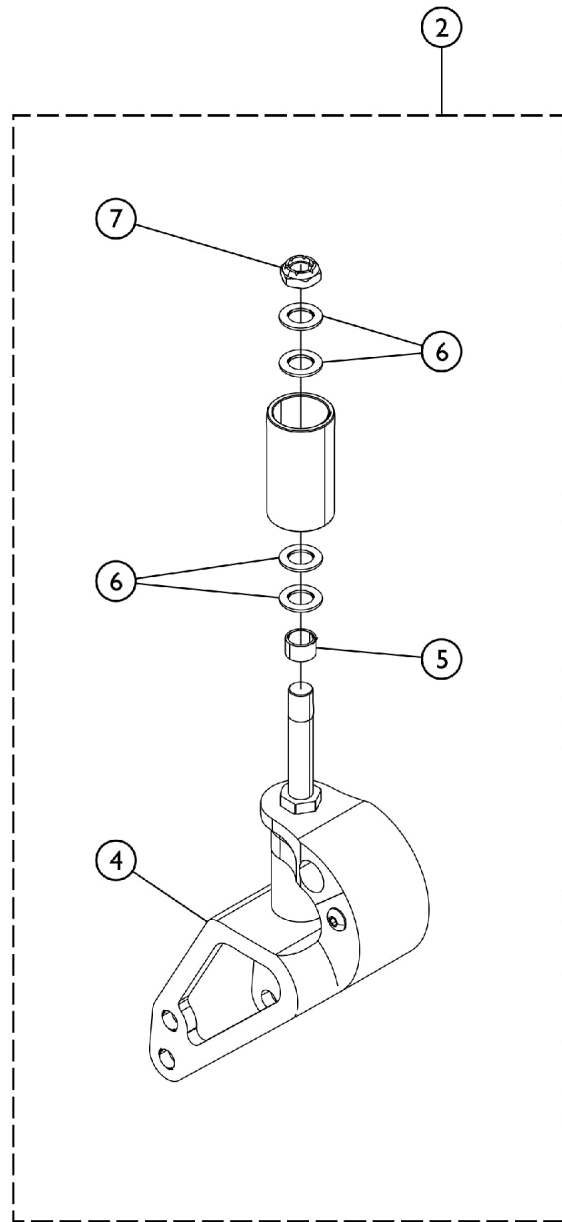

**Suspension Fork for Options
1259, 81DC, 815DC
All Seat to Floor
Dimensions**

Fork Assembly - Suspension (Option FLSF)

| Item Number | Note | Part Number | Description | Quantity Per | | | |
|-------------|----------------------------|-------------|--|--------------|---|---|---|
| | | | | Package | Assembly | | |
| 1 | A | 1121372-NLA | NLA - Kit, Fork, Standard Shock Absorber | 1 | 1 | | |
| 3 | | | NLA - Fork, Suspension, Firm Setting - Standard (Pair) | 1 | 2 | | |
| 5 | | | Spacer (1/2 x 5/8 x 3/8") | 1 | 2 | | |
| 6 | | | Washer (1/2 x 7/8 x 3/64") | 1 | 8 | | |
| 7 | | | Locknut (1/2-20) | 1 | 2 | | |
| 2 | | | B | 1192591 | Kit Fork, Shocks Big Rig | 1 | 1 |
| 4 | | | | | Fork, Suspension, Firm Setting - Large (Pair) | 1 | 2 |
| 5 | Spacer (1/2 x 5/8 x 3/8") | 1 | | | 2 | | |
| 6 | Washer (1/2 x 7/8 x 3/64") | 1 | | | 8 | | |
| 7 | Locknut (1/2-20) | 1 | 2 | | | | |
| 8 | | 1088523 | Plug Button, Head Tube, Black | 1 | 2 | | |

NOTE: A - Kit includes items 3, 5, 6, and item 7.
 B - Kit includes items 4, 5, 6, and item 7.

#1 Fork, Suspension w/ Stem Hardware, Firm Setting - Standard (Pair)

| Caster Option | Rear Wheel (w/ Tire Type) | Seat to Floor Dimension | Part Number |
|--|--|-------------------------|-------------|
| 4SPLT, 4ALUM2 41DC, 415DC 5SPLT, 5ALUM2 5SOFT, 5ALUM 51DC, 515DC AT902, AT903 6ALUM2 61DC | 12", 16", 18" 20", 22", 24" (All Tire Types) | All STF Dimensions | 1121372 |

#2 Fork, Suspension w/ Stem Hardware, Firm Setting - Large (Pair)

| Caster Option | Rear Wheel (w/ Tire Type) | Seat to Floor Dimension | Part Number |
|-----------------------------|--|-------------------------|-------------|
| 1259, 1228 815DC 81DC | 12", 16", 18" 20", 22", 24" (All Tire Types) | All STF Dimensions | 1166084 |