




Motor, GB (TrueTrack) HD and Mounting Hardware TDX SR

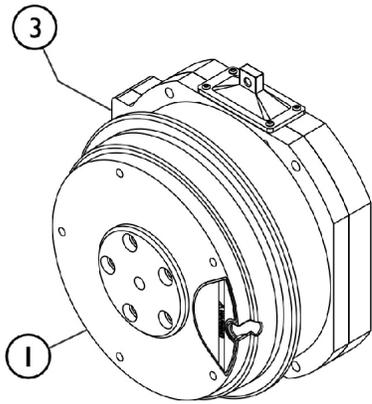

GBTT HD Motor Before 6/29/15

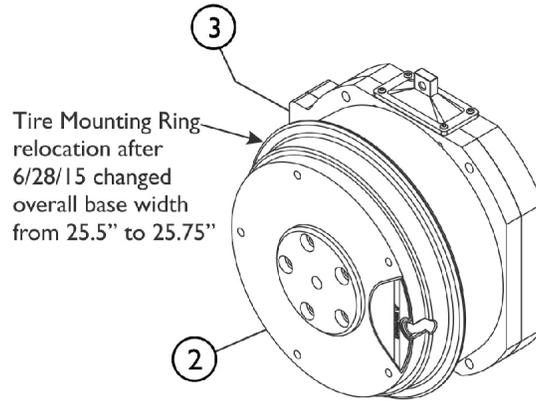

GBTT HD Motor After 6/28/15

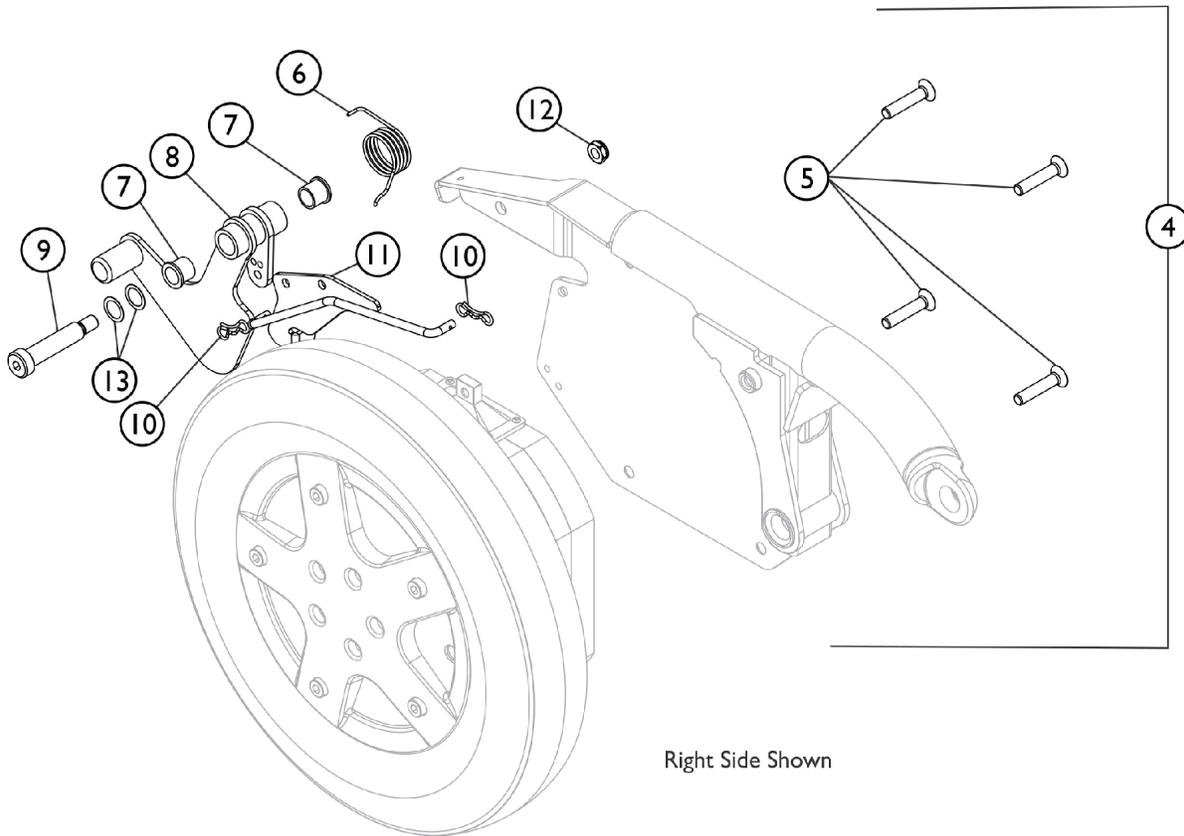

Right Side Shown

Motor, GB (TrueTrack) HD and Mounting Hardware TDX SR

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1,2		See Chart	Item #1 and Item #2 GB (TrueTrack) Heavy Duty Motor Kits	1	1
	B	1122137	Instruction Sheet, GB HD Motor Installation For TDX	1	1
3		1133188-NLA	NLA - Gasket, Motor Connector	1	1
	B,C	1144925	Kit, Stand-Off Screw with Lockwasher	1	2
	B,E,E1	1134214	Kit, GB HD Controller Gasket Connector	1	1
4	A,D	1149338-NLA	NLA - Kit, GB Motor Mounting Hardware TDX SR - Right	1	1
5			NLA - Screw, Socket Flat Head (M8 x 1.25 x 40mm)	1	4
6			Spring, GB Brake Release - Right	1	1
7			Bushing (1/2 x 5/8")	1	2
8			NLA - Lever, Short Release - Right	1	1
9			NLA - Screw, Shoulder Hex Socket Head (1/2 x 2", 3/8-16)	1	1
10			NLA - Cotter, Locking Bow-Tie (1/4" Shaft)	1	2
11			Rod, Linkage - Right	1	1
12			Locknut, Low Profile (3/8-16)	1	1
13			Washer, Wave (17/32 x 3/4 x 1/64")	1	2
4	A,D	1149339-NLA	NLA - Kit, GB Motor Mounting Hardware TDX SR - Left	1	1
5			NLA - Screw, Socket Flat Head (M8 x 1.25 x 40mm)	1	4
6			Spring, GB Brake Release - Left	1	1
7			Bushing (1/2 x 5/8")	1	2
8			NLA - Lever, Short Release - Left	1	1
9			NLA - Screw, Shoulder Hex Socket Head (1/2 x 2", 3/8-16)	1	1
10			NLA - Cotter, Locking Bow-Tie (1/4" Shaft)	1	2
11			Rod, Linkage - Left	1	1
12			Locknut, Low Profile (3/8-16)	1	1
13			Washer, Wave (17/32 x 3/4 x 1/64")	1	2
5		1142378-NLA	NLA - Screw, Socket Flat Head (M8 x 1.25 x 40mm)	1	4

NOTE: A - Includes items 5-13.
 B - Not Shown.
 C - The Stand-Off Screw with Lockwasher is used to secure the controller cable on the GB TT HD Motor.
 D - Item #8 release lever contour was revised 5/05/08 and is backward compatible to all chairs in the field. It is not required to order a right and left side release lever together.
 E - Kit was created 10/26/07 and attaches to the controller motor cables. Includes 2 ea. GB HD Controller Gasket Connectors, 2 ea. Silicon Lubricant Packets, 1 ea. Instruction Sheet. Individual components are non-saleable.
 E1 - The kit can retrofit to existing GB TT HD Controllers in the field.

Motor, GB (TrueTrack) HD and Mounting Hardware TDX SR

Item #1 and Item #2 GB (TrueTrack) Heavy Duty Motor Kits

Item Number/ Date	Side	NEW GB TT HD Motor Kit	WARR RPLMT/ Reconditioned GB TT HD Motor Kit
#1 Before 6/29/15	Right	1191489-NLA See Note(s) B, A	1191489E See Note(s) B, A
#1 Before 6/29/15	Left	1191490-NLA See Note(s) A, B	1191490E See Note(s) A, B
#2 After 6/28/15	Right	1152665-NLA See Note(s) A, B, C	1152665E See Note(s) C, A, B
#2 After 6/28/15	Left	1152666-NLA See Note(s) A, B, C	1152666E See Note(s) C, B, A
NOTE: A - Kit includes the GB TT HD Motor, item #11 Linkage Rod, item #6 Brake Release Spring, item #13 Wave Washers and GB TT HD Motor Installation Instruction Sheet. B - The GB TT HD Motor will include a Controller Gasket Connector Kit created 10/26/07 that attaches to the controller motor cables. The kit can retrofit to existing GB TT HD Controllers in the field. C - GB TT HD Motors after 6/28/15 changed overall base width from 25.5" to 25.75". Refer to illustration for reference.			