


Push Handles

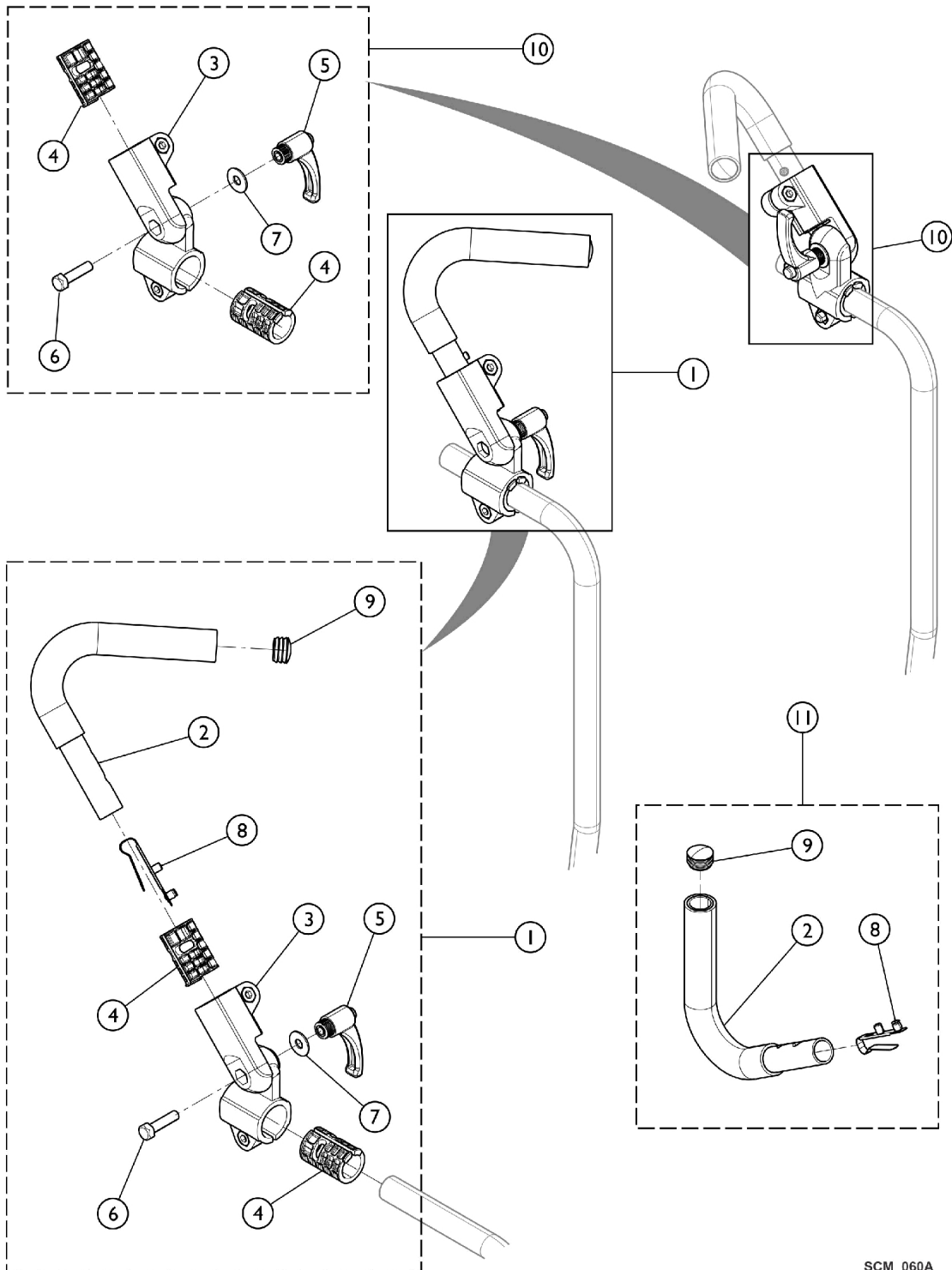

SCM_060A

Push Handles

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1557041	Kit, Push Handles w/ Hardware (7/8" Cane)	1	1
2			Push Bar Assembly	1	2
3			Swivel Coupling	1	2
4			Ring, Push, Reduction Bar (7/8")	1	4
5			Lever, Indexable, Push Handle	1	2
6			Screw (M8 x 35 A2K 8.8 DIN931)	1	2
7			Washer (22 x 8.4 x 1.5 ZP NFE25513)	1	2
8			Spring, Torsion with Button Lock	1	2
9			Button, Plug (22mm)	1	2
10	B	1557061	Kit, Swivel Coupling w/ Lever	1	1
3			Swivel Coupling	1	2
4			Ring, Push, Reduction Bar (7/8")	1	4
5			Lever, Indexable, Push Handle	1	2
6			Screw (M8 x 35 A2K 8.8 DIN931)	1	2
7			Washer (22 x 8.4 x 1.5 ZP NFE25513)	1	2
11	C	1557062	Kit, Push Arm Assembly w/ Hardware	1	1
2			Push Bar Assembly	1	2
8			Spring, Torsion with Button Lock	1	2
9			Button, Plug (22mm)	1	2
NOTE: A - Kit includes the following items 2 - 9. B - Kit includes the following items 3 - 7. C - Kit includes the following items 2, 8, and 9.					