


# Seat Pan Mtg. Hdwr. - Flush Mount (12 - 19" D)

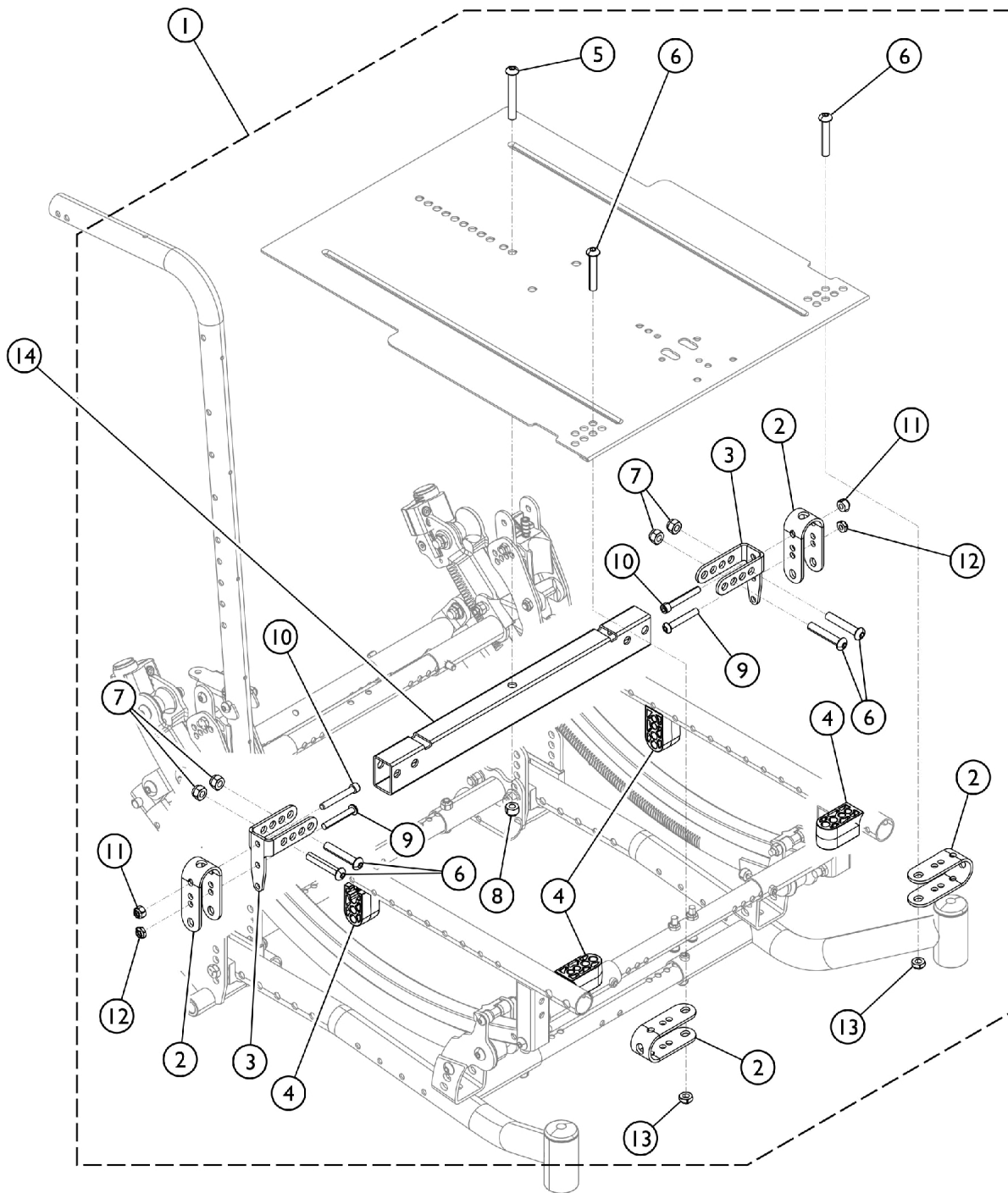

**Flush Mounted Seat Pan - Standard**  
(Small and Medium Frames)

SCM\_192A

## Seat Pan Mtg. Hdwr. - Flush Mount (12 - 19" D)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1166085	Kit, Seat Pan Mounting Hardware, Drop/Flush Mount (12" - 19" D)	1	1
2			Bracket, Seat Pan	1	4
3			Bracket, Seat Pan, Adjustable	1	2
4			Spacer, Block, Black	1	4
5			Screw, Button Head, Socket (5/16-18 x 2-1/4")	1	1
6			Screw, Socket Button Head (5/16-24 x 1-3/4")	1	6
7			Locknut (5/16-24)	1	4
8			Locknut (5/16-18)	1	1
9			Screw, Socket Button Head (1/4-20 x 1-5/8")	1	2
10			Screw, Socket Head (1/4-20 x 1-3/4")	1	2
11			Locknut (1/4-20)	1	2
12			Locknut, Thin (1/4-20)	1	2
13			Locknut (5/16-24)	1	1
14		See Chart	#14. Seat Pan Rear Crossbrace Tube	1	1

NOTE: A - Kit include items 2 - 13.

### #14. Seat Pan Rear Crossbrace Tube

Seat Mounting Type	Seat Frame Width	Part Number
Flush Mounted (FLSEAT)	12" - 13"	1165779
Flush Mounted (FLSEAT)	14" - 15"	1165780
Flush Mounted (FLSEAT)	16" - 17"	1165781
Flush Mounted (FLSEAT)	18" - 19"	1165782
Flush Mounted (FLSEAT)	20" - 21"	1165783
Flush Mounted (FLSEAT)	22" - 23"	1165784
Flush Mounted (FLSEAT)	24"	1165785