


Seat Pan Hdwr, Drop/Flush Mount - HD (18 - 22" D)

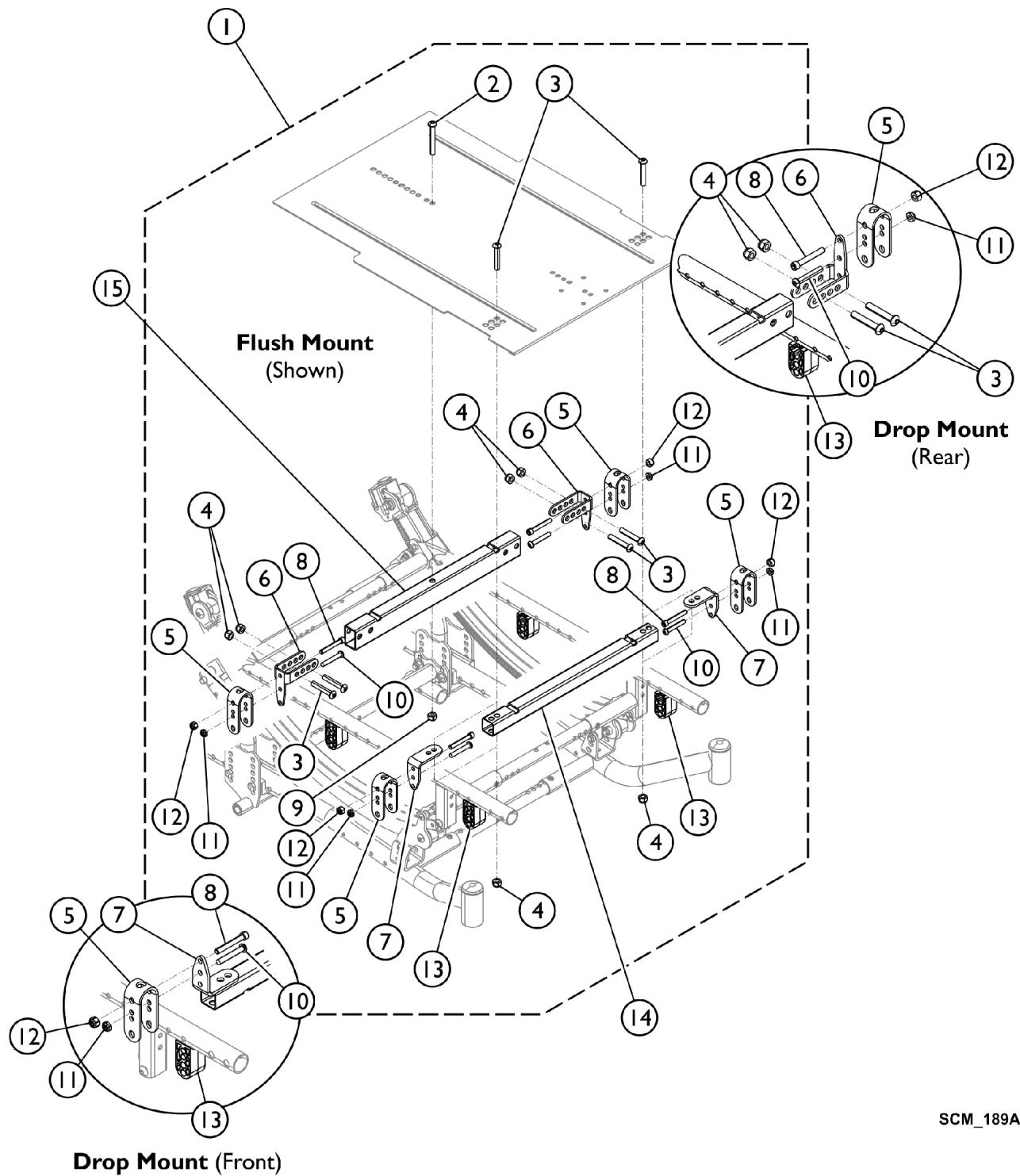

SCM_189A

Seat Pan Hdwr, Drop/Flush Mount - HD (18 - 22" D)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1190141	Kit, Hardware, Seat Pan, Heavy Duty (18-22" D)	1	1
2			Screw, Button Head, Socket (5/16-18 x 2-1/4")	1	1
3			Screw, Socket Button Head (5/16-24 x 1-3/4")	1	6
4			Locknut (5/16-24)	1	6
5			Bracket, Seat Pan	1	4
6			Bracket, Seat Pan, Adjustable	1	2
7			NLA - Bracket, Seat Pan, Front (Long)	1	2
8			Screw, Socket Head (1/4-20 x 1-3/4")	1	4
9			Locknut (5/16-18)	1	1
10			Screw, Socket Button Head (1/4-20 x 1-5/8")	1	4
11			Locknut, Thin (1/4-20)	1	4
12			Locknut (1/4-20)	1	4
13			Spacer, Block, Black	1	4
14		See Chart	#14. Seat Pan Front Crossbraces	1	1
15		See Chart	#15. Seat Pan Rear Crossbrace	1	1

NOTE: A - Kit includes items 2 - 13.

#14. Seat Pan Front Crossbraces (SDR1822)

Option	Chair Width	Part Number
SDR1822	14-15" W	1190117
SDR1822	16-17" W	1190118
SDR1822	18-19" W	1190119
SDR1822	20-21" W	1190120
SDR1822	22-23" W	1190121
SDR1822	24" W	1190122

#15. Seat Pan Rear Crossbrace

Option	Chair Width	Part Number
Flush Mount/Drop Mount	12-13" W	1165779
Flush Mount/Drop Mount	14-15" W	1165780
Flush Mount/Drop Mount	16-17" W	1165781
Flush Mount/Drop Mount	18-19" W	1165782
Flush Mount/Drop Mount	20-21" W	1165783
Flush Mount/Drop Mount	22-23" W	1165784
Flush Mount/Drop Mount	24" W	1165785