



Adjustable Angle Footplates with/without Pivot Tubes (AT5543)

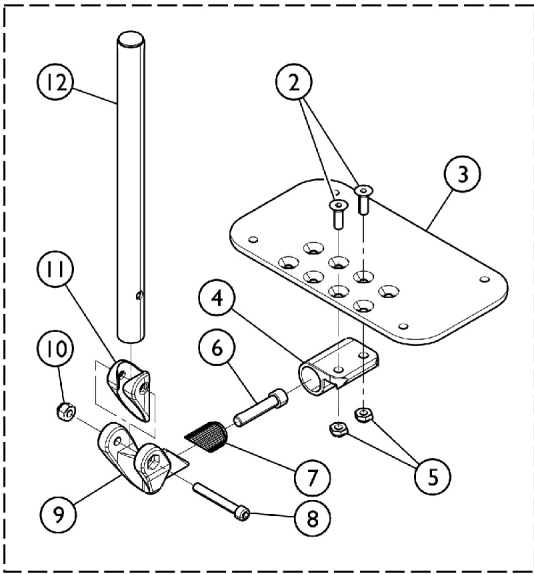


**70 Degree Lift-Off and 80 Degree No Taper
Swingaway, Articulating Legrests**

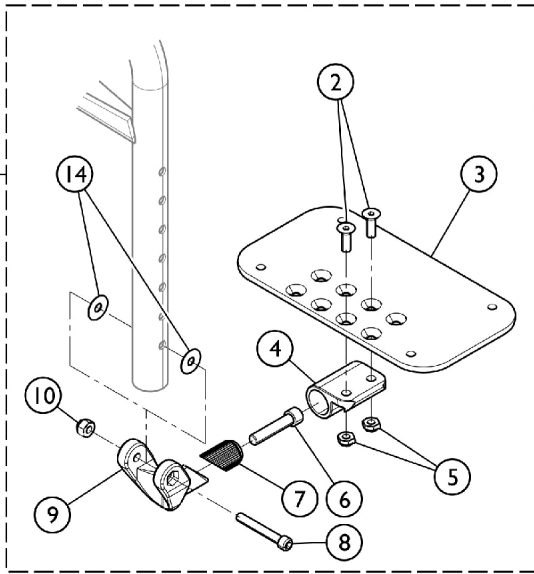

**70 Degree MFX, 80 Degree MFX and
90 Degree Swingaway
Front Riggings**

Used on All Front Riggings EXCEPT Wedge Style Elevating Legrests (AT5044/AT5044LIFT)

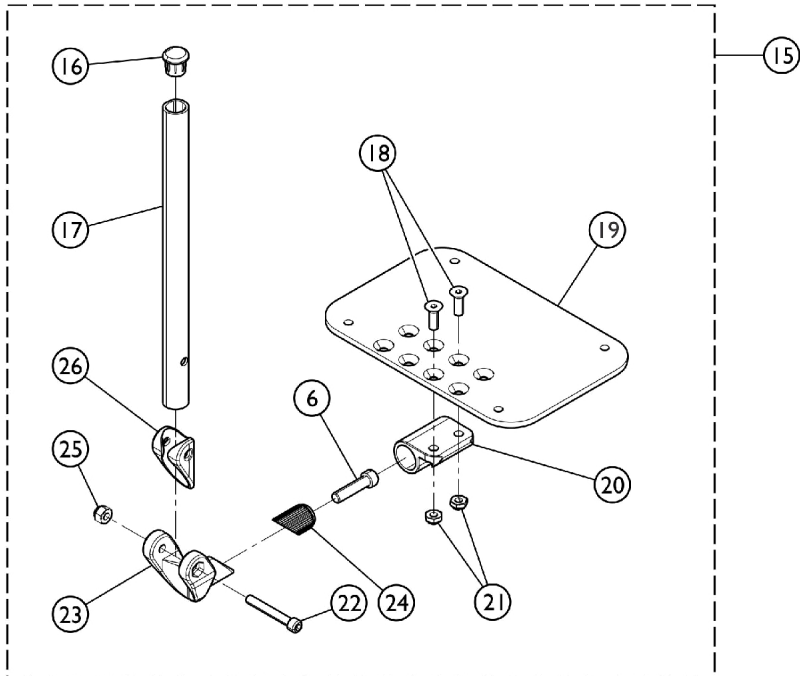

Used on Wedge Style Elevating Legrests (AT5044/AT5044LIFT)

Adjustable Angle Footplates with/without Pivot Tubes (AT5543)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	See Chart	#1 60 Degree, 60 Degree MFX, 70 and 80 Degree Non Tapered, 70 Degree Lift Off Footrests	1	1
1	D	See Chart	#1 70 Degree Tapered Footrests	1	1
1	D	See Chart	#1 ATL4A Articulating Legrests	1	1
2		1113578	Screw, Socket Flat Head (1/4-28 x 3/4")	1	2
3		1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
3		1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
3		1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3		1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
4		1113528	Bracket, Adjustable Angle Footplate, Black	1	1
5		1025765	Locknut (1/4-28)	1	2
6		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	1
7		1113530	Attachment, Hinge	1	1
8		1096810	Screw, Socket Head (1/4-28 x 1-3/4")	1	1
9		1113529	Hinge, Adjustable Angle Footrest	1	1
10		1065362	Locknut (1/4-28)	1	1
11		1114688	Spacer, Hinge, Black	1	1
12	A	1114699	Tube, Pivot/Slide, Straight, Black (10-1/4")	1	1
12	B	1114700	Tube, Pivot/Slide, Straight, 2" Longer ALPT, Black (12-1/4")	1	1
12	C	1114707	Tube, Pivot/Slide, Straight, Chrome (11-7/8")	1	1
13	E	See Chart	#13 70/80 Degree MFX, 90 Degree, and 70/90 Degree MFX Lift Off Footrests	1	1
14		1025456	Washer, Nylon (3/4 x 1/4 x 1/32")	1	2
15	F	See Chart	#15 Wedge Style AT5044 Elevating Legrests	1	1
16		1134427	Package, Plug Button, Black, "D" Shaped (3/4")	10	1
17	G	1134433	Pivot/Slide Tube w/ D Insert, Black	1	1
17	G	1134432	Pivot/Slide Tube w/ D Insert, Black (2" Longer)	1	1
18		1113578	Screw, Socket Flat Head (1/4-28 x 3/4")	1	2
19		1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
19		1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
19		1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
20		1113528	Bracket, Adjustable Angle Footplate, Black	1	1
21		1025765	Locknut (1/4-28)	1	2
22		1096810	Screw, Socket Head (1/4-28 x 1-3/4")	1	1
23		1113529	Hinge, Adjustable Angle Footrest	1	1
24		1113530	Attachment, Hinge	1	1
25		1065362	Locknut (1/4-28)	1	1
26		1114688	Spacer, Hinge, Black	1	1

Adjustable Angle Footplates with/without Pivot Tubes (AT5543)

NOTE: A - 60 Degree, 60 Degree MFX, 70 Degree Tapered, 70 Degree Non Tapered, 70LIFTNT, 80 Degree Non Tapered Footrests
B - ALPT 2" Longer
C - ATL4A Articulating Legrests
D - Includes items 2-12
E - Includes items 2-10 & 14
F - Includes items 16-27
G - Wedge Style AT5044 Legrest

#1 Adjustable Angle Footplates w/ Pivot Tubes

SWING1319, SWING914, 70NTAPER, 70LIFTNT, and 80NTAPER Footrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
12" - 13" Wide (4-1/4" Footplate)	1114640	1114641	12" - 13" Wide (4-1/4" Footplate)	1114650	1114651
14" - 15" Wide (5-1/4" Footplate)	1114642	1114643	14" - 15" Wide (5-1/4" Footplate)	1114652	1114653
16" - 17" Wide (6-1/4" Footplate)	1114644	1114645	16" - 17" Wide (6-1/4" Footplate)	1114654	1114655
18" - 24" Wide (7-1/4" Footplate)	1114646	1114647	18" - 24" Wide (7-1/4" Footplate)	1114656	1114657

#1 Adjustable Angle Footplates w/ Pivot/Slide Tubes

70STAPER Footrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
16" Wide (4-1/4" Footplate)	1114640	1114641	16" Wide (4-1/4" Footplate)	1114650	1114651
17" - 18" Wide (5-1/4" Footplate)	1114642	1114643	17" - 18" Wide (5-1/4" Footplate)	1114652	1114653
19" - 24" Wide (6-1/4" Footplate)	1114644	1114645	19" - 24" Wide (6-1/4" Footplate)	1114654	1114655

#1 Adjustable Angle Footplates w/ Pivot Tubes

ATL4A Articulating Legrests

Straight Slide Tube (10-1/4")	Right	Left
14" - 15" Wide (5-1/4" Footplate)	1114708	1114709
16" - 17" Wide (6-1/4" Footplate)	1114710	1114711
18" - 24" Wide (7-1/4" Footplate)	1114712	1114713

Adjustable Angle Footplates with/without Pivot Tubes (AT5543)

#13 Adjustable Angle Footplates w/o Pivot Tubes

70MFX, 80MFX, 90SWING, 70LIFTMX, and 90LIFTMX Footrests

No Slide Tube - Bolts Through Support	Right	Left
12" - 13" Wide (4-1/4" Footplate)	1114630	1114631
14" - 15" Wide (5-1/4" Footplate)	1114632	1114633
16" - 17" Wide (6-1/4" Footplate)	1114634	1114635
18" - 24" Wide (7-1/4" Footplate)	1114636	1114637

#15 Adjustable Angle Footplates w/ Pivot Tubes

Wedge Style AT5044 Elevating Legrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
14" - 15" Wide (5-1/4" Footplate)	1132061	1132062	14" - 15" Wide (5-1/4" Footplate)	1132089	1132090
16" - 17" Wide (6-1/4" Footplate)	1132063	1132064	16" - 17" Wide (6-1/4" Footplate)	1132091	1132092
18" - 24" Wide (7-1/4" Footplate)	1132065	1132066	18" - 24" Wide (7-1/4" Footplate)	1132093	1132094