

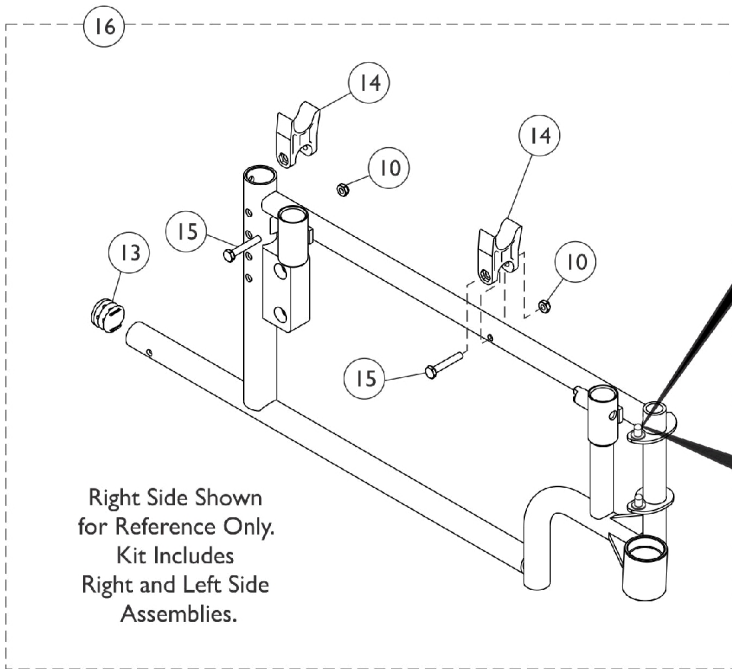



Bariatric Side Frame and Hardware

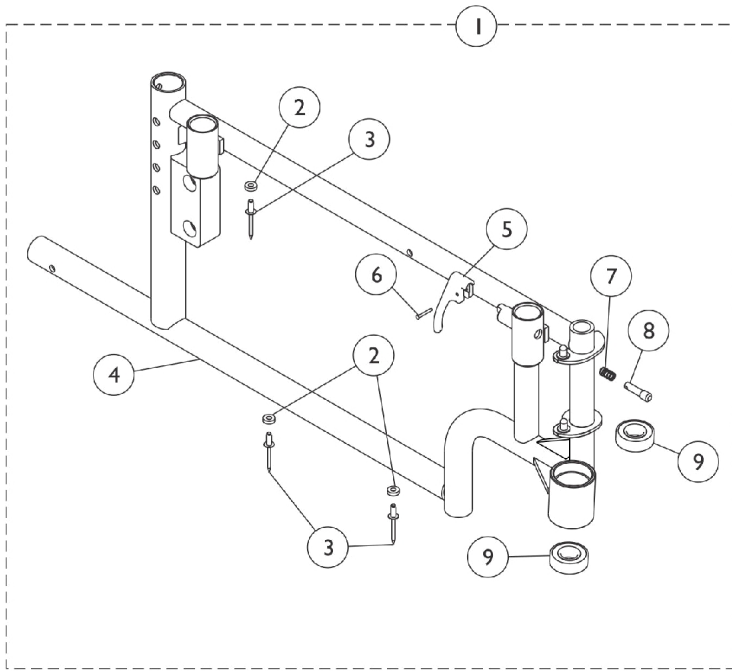

After 4/11/13

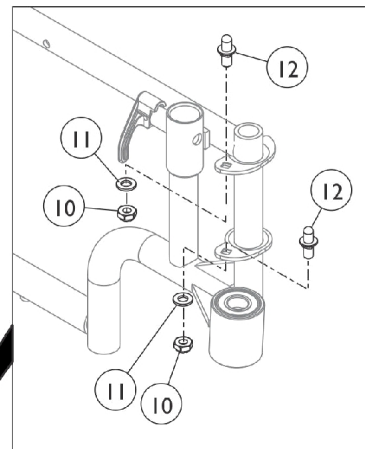

Before 4/12/13

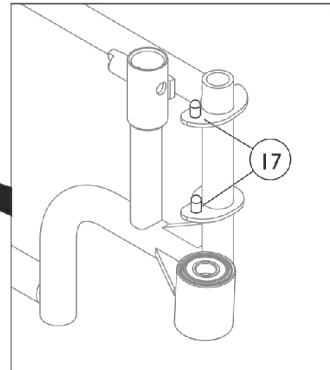

Bariatric Side Frame and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D,D1	8881140874	Frame Assembly, Side, Color - Right	1	1
2			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
3			NLA - Rivet (3/16 x 7/16")	1	3
4			Frame, Side, Color - Right	1	1
5			Handle, Lock, Black - Right	1	1
6			Rivet, Tubular	1	1
7			Spring, Arm Lock	1	1
8			Pin, Locking	1	1
9			Bearing (5/8")	1	2
1	A,D,D1	8881140875	Frame Assembly, Side, Color - Left	1	1
2			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	3
3			NLA - Rivet (3/16 x 7/16")	1	3
4			Frame, Side, Color - Left	1	1
5			Handle, Lock, Black - Left	1	1
6			Rivet, Tubular	1	1
7			Pin, Locking	1	1
8			Spring, Arm Lock	1	1
9			Bearing (5/8")	1	2
16	B	1100792	Kit, Frame Hardware (Pair)	1	1
10			Locknut (1/4-20)	1	8
11			Washer, Plain (1/4 x 1/2 x 9/16")	1	4
12			Pin, Hinge	1	4
13			Plug Button, Black (3/4")	1	2
14			H-Block	1	4
15			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
17	D,D1	1096781	Kit, Hinge Pin Replacement Hardware (Pair)	1	1
			NLA - Wrench, Allen (3/32")	1	1
			Cap, Threaded	1	4
			Washer, Locking Split (#10)	1	4
			Screw, Socket Button Head Cap, Regulator (8-32 x 3/8")	1	4
	C	7100053	Arm Lock Assembly, Black - Right	1	1
5			Handle, Lock, Black - Right	1	1
6			Rivet, Tubular	1	1
7			Spring, Arm Lock	1	1
8	C	7100054	Pin, Locking	1	1
			Arm Lock Assembly, Black - Left	1	1
5			Handle, Lock, Black - Left	1	1
6			Rivet, Tubular	1	2
7			Spring, Arm Lock	1	2
8			Pin, Locking	1	2

NOTE: A - Includes items 2-9. Specify color when ordering. See category titled, "Frame Finishes, Packaging and Upholstery Colors."
 B - Includes items 10-15.
 C - Not shown. Includes items 5-8.
 D - Hinge Pin Replacement Hardware Kit is only used on Side Frames w/ welded-on Hinge Pins manufactured before 4/12/13. Side Frames after 4/11/13 have Hinge Pin Saddle Plates w/ square holes to attach REMOVABLE Non-Welded Hinge Pins.
 D1 - All Side Frames ordered after 4/11/13 require new Side Frame Hardware that will include the removable Hinge Pins also. Removable Hinge Pins are not sold separately.