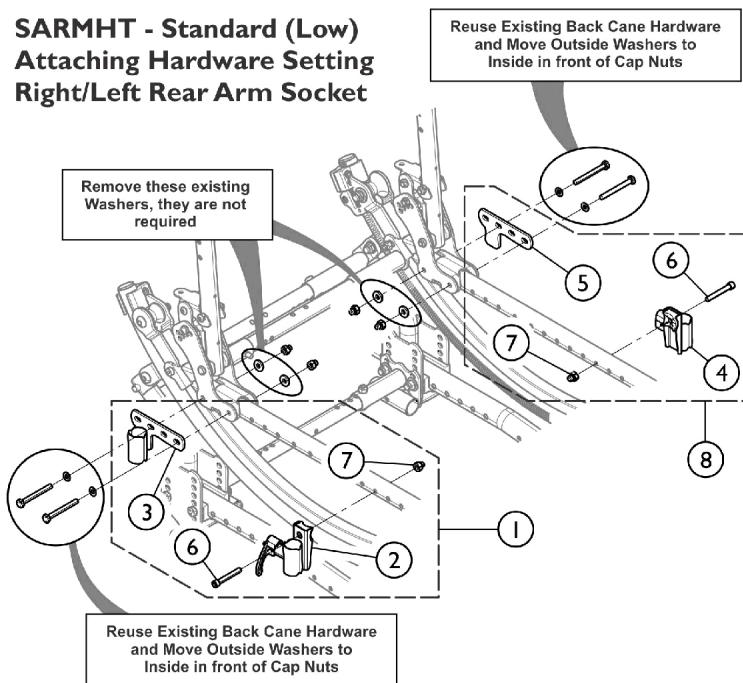
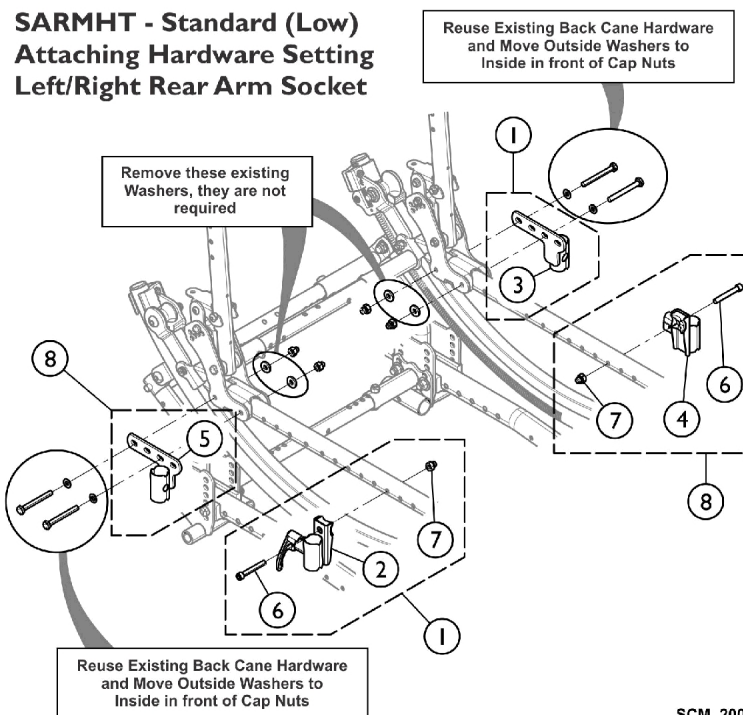


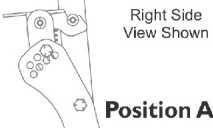
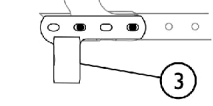
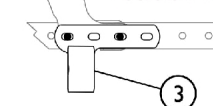
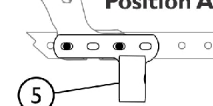
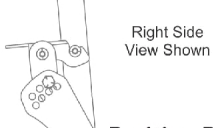
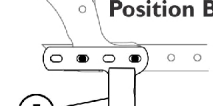
Arms - Non-Flip Back Attaching Hardware (SARMHT Option)

SARMHT - Standard (Low) Attaching Hardware Setting Right/Left Rear Arm Socket



SARMHT - Standard (Low) Attaching Hardware Setting Left/Right Rear Arm Socket



Rear Arm Socket Right, Standard  Position A 	SDR1216	D15
	SDR1519	D15
	SDR1822	D18
Rear Arm Socket Right, Standard  Position B 	SDR1216	D16
	SDR1519	N/A
	SDR1822	N/A
Rear Arm Socket Left, Standard  Position A 	SDR1216	N/A
	SDR1519	D17 D18 D19
	SDR1822	D20 D21 D22
Rear Arm Socket Left, Standard  Position B 	SDR1216	N/A
	SDR1519	D16
	SDR1822	D19

SCM_200A

Arms - Non-Flip Back Attaching Hardware (SARMHT Option)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1164104	Kit, Arm Attaching Hardware - Right	1	1
2			Socket, Arm, Rear, Black - Right	1	1
3			Socket, Arm, Front, Black - Right	1	1
6			NLA - Screw (1/4-20 x 1-3/4")	1	1
7			Nut, Hex with Nylon Cap (1/4-20)	1	1
8	B,C	1164105	Kit, Arm Attaching Hardware - Left	1	1
4			Socket, Arm, Rear, Black - Left	1	1
5			Socket, Arm, Front, Black - Left	1	1
6			NLA - Screw (1/4-20 x 1-3/4")	1	1
7			Nut, Hex with Nylon Cap (1/4-20)	1	1
		See Chart	Arm Settings and Hardware (SARMHT Option)	1	1
NOTE: A - Kit includes items 2, 3, 6, and item 7. Parts are not sold separately. B - Kit includes items 4, 5, 6, and item 7. Parts are not sold separately. C - For further information, see "Arm Settings and Hardware (SARMHT Option)" Chart.					

Arm Settings and Hardware (SARMHT Option)

See Illustration for Corresponding Information

SARMHT Option	Rear Arm Socket Right (Item 3)	Seat Depth (In.)	Arm Option
1164104	SDR1216	D15	
Right	SDR1519	D15	
Position A	SDR1822	D18	
1164104	SDR1216	D16	
Right	SDR1519	N/A	
Position B	SDR1822	N/A	
	Item 3 Mounted on Right Side		FXDESK - FXFULL ADJDESK - ADJFULL
SARMHT Option	Rear Arm Socket Left (Item 5)	Seat Depth (In.)	Arm Option
1164105	SDR1216	N/A	
Left	SDR1519	D17, D18, D19	
Position A	SDR1822	D20, D21, D22	
1164105	SDR1216	N/A	
Left	SDR1519	D16	
Position B	SDR1822	D19	
	Item 5 Mounted on Right Side		FXDESK - FXFULL ADJDESK - ADJFULL