


Arms - Swing Away - Front Mount - Transport Brackets

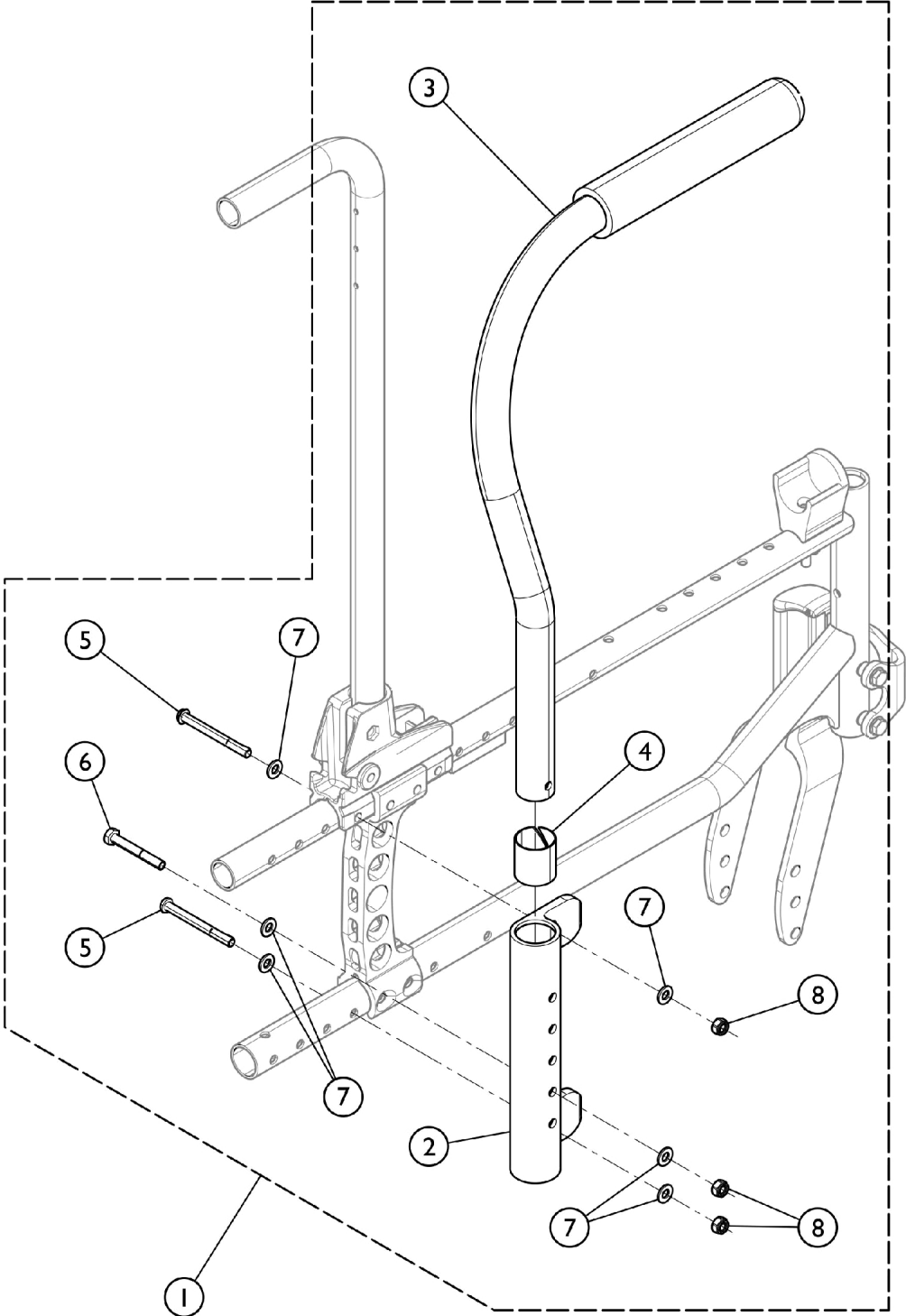

SCM_204A

Arms - Swing Away - Front Mount - Transport Brackets

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1190148	Kit, Swingaway Arm Mounting w/ Hardware	1	1
2			Socket, Arm	1	2
3			Arm, Swing Away, Half, Padded - Right	1	1
3			Arm, Swing Away, Half, Padded - Left	1	1
4			Sleeve, Bearing	1	2
5			Screw (M6 x 1 x 60mm)	1	4
6			Bolt, Hex Head (M6 x 45mm)	1	2
7			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	12
8			NLA - Locknut (M6 x 1)	1	6

NOTE: A - Kit includes items 2 thru 8.