


Frame, Pan Deck

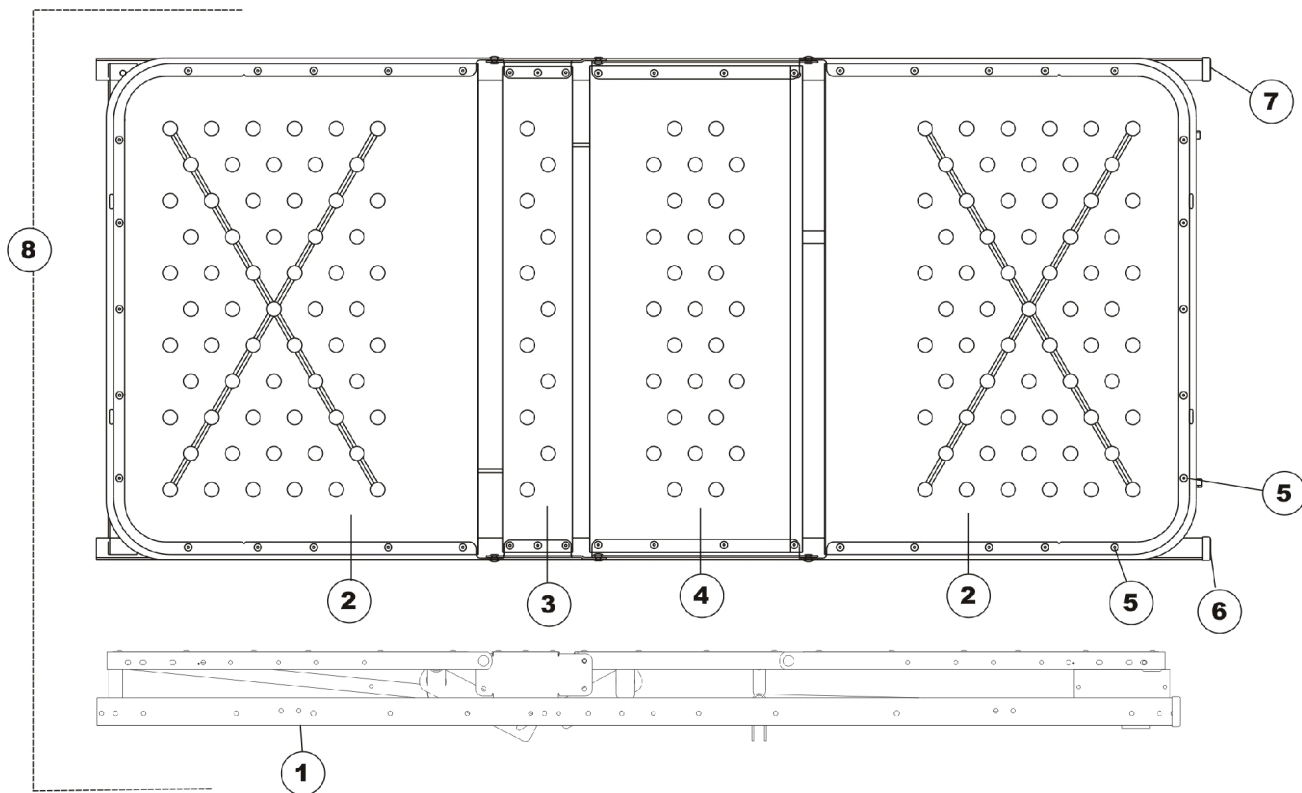

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1124062-NLA	NLA - Frame & Deck Assembly, Taupe	1	1
2		026506	Pan Deck Assembly (76")	1	2
3		026507	Pan Deck - Seat Section Assembly	1	1
4		026508	Pan Deck, Thigh Section	1	1
5		026623	Rivet (1/4")	1	44
6		1125002	Cap, Angle - Left (2 x 1-1/2 x 1/8")	1	1
7		1125015	Cap, Angle - Right (2 x 1-1/2 x 1/8")	1	1
8	A	1124073-NLA	NLA - Frame & Pan Deck Assembly, Taupe	1	1
	B	1124029-NLA	NLA - Kit, Corner Bracket, Taupe	1	1
	B	1124031	Kit, Mattress Keeper Includes: 4 ea. Mattress Keepers, 8 ea. Nuts, 8 ea. Screws, and 1 ea. Poly Bag	1	1

NOTE: A - Includes: Items 1-7.
B - Not Shown.