


Carriage Assembly - 820DLX & 820-3MDLX

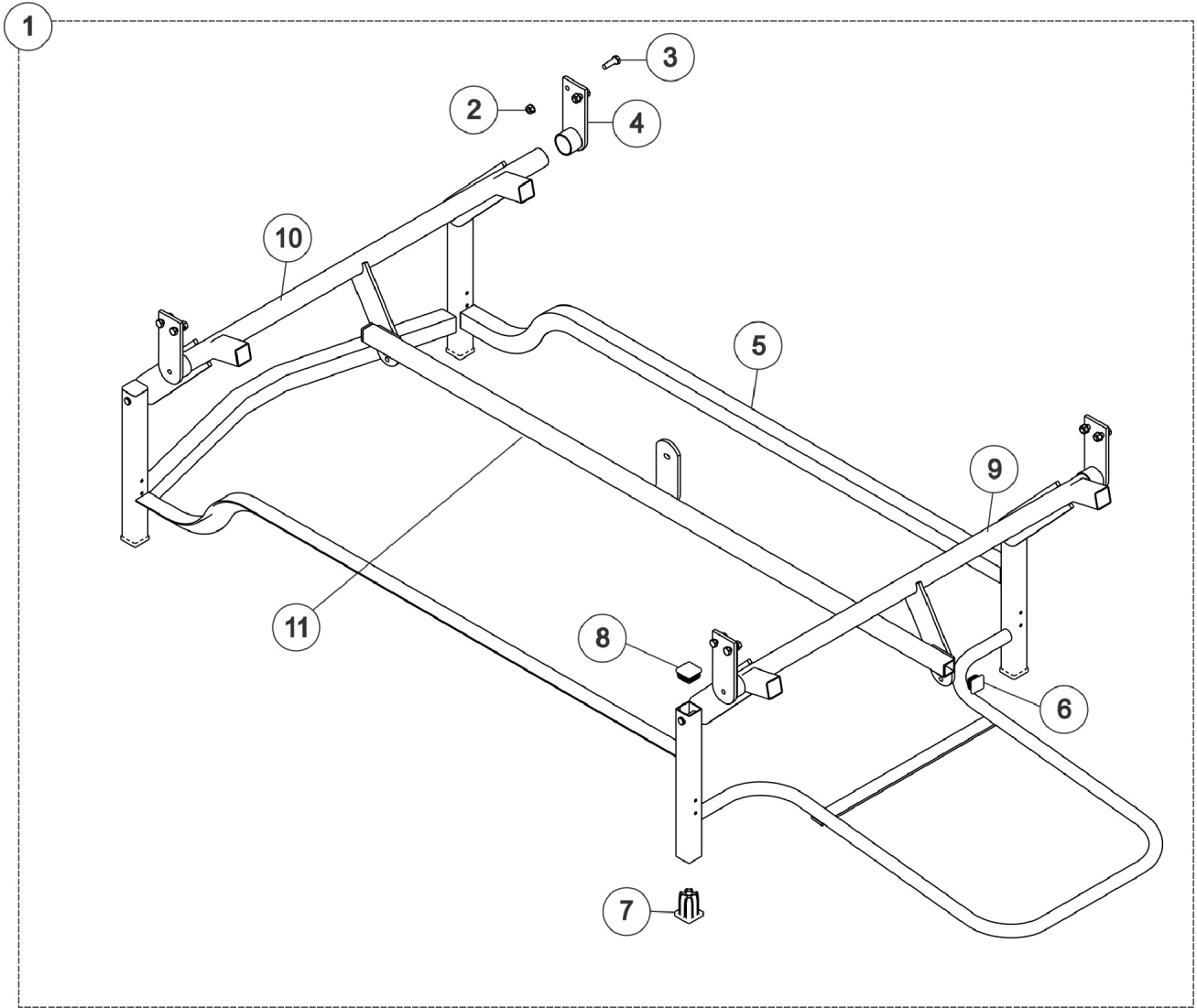

Carriage Assembly - 820DLX & 820-3MDLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140821	Carriage, Floor Lock Reverse Assembly	1	1
2		000037	NLA - Nut, Keps (5/16-18)	1	8
3		001758	Screw, Hex Head (5/16-18 x 1")	1	8
4		025187	Support, Lift Crank	1	1
5		1140820	Frame, Carriage Lock Floor Reverse	1	1
6		020717	Glide, Cap	1	2
7		1131077	Foot, Rubber	1	4
8		1012756	Plug Button, Black (1-1/4")	1	4
	B,C	009077	Kit, Caster	1	1
	B,D	51000M083	Socket, Caster	1	4
	B,D	1123551	Caster Assembly, Swivel w/ Brake (3")	1	2
	B,D	1123549	Caster Assembly, Swivel w/o Brake (3")	1	2
9		1124051	Head End Lift Crank	1	1
10		1124052	Foot End Crank	1	1
11		1124032	Lift Bar	1	1

NOTE: A - Includes: Items 1-8.
 B - Not Shown
 C - Includes: 4 Sockets, 2 Locking Casters and 2 Non-Locking Casters.
 D - Included in part # 009077.