
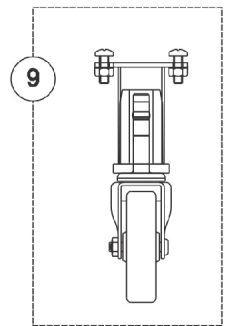


Carriage Assembly - SC900DLX

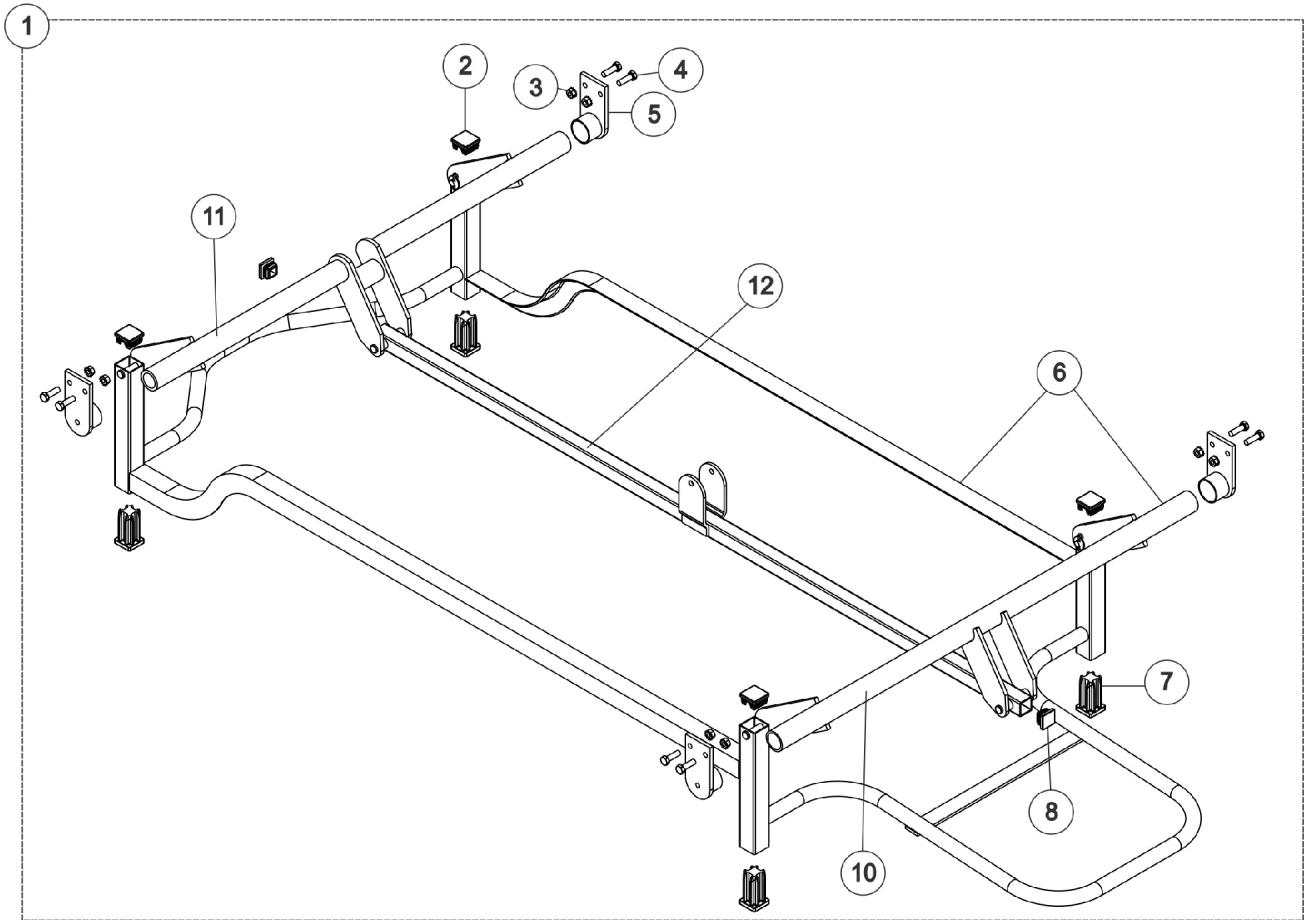

Carriage Assembly - SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1137378	Carriage Assembly - SC900DLX	1	1
2		027047	Plug Button with Hole (1-1/4" Square)	1	4
3		000037	NLA - Nut, Keps (5/16-18)	1	8
4		001758	Screw, Hex Head (5/16-18 x 1")	1	8
5		1133050	Support, Lift Crank	1	2
6		1137377	Frame, Carriage - SC900DLX	1	1
7		1131077	Foot, Rubber	1	4
8		020717	Glide, Cap	1	2
9	B	027027	Kit, Caster 3" Swivel w/ Bracket	1	1
	C	000050	Nut, Keps (1/4-20)	1	8
	C	51000M789	Screw, Phillips Pan Head (1/4-20 x 3/4")	1	8
	C	027025	Bracket, Caster	1	4
	C	51000M083	Socket, Caster	1	4
	C	1123551	Caster Assembly, Swivel w/ Brake (3")	1	2
	C	1123549	Caster Assembly, Swivel w/o Brake (3")	1	2
10		027016	Crank, Lift - Head End	1	1
11		026999	Crank, Lift - Foot End	1	1
12		027001	Bar, Lift	1	1

NOTE: A - Includes: Items 2-8.
 B - Includes: 4 Brackets, 4 Sockets, 8 Screws, 8 Nuts, 2 Locking Casters and 2 Non-Locking Casters. See page titled "Caster Parts" for additional details"
 C - Included in part number 027027 - Caster Kit