


# Motors and Components, Full Electric - 820-3MDLX & SC900DLX

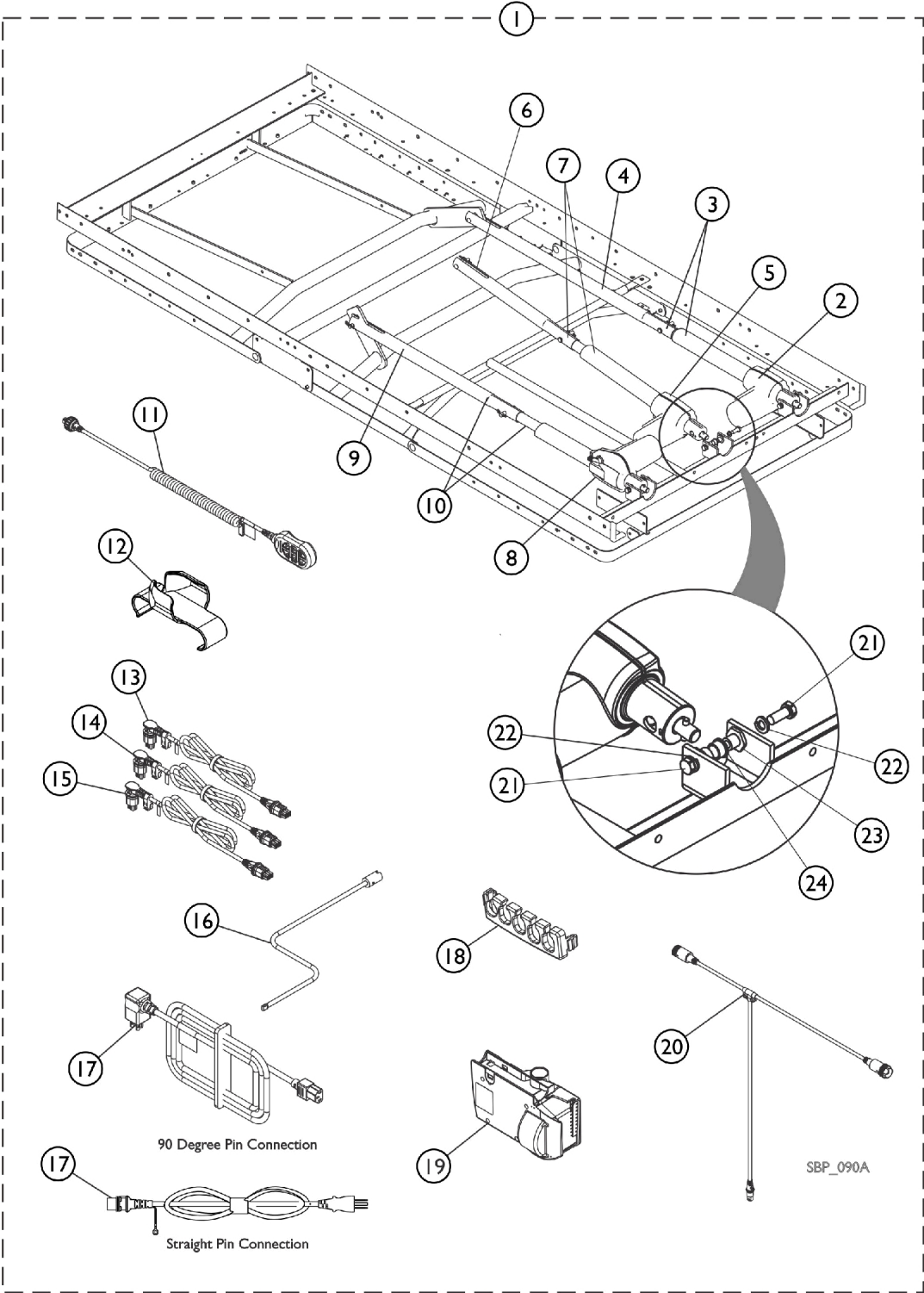

## Motors and Components, Full Electric - 820-3MDLX & SC900DLX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1140814	Motor & Component Assembly - 820-3MDLX	1	1
1	B	1140803	Motor & Component Assembly - SC900DLX	1	1
2	C	1115291-NLA	NLA - Head Actuator - (Linak #LA27-0006-04)	1	1
3		1137384	Motor/Gear Assembly, Head	1	1
4		1137343	Head Actuator Pull Tube	1	1
5		1137338	HiLo Actuator - 820-3MDLX (Linak #LA27-U026)	1	1
5		1137337	HiLo Actuator - SC900DLX (Linak #LA27-U025-00)	1	1
6		1137341	HiLo Actuator Pull Tube - 820-3MDLX	1	1
6		1137381	HiLo Actuator Pull Tube - SC900DLX	1	1
7	E	1140806	Motor/Gear Assembly, Hi-Lo - 820-3MDLX	1	1
7	F	1137383	Motor/Gear Assembly, Hi-Lo - SC900DLX	1	1
8	D	1115292-NLA	NLA - Foot Actuator - (Linak #LA27-0007-03)	1	1
9		1137342	Foot Actuator Pull Tube	1	1
10		1137385	Motor/Gear Assembly, Foot	1	1
11		1180040	Pendant, 6 Channel	1	1
12		1133841	Holster, Pendant	1	1
13		1116640	Cable, Hi-Lo Motor w/ Red O-Ring (Head End)	1	1
14		1116641	Cable, Head Actuator w/ Blue O-Ring	1	1
15		1116642	Cable, Foot Actuator w/ Yellow O-Ring	1	1
16		1133062	Crank, Emergency	1	1
17		1132002	Power Cord, 10-1/2' (Hospital Grade) with Angled Pin Connectors	1	1
17		1144786	Power Cord, 17' with Straight Pin Connectors	1	1
18		1116655	Lock, Cable	1	1
19		1117178	Controller/Junction Box, 6 Channel	1	1
20		1140812	Cable, "T"	1	1
21		001741	Screw, Hex Head (5/16-18 x 3/4")	1	2
22		001008	Washer, Split Lock (5/16")	1	2
23		001443	Bushing, Shoulder - Pivot	1	1
24		001405	Bushing, Flanged, Nylon	1	1

NOTE: A - Includes items 2-5, 6, 7, 8-24  
 B - Includes items 2-4, 5, 6, 7, 8-24  
 C - No Longer Available separately. Must order item #3 1137384 that includes 1115291 Head Motor and Pull Tube w/ Hardware.  
 D - No Longer Available separately. Must order item #10 1137385 that includes 1115292 Foot Motor and Pull Tube w/ Hardware.  
 E - Includes item #5 1137338 HiLo Actuator 820-3MDLX and Pull Tube w/ Hardware.  
 F - Includes item #5 1137337 HiLo Actuator SC900DLX and Pull Tube w/ Hardware.