


Forks and Hardware

9000SL / 9000XT / 9000XT Recliner

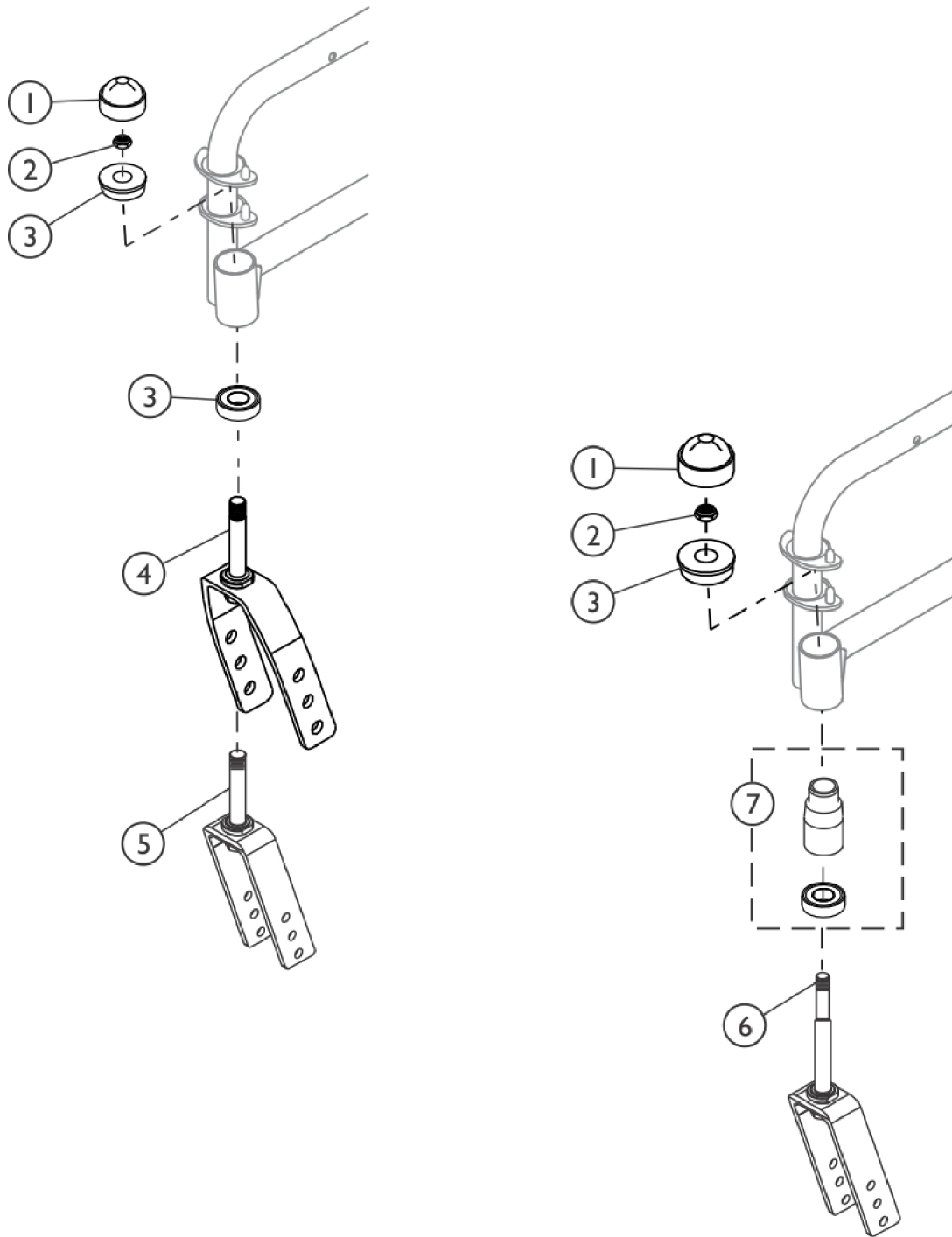

SSM_030B

Forks and Hardware

9000SL / 9000XT / 9000XT Recliner

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1001843	Cap, Head Tube, Black	1	1
1	E	1131189	Package, Head Tube Cap	10	1
1	F	1192854	Cap, Head Tube, Black	1	1
2		1001979	Locknut (1/2-20)	1	1
3	E	1001801	Bearing, Flange (1/2")	1	2
3	F	1192848	Bearing, Flange (5/8")	1	2
4	A,E	1056640	Fork and Stem Assembly, Black (8" & 6 x 2" Caster)	1	1
4	A,F	1192855	Fork and Stem Assembly, Black (8" & 6 x 2" Caster)	1	1
5	B	1037420	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
5	D	1194267	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
6	C	1067784	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
7	C	1066589	Bottom Sleeve and Bearing Assembly, Black	1	1

NOTE: A - Used for 9XT/ 9SL and 9RC models.
 B - Not available for 9RC model. Used for 9XT Before 10/12/16 and ("ALL" 9SL models with 6" Casters).
 C - Only used for 9XT models with Adult S-T-F setting, AT902 6" Caster Option and a 24" Rear Wheel.
 D - Not available for 9RC model. Used for 9XT After 10/11/16 with 6" Caster.
 E - Used for 9XT, 9RC models Before 10/12/16 and ("ALL" 9SL models with 8" casters).
 F - Used for 9XT and 9RC models After 10/11/16.