


# Rear Wheels - Wheel Locks

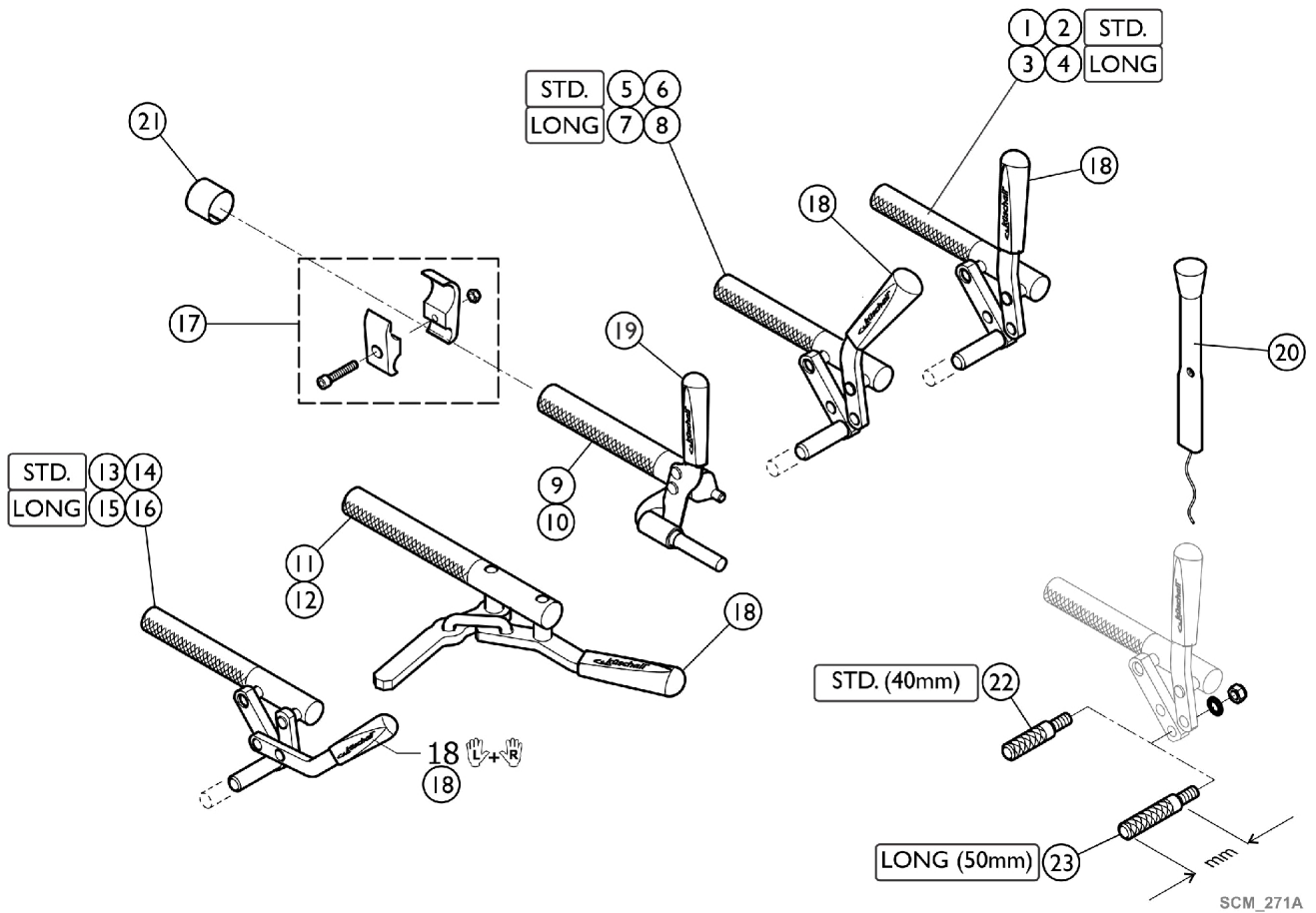

## Rear Wheels - Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1512127	Brake, Straight Lever, Left	1	1
2		1512128	Brake, Straight Lever, Right	1	1
3		1517658	Brake, Straight Lever, Long Shoe, Left	1	1
4		1517657	Brake, Straight Lever, Long Shoe, Right	1	1
5		1512125	Brake, Bent Lever, Left	1	1
6		1512126	Brake, Bent Lever, Right	1	1
7		1517656	Brake, Lever Bent, Long Shoe, Left	1	1
8		1517655	Brake, Lever Bent, Long Shoe, Right	1	1
9		SP1512131	Performance Brake Left	1	1
10		SP1512132	Performance Brake Right	1	1
11		SP1512133	Active Brake Left	1	1
12		SP1512134	Active Brake Right	1	1
13		1512129	Pull Brake Left	1	1
14		1512130	Pull Brake Right	1	1
15		1517660	Pull Brake, Long Shoe Left	1	1
16		1517659	Pull Brake, Long Shoe Right	1	1
17		1513176	Clamping piece , Brake, set	1	1
18		1513177	Brake Handle Standard (Pair)	2	1
19		1513178	Brake Handle, Performance Brake (Pair)	2	1
20		1513179	Brake Lever Extension (Pair)	2	1
21		1544424	Sleeves, Brake Clamping, Titanium Frame (Pair)	2	1
22		1579328	Kit, Brake Shoe Standard (40mm)	1	1
23		1579329	Kit, Brake Shoe Long (50mm), 4 Pcs.	1	1