


Backrest - Standard

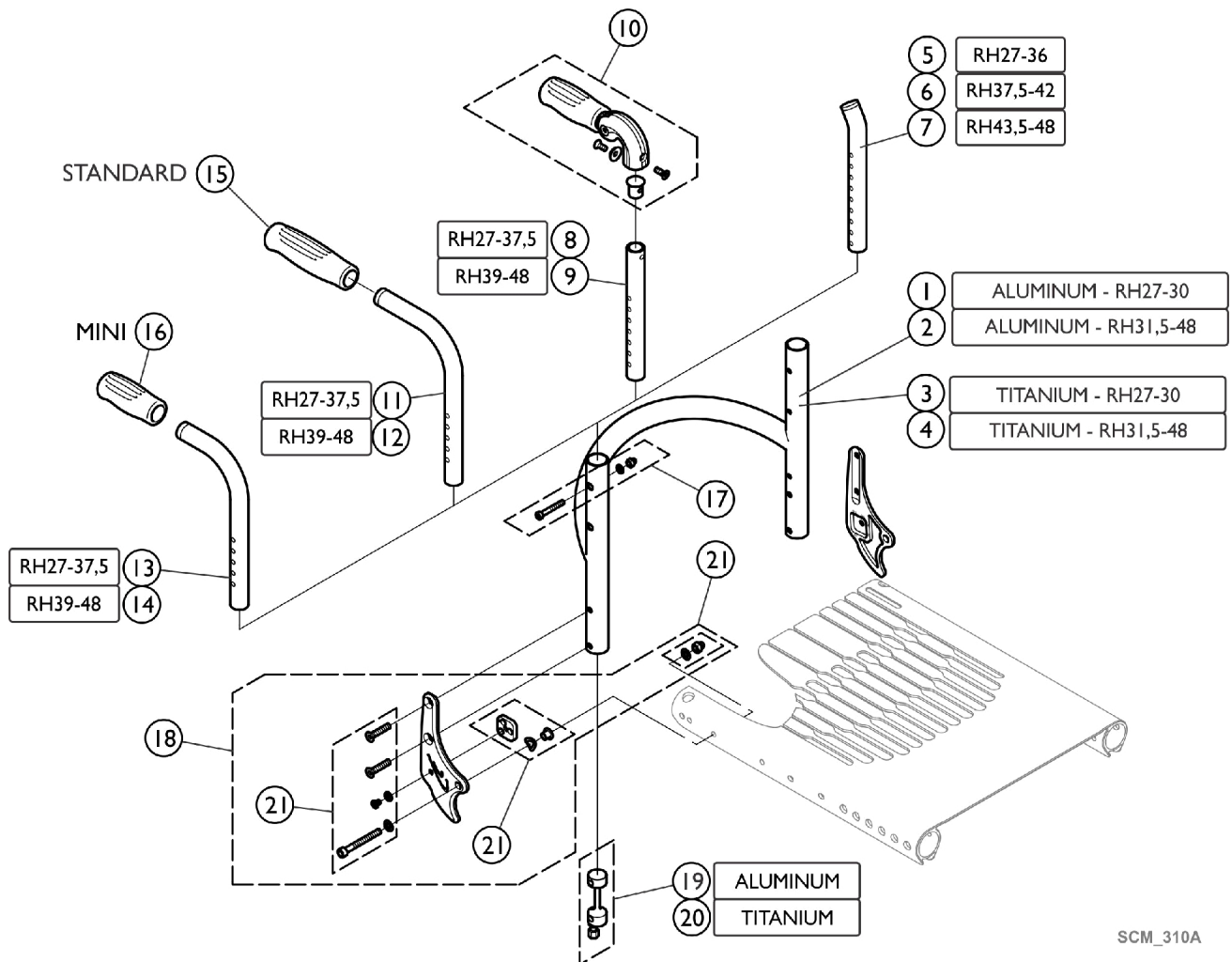

SCM_310A

Backrest - Standard

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-4		See Chart	Backrest - Standard (Backrest Tube)	1	1
5		1546913	Telescopic Back Tube, Standard - Short (Pair)	2	1
6		1546914	Telescopic Back Tube, Standard - Medium (Pair)	2	1
7		1546946	Telescopic Back Tube, Standard - Long (Pair)	2	1
8		1524956	Telescopic Tube Short, Foldable Push Handle (Pair)	2	1
9		1525121	Telescopic Tube Medium, Foldable Push Handle (Pair)	2	1
10		1507733	Foldable Push Handles (Pair)	2	1
11		1525122	Backrest Tube Standard, Short (Pair)	2	1
12		1525123	Backrest Tube Standard, Long (Pair)	2	1
13		1525124	Backrest Tube Mini, Short (Pair)	2	1
14		1525125	Backrest Tube Mini, Long (Pair)	2	1
15		1507734	Handle, Black (Pair)	2	1
16		1525126	Handle Black, Mini (Pair)	2	1
17		1546947	Kit, Hardware, Backrest Tube	1	1
18		See Chart	Item #18. Backrest - Joint Plate (Pair)	1	1
19		1546948	Sleeves, Backrest Tube, Aluminum, Complete (Pair)	2	1
20		1546949	Sleeves, Backrest Tube, Titan, Complete (Pair)	2	1
21		1560913	Kit, Hardware, Backrest Joint	1	1

Backrest - Standard (Backrest Tube)

Tube Sizes	Item #1 Aluminum Short	Item #2 Aluminum Long	Item #3 Titanium Short	Item #4 Titanium Long
SB34	1546694	1546702	1546810	1546818
SB36	1546695	1546739	1546811	1546819
SB38	1546696	1546740	1546812	1546820
SB40	1546697	1546741	1546813	1546821
SB42	1546698	1546806	1546814	1546822
SB44	1546699	1546807	1546815	1546823
SB46	1546700	1546808	1546816	1546824

Item #18. Backrest - Joint Plate (Pair)

Joint Plate Color	Part Number
Polished	1560881-0001
Black	1560881-0004
Champagne	1560881-0182
Techno-Anthracite	1560881-0400
Comet Red	1560881-0440

Backrest - Standard

Item #18. Backrest - Joint Plate (Pair)

Joint Plate Color	Part Number
Artic Blue	1560881-0441