
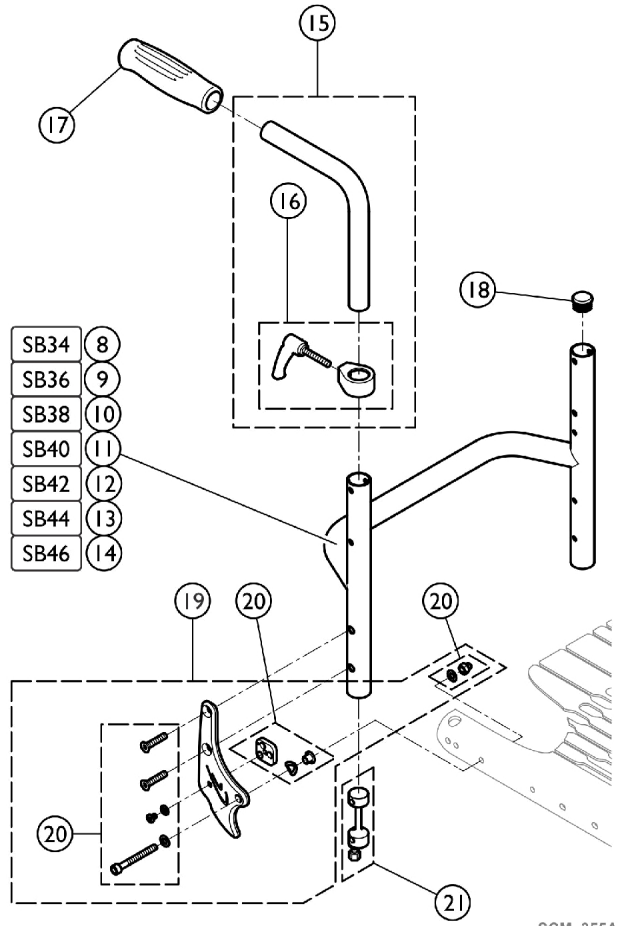


Backrest - Matrix Back

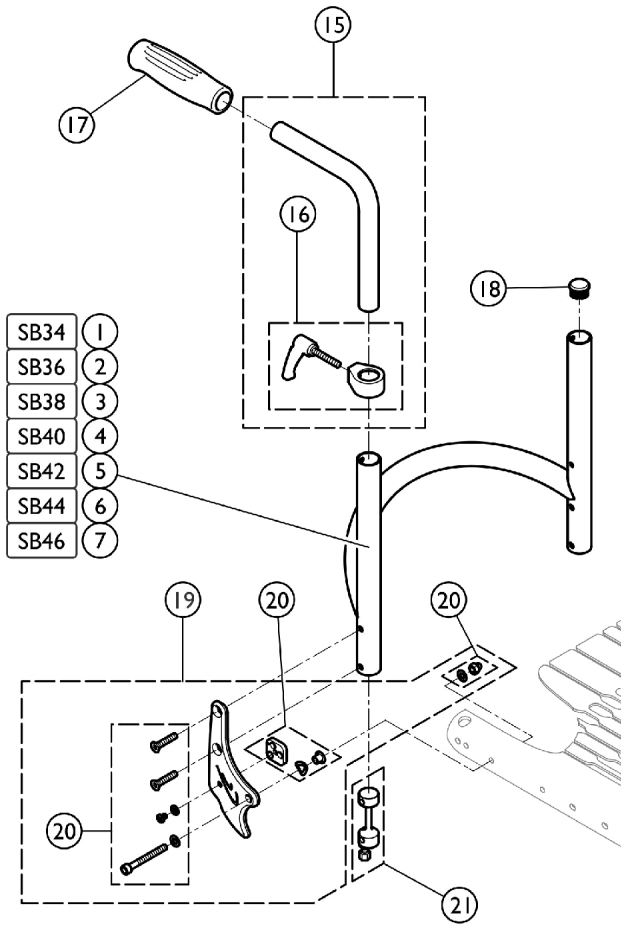

Backrest - MatrX Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1560914	Backrest, Fixed Backrest, SB34	1	1
2		1561012	Backrest, Fixed Backrest, SB36	1	1
3		1561013	Backrest, Fixed Backrest, SB38	1	1
4		1561014	Backrest, Fixed Backrest, SB40	1	1
5		1561015	Backrest, Fixed Backrest, SB42	1	1
6		1561016	Backrest, Fixed Backrest, SB44	1	1
7		1561017	Backrest, Fixed Backrest, SB46	1	1
8		1582698	Straight Bar, Backrest Tube, Aluminum, SB34 Long	1	1
9		1582699	Straight Bar, Backrest Tube, Aluminum, SB36 Long	1	1
10		1582700	Straight Bar, Backrest Tube, Aluminum, SB38 Long	1	1
11		1582701	Straight Bar, Backrest Tube, Aluminum, SB40 Long	1	1
12		1582702	Straight Bar, Backrest Tube, Aluminum, SB42 Long	1	1
13		1582703	Straight Bar, Backrest Tube, Aluminum, SB44 Long	1	1
14		1582704	Straight Bar, Backrest Tube, Aluminum SB46 Long	1	1
15		1438203	Kit, Adjustable Height, Integrated Push Handles, Standard, Short	1	1
16		1549985	Adjusting Lever, Adjustable Height, Integrated Push Handles (Pair)	1	1
17		1507734	Handle, Black (Pair)	2	1
18		1561018	Tube, Plug Ø24mm (Pair)	2	1
19		1560881-0001	Backrest Joint Plate (Pair), Chrome	2	1
19		1560881-0004	Backrest Joint Plate (Pair), Black	2	1
19		1560881-0013-N LA	NLA - Backrest Joint Plate (Pair), Dark Grey	2	1
19		1560881-0182	Backrest Joint Plate (Pair), Champagne	2	1
19		1560881-0400	Backrest Joint Plate (Pair), Techno-Anthrazit	2	1
19		1560881-0440	Backrest Joint Plate (Pair), Comet Red	2	1
19		1560881-0441	Backrest Joint Plate (Pair), Arctic Blue	2	1
20		1560913	Kit, Hardware, Backrest Joint	1	1
21		1546948	Sleeves, Backrest Tube, Aluminum, Complete (Pair)	2	1