

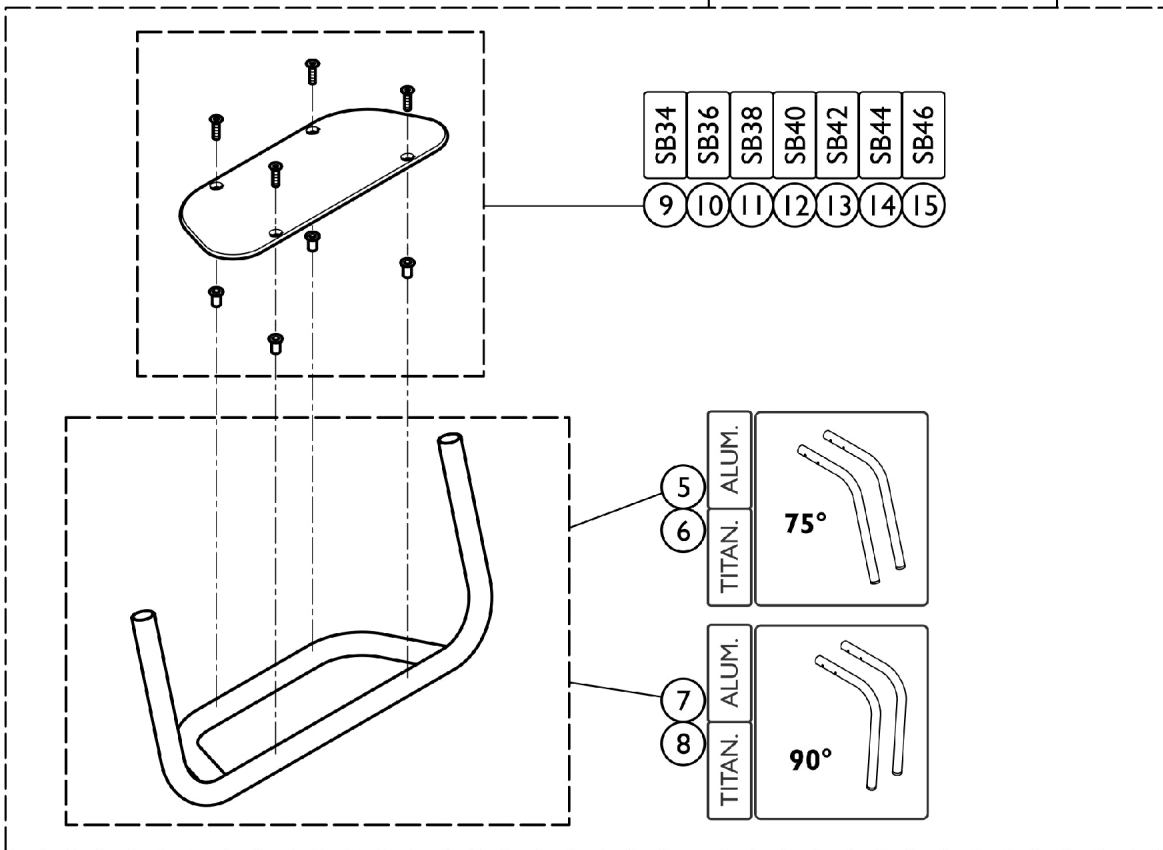
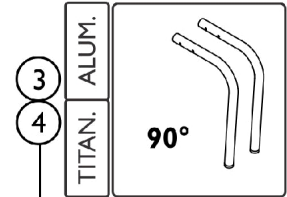
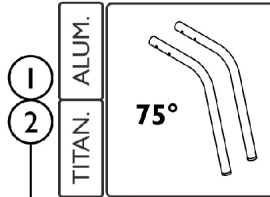
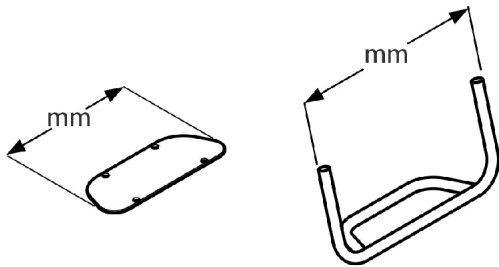


# Footrest - Standard Frame

## STANDARD FOOTREST

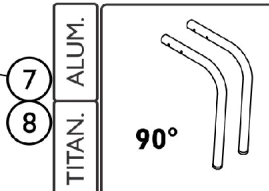

- |      |      |      |      |      |      |      |
|------|------|------|------|------|------|------|
| SB34 | SB36 | SB38 | SB40 | SB42 | SB44 | SB46 |
| 9    | 10   | 11   | 12   | 13   | 14   | 15   |

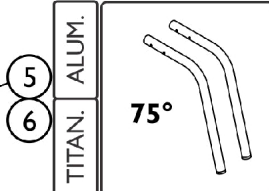

## Footrest - Standard Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-4		See Chart	Items #1-4. Footrest Assembly	1	1
5-8		See Chart	Items #5-8. Footrest Tube	1	1
9		1561130	Footplate 203mm	1	1
10		1561131	Footplate 223mm	1	1
11		1561132	Footplate 243mm	1	1
12		1561133	Footplate 263mm	1	1
13		1561134	Footplate 283mm	1	1
14		1561135	Footplate 303mm	1	1
15		1561136	Footplate 323mm	1	1

### Items #1-4. Footrest Assembly

	75 Degree	75 Degree	90 Degree	90 Degree
Option	Item #1 Aluminum	Item #2 Titanium	Item #3 Aluminum	Item #4 Titanium
SB34	1561090	1561110	1561100	1561120
SB36	1561091	1561111	1561101	1561121
SB38	1561092	1561112	1561102	1561122
SB40	1561093	1561113	1561103	1561123
SB42	1561094	1561114	1561104	1561124
SB44	1561095	1561115	1561105	1561125
SB46	1561096	1561116	1561106	1561126

### Items #5-8. Footrest Tube

	75 Degree	75 Degree	90 Degree	90 Degree
Option	Item #5 Aluminum	Item #6 Titanium	Item #7 Aluminum	Item #8 Titanium
SB34	1561031	1561063	1561053	1561073
SB36	1561032	1561064	1561054	1561074
SB38	1561033	1561065	1561055	1561075
SB40	1561046	1561066	1561056	1561076
SB42	1561047	1561067	1561057	1561077
SB44	1561048	1561068	1561058	1561078
SB46	1561049	1561069	1561059	1561086