


# Footrest - Adducted Frame - High Mounted

## ANGLE ADJUSTABLE - HIGHMOUNTED

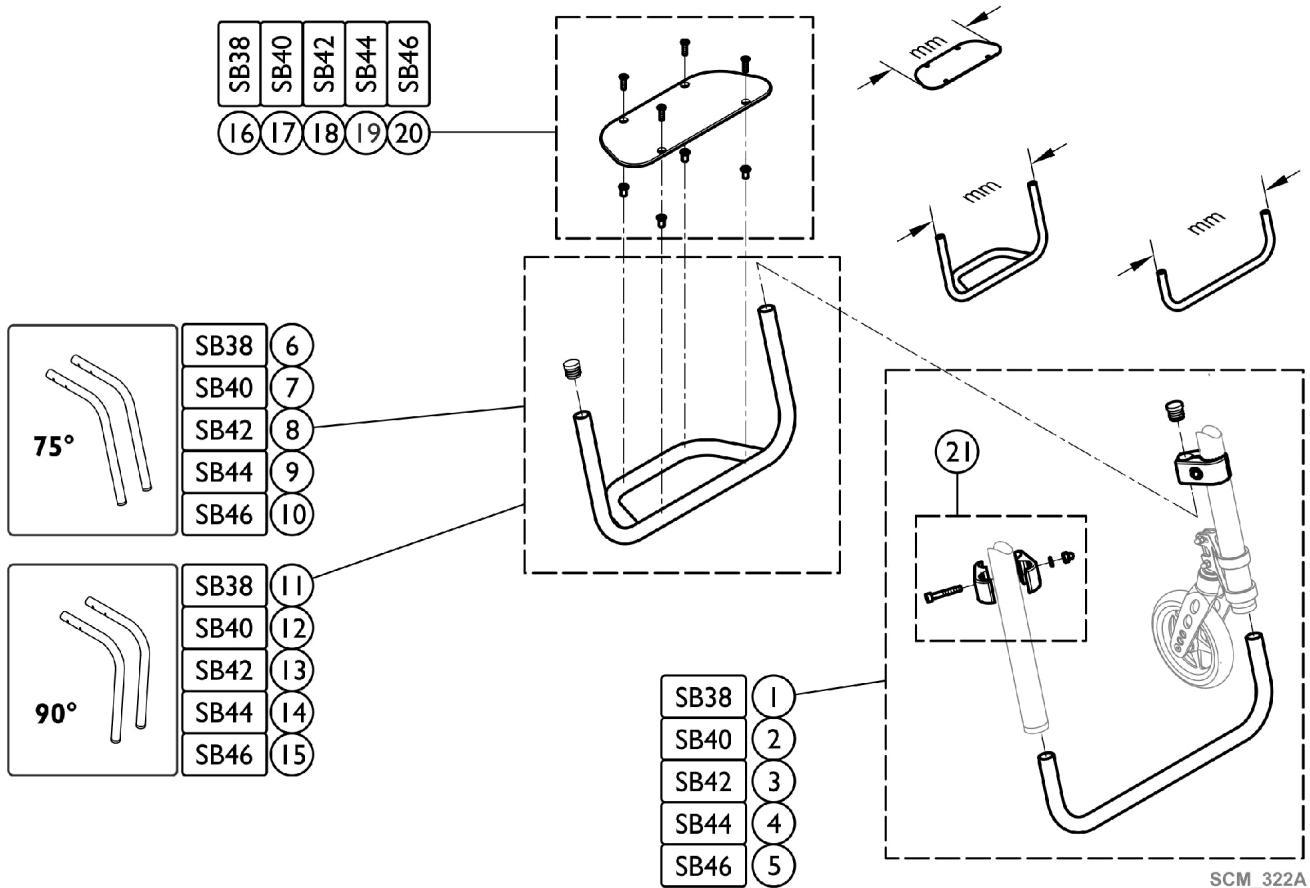

## Footrest - Adducted Frame - High Mounted

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1561215	Footrest - Adducted Frame - High Mounted	1	1
2		1561216	Tube Set, Footrest, High Position, 335mm	1	1
3		1561217	Tube Set, Footrest, High Position, 355mm	1	1
4		1561218	Tube Set, Footrest, High Position, 375mm	1	1
5		1561219	Tube Set, Footrest, High Position, 395mm	1	1
6		1561224	Footrest, Aluminum 295mm, Frame 75° Footrest, High Mount	1	1
7		1561225	Footrest, Aluminum 315mm, Frame 75° Footrest, High Mount	1	1
8		1561292	Footrest, Aluminum 335mm, Frame 75° Footrest, High Mount	1	1
9		1561293	Footrest, Aluminum 355mm, Frame 75° Footrest, High Mount	1	1
10		1561294	Footrest, Aluminum 375mm, Frame 75° Footrest, High Mount	1	1
11		1561300	Footrest, Aluminum 295mm, Frame 90° Footrest, High Mount	1	1
12		1561301	Footrest, Aluminum 315mm, Frame 90° Footrest, High Mount	1	1
13		1561302	Footrest, Aluminum 335mm, Frame 90° Footrest, High Mount	1	1
14		1561303	Footrest, Aluminum 355mm, Frame 90° Footrest, High Mount	1	1
15		1561304	Footrest, Aluminum 375mm, Frame 90° Footrest, High Mount	1	1
16		1561129	Footplate 183mm	1	1
17		1561130	Footplate 203mm	1	1
18		1561131	Footplate 223mm	1	1
19		1561132	Footplate 243mm	1	1
20		1561133	Footplate 263mm	1	1
21		1544613	Kit, Clamping, Footrest, High Mounted (Pair)	2	1