


Frame - Quick Release - 75 degree

See "Footrest - Quick Release Frame" to match Footrest to Frame

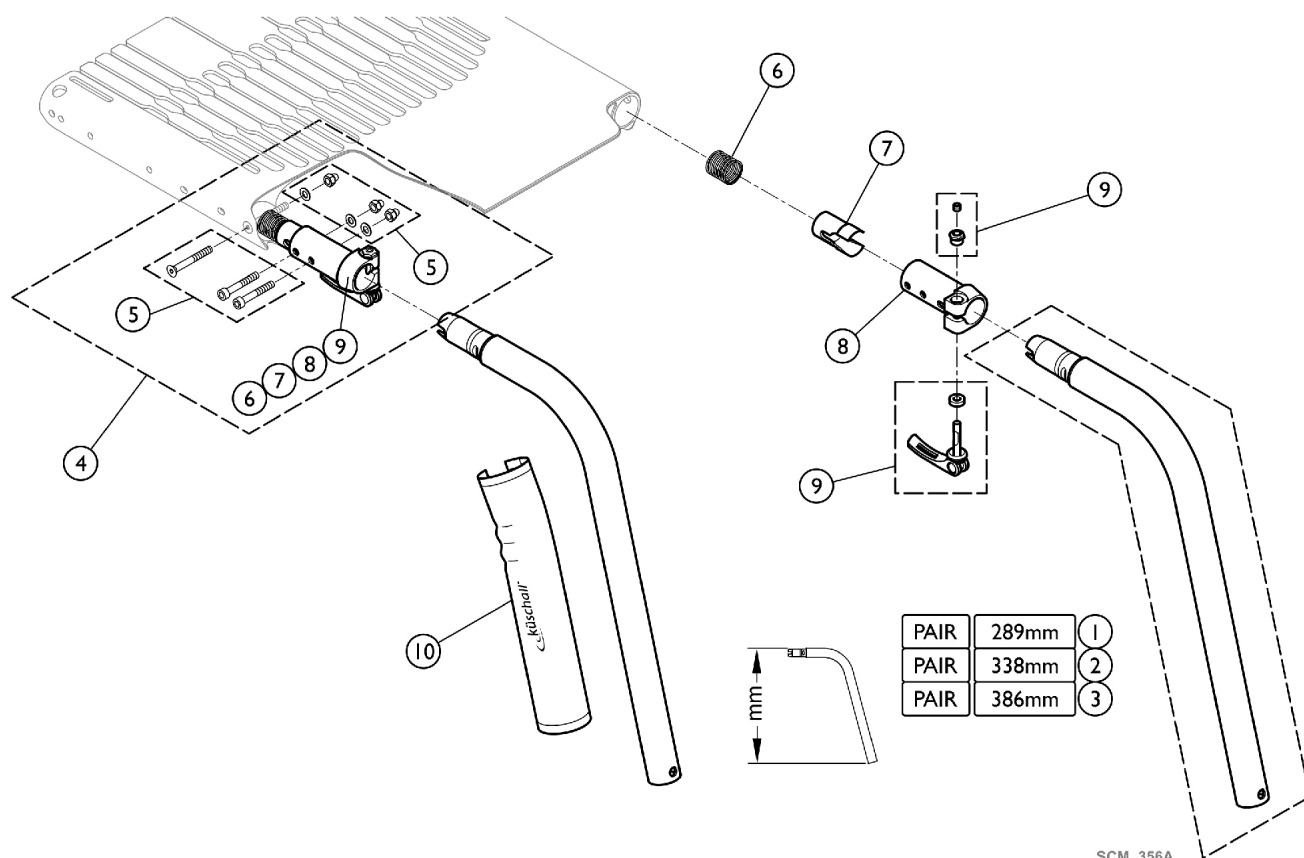

SCM_356A

Frame - Quick Release - 75 degree

See "Footrest - Quick Release Frame" to match Footrest to Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-3		See Chart	Frame - Quick Release - 75 Degree	2	1
4		1567250	Kit, Female Coupling, Quick Release Frame, Complete (Pair)	2	1
5		1567251	Kit, Hardware, Seat Module Coupling Female, Quick Release Frame (Pair)	2	1
6		1567252	Spring, Quick Release Frame (Pair)	2	1
7		1567320	Sleeve, Quick Release Frame (Pair)	2	1
8		1567321	Female Coupling, Quick Release Frame (Pair)	2	1
9		1567322	Lever, Complete, Quick Release Frame (Pair)	2	1
10		1529372	Front Frame Protectors (Pair)	2	1

Frame - Quick Release - 75 Degree

Color	Item #1 - Short (Pair)	Item #2 - Medium (Pair)	Item #3 - Long (Pair)
Mercury Blue	1567244-0184	1567245-0184	1567246-0184
Dynamite Orange	1567244-0185	1567245-0185	1567246-0185
Polished	1567244-0410	1567245-0410	1567246-0410
Matte Black	1567244-0411	1567245-0411	1567246-0411
Ivory White	1567244-0445	1567245-0445	1567246-0445
Flashy Yellow	1567244-0451	1567245-0451	1567246-0451
Royal Red	1567244-3524	1567245-3524	1567246-3524
Royal Blue	1567244-5969	1567245-5969	1567246-5969