


Seating - Seat Module

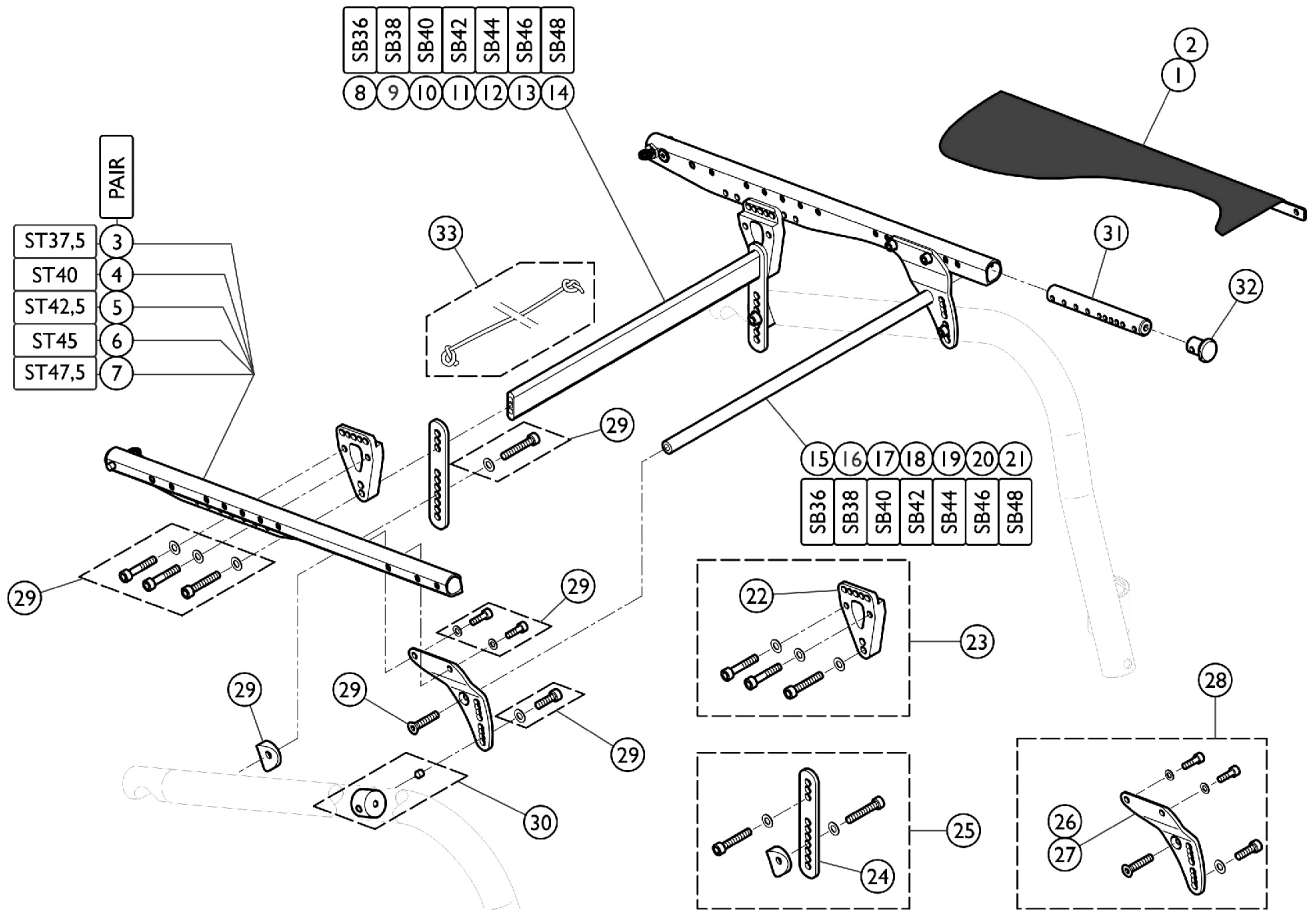

Seating - Seat Module

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SPS1584000	See Category "Seating - Standard Seat Cover"	1	1
2		SPS1584001	See Category "Seating - Adjustable Cover"	1	1
3-7		See Chart	Items #3 - 7. Seat Module	1	1
8		1583976	Seat Brace Horizontal Rear, 282mm	1	1
9		1583977	Seat Brace Horizontal Rear, 302mm	1	1
10		1583978	Seat Brace Horizontal Rear, 322mm	1	1
11		1583979	Seat Brace Horizontal Rear, 342mm	1	1
12		1583980	Seat Brace Horizontal Rear, 362mm	1	1
13		1583981	Seat Brace Horizontal Rear, 382mm	1	1
14		1583982	Seat Brace Horizontal Rear, 402mm	1	1
15		1583983	Seat Brace Horizontal, 289mm	1	1
16		1583984	Seat Brace Horizontal, 309mm	1	1
17		1583985	Seat Brace Horizontal, 329mm	1	1
18		1583986	Seat Brace Horizontal, 349mm	1	1
19		1583987	Seat Brace Horizontal, 369mm	1	1
20		1583988	Seat Brace Horizontal, 389mm	1	1
21		1583989	Seat Brace Horizontal, 409mm	1	1
22		1583990	Hinge Bracket Seat Frame, Rear	1	1
23		1583991	Hinge Bracket-Set Seat Frame, Rear (Pair)	2	1
24		1583992	Seat Brace Vertical	1	1
25		1583993	Seat Brace-Set Vertical (Pair)	2	1
26		1583994	Hinge Bracket Front Seat, Right	1	1
27		1583995	Hinge Bracket Front Seat, Left	1	1
28		1583996	Hinge Bracket-Set Front Seat Frame (Pair)	2	1
29		1583997	Kit, Hardware, Hinge Brackets	1	1
30		1583998	Threadet Sleeve (Pair)	2	1
31		1583999	Thread Insert Front Seat Frame (Pair)	2	1
32		1486539-0004	Seat Module Plug, Black Anodized (Pair)	2	1
33		1554427	Cord, Backrest, Foldable	1	1

Items #3 - 7. Seat Module

Item Number	Size	Pair Part No.
3	ST37,5	1583961
4	ST40	1583962
5	ST42,5	1583963
6	ST45	1583964
7	ST47,5	1583965