


Footrest - High Mounted - Adducted Frame

FOOTREST - HIGH MOUNTED - ADDUCTED FRAME

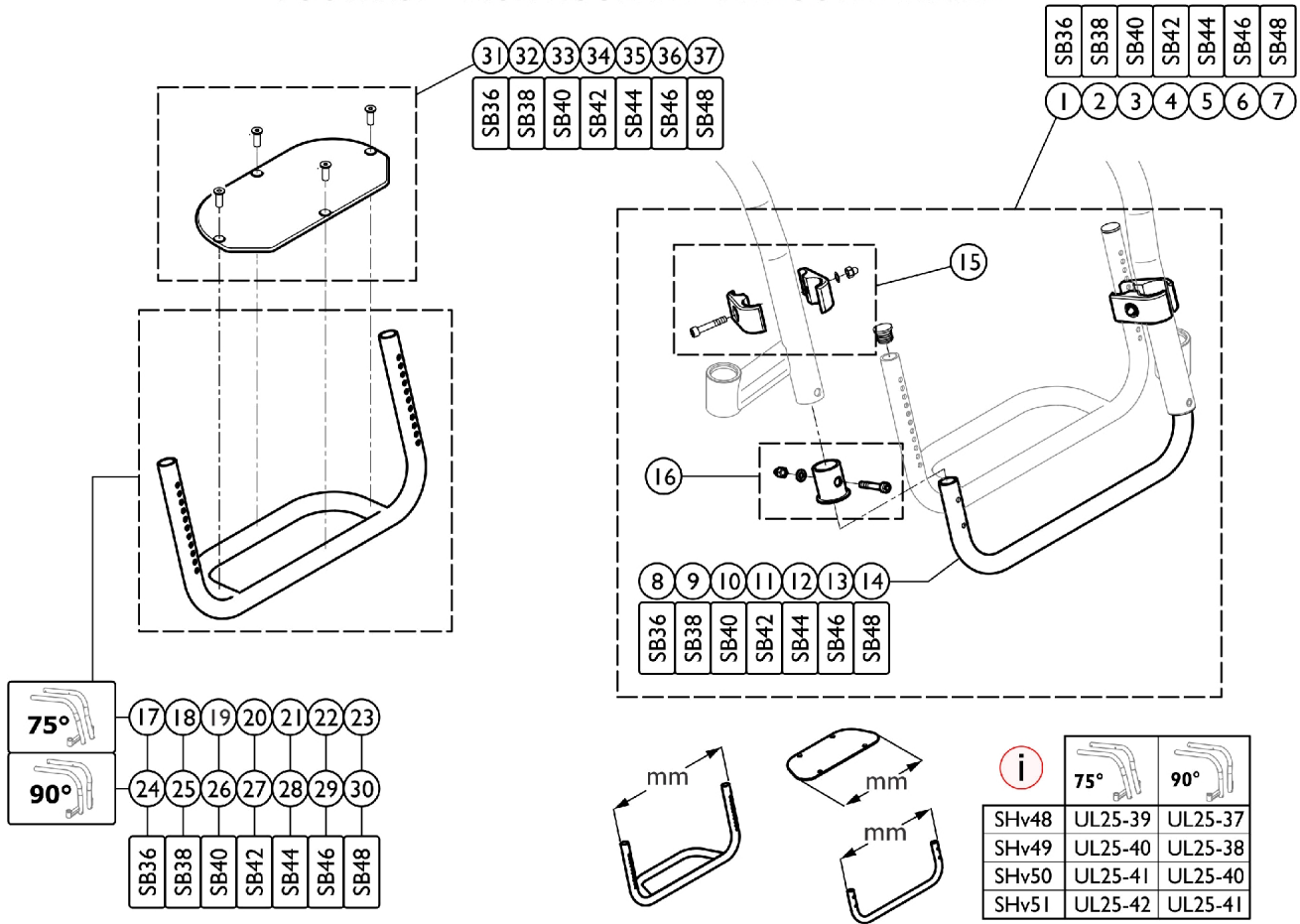

Footrest - High Mounted - Adducted Frame

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1583641 | Tube Set, Footrest, High Position, Adducted Frame (310mm) | 1 | 1 |
| 2 | | 1583642 | Tube Set, Footrest, High Position, Adducted Frame (330mm) | 1 | 1 |
| 3 | | 1583643 | Tube Set, Footrest, High Position, Adducted Frame (350mm) | 1 | 1 |
| 4 | | 1583644 | Tube Set, Footrest, High Position, Adducted Frame (370mm) | 1 | 1 |
| 5 | | 1583646 | Tube Set, Footrest, High Position, Adducted Frame (390mm) | 1 | 1 |
| 6 | | 1583647 | Tube Set, Footrest, High Position, Adducted Frame (410mm) | 1 | 1 |
| 7 | | 1583648 | Tube Set, Footrest, High Position, Adducted Frame (430mm) | 1 | 1 |
| 8 | | 1583650 | Tube, Footrest, High Position, Adducted Frame (310mm) | 1 | 1 |
| 9 | | 1583651 | Tube, Footrest, High Position, Adducted Frame (330mm) | 1 | 1 |
| 10 | | 1583652 | Tube, Footrest, High Position, Adducted Frame (350mm) | 1 | 1 |
| 11 | | 1583653 | Tube, Footrest, High Position, Adducted Frame (370mm) | 1 | 1 |
| 12 | | 1583654 | Tube, Footrest, High Position, Adducted Frame (390mm) | 1 | 1 |
| 13 | | 1583655 | Tube, Footrest, High Position, Adducted Frame (410mm) | 1 | 1 |
| 14 | | 1583656 | Tube, Footrest, High Position, Adducted Frame (430mm) | 1 | 1 |
| 15 | | 1495577 | Kit, Clamping Pieces, Footrest, High Position | 1 | 1 |
| 16 | | 1583602 | Footrest Sleeve, Aluminum | 1 | 1 |
| 17-30 | | See Chart | Items #17-30. Footrest Tube | 1 | 1 |
| 31 | | 1583595 | Footplate (174mm) | 1 | 1 |
| 32 | | 1583596 | Footplate (194mm) | 1 | 1 |
| 33 | | 1583597 | Footplate (214mm) | 1 | 1 |
| 34 | | 1583598 | Footplate (234mm) | 1 | 1 |
| 35 | | 1583599 | Footplate (254mm) | 1 | 1 |
| 36 | | 1583600 | Footplate (274mm) | 1 | 1 |
| 37 | | 1583601 | Footplate (294mm) | 1 | 1 |

Items #17-30. Footrest Tube

| Sizes | 75 Degree Part Number | Sizes | 90 Degree Part Number |
|------------------|-----------------------|------------------|-----------------------|
| SB36 Item #17 | 1583657 | SB36 Item #24 | 1583703 |
| SB38 Item #18 | 1583658 | SB38 Item #25 | 1583750 |

Footrest - High Mounted - Adducted Frame

Items #17-30. Footrest Tube

| Sizes | 75 Degree Part Number | Sizes | 90 Degree Part Number |
|------------------|--------------------------|------------------|--------------------------|
| SB40 Item #19 | 1583659 | SB40 Item #26 | 1583942 |
| SB42 Item #20 | 1583660 | SB42 Item #27 | 1583943 |
| SB44 Item #21 | 1583661 | SB44 Item #28 | 1583944 |
| SB46 Item #22 | 1583701 | SB46 Item #29 | 1583945 |
| SB48 Item #23 | 1583702 | SB48 Item #30 | 1583946 |