
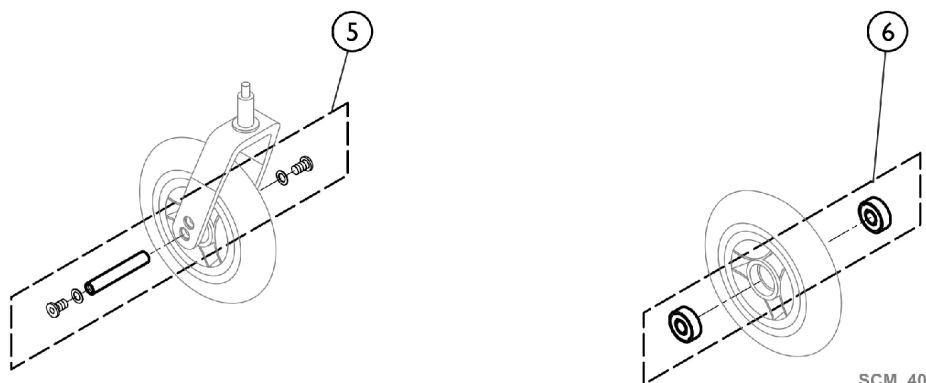


Front Wheels - Casters

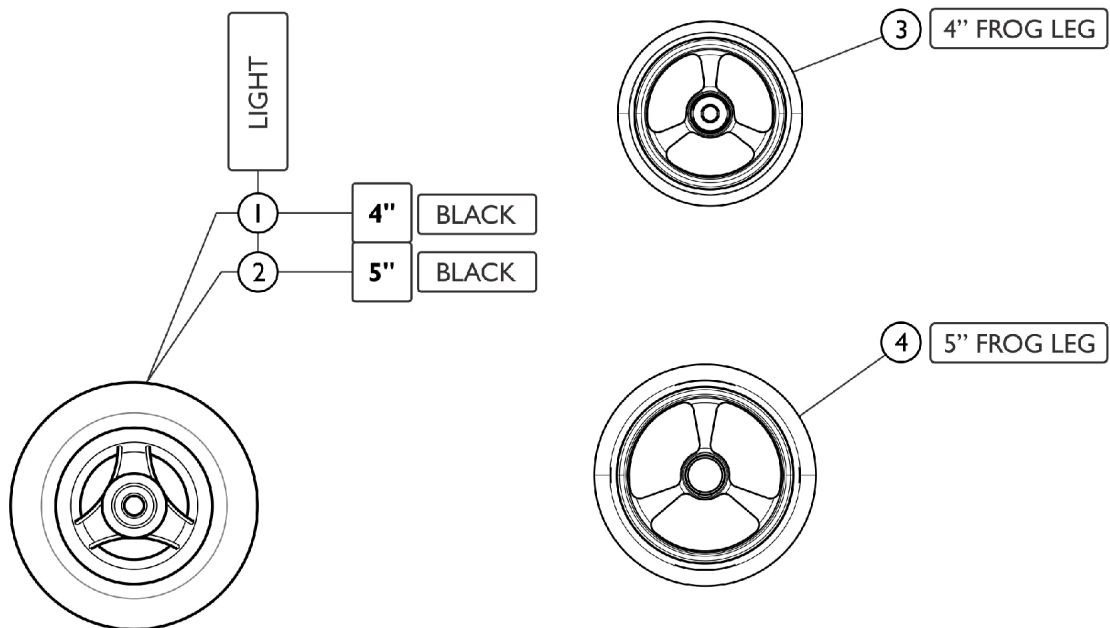

SCM_403B

Front Wheels - Casters

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1562626-NLA	NLA - 4" Everyday Wheel, Three Spokes, Black	1	2
2		SP1638071	5" Everyday Wheel, 3 Spokes, Black (Pair)	2	1
3		SP1638032-NLA	NLA - 4" Caster, Black, Aluminum Rim, Frog Legs	1	2
3		SP1638083	4" Caster, Silver, Aluminum Rim, Frog Legs	1	2
4		SP1638084	5" Caster, Black, Aluminum Rim, Frog Legs	1	2
4		SP1638085	5" Caster, Silver, Aluminum Rim, Frog Legs	1	2
5		1570108	Axle Set w/out Spacer	1	2
6		1570105	Ball Bearing 6000-2RS (Pair)	2	2