


Rear Wheels - Wheel Locks

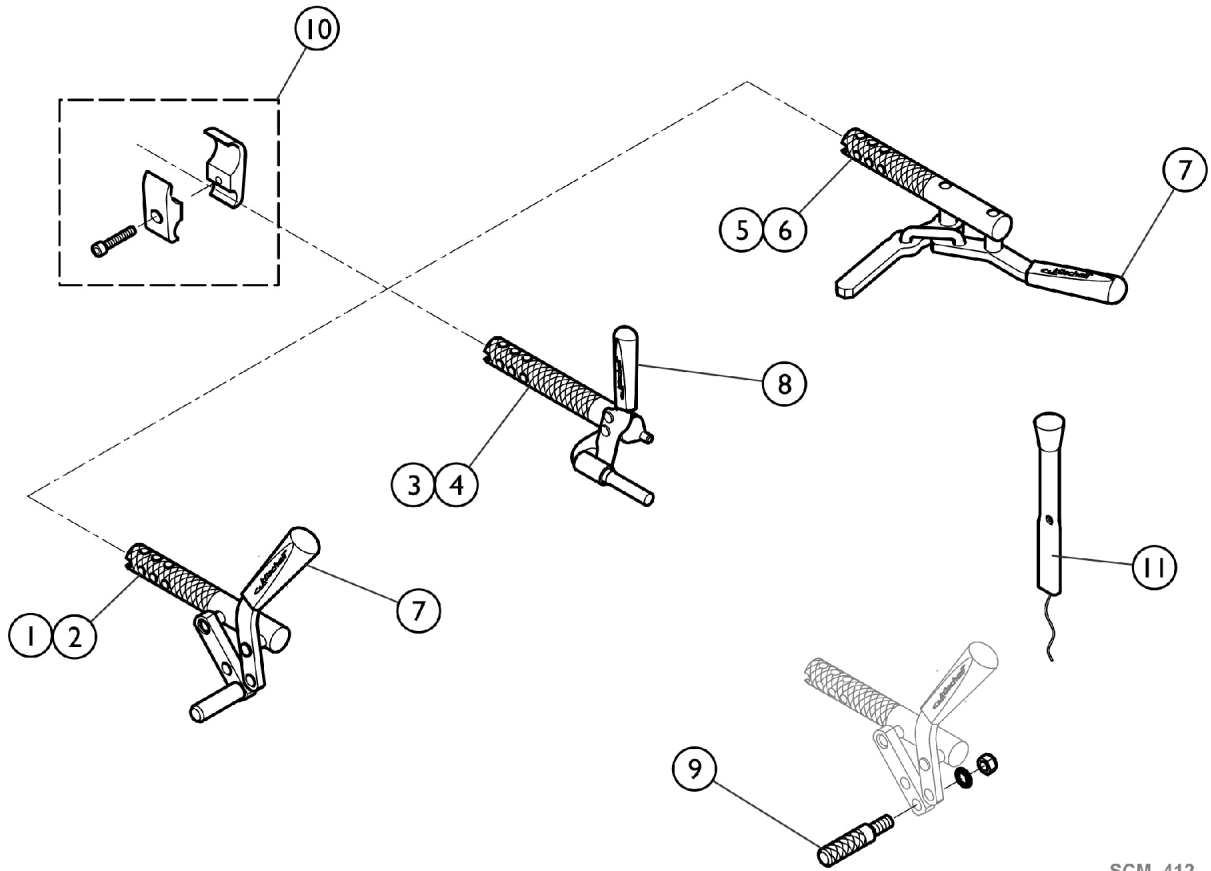

SCM_412

Rear Wheels - Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1562583	Brake, Bent Lever, Left	1	1
2		1562584	Brake, Bent Lever, Right	1	1
3		1562591	Performance Brake, Left	1	1
4		1562592	Performance Brake, Right	1	1
5		SP1512133	Active Brake Left	1	1
6		SP1512134	Active Brake Right	1	1
7		1513177	Brake Handle Standard (Pair)	2	1
8		1513178	Brake Handle, Performance Brake (Pair)	2	1
9		1579328	Kit, Brake Shoe Standard (40mm)	4	1
10		1562599	Brake Clamp, Set	1	2
11		1513179	Brake Lever Extension (Pair)	2	1