

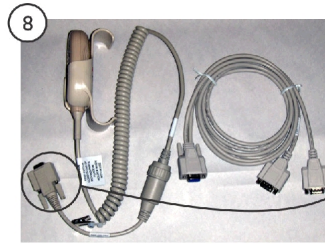
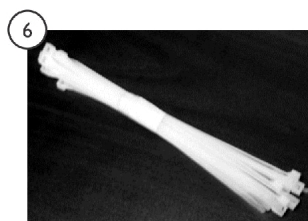
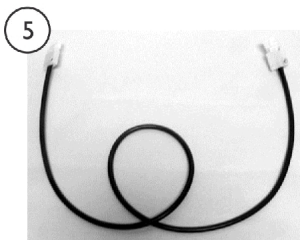
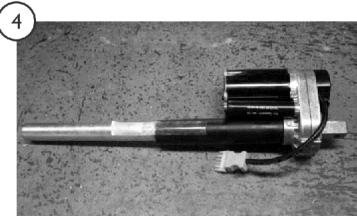


Electronics & Pendants - BD0880/76

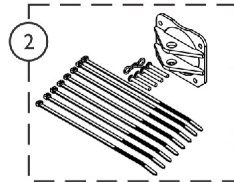




DB9
Connector

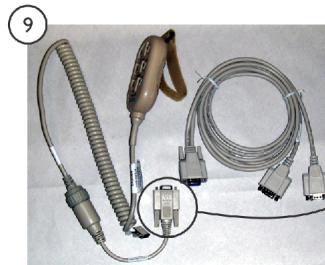

DB9
Connector



DB9
Connector

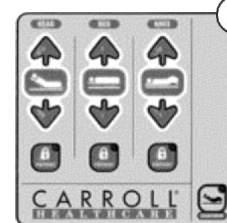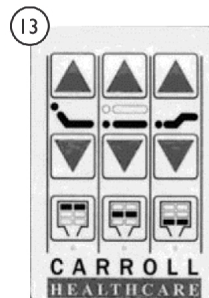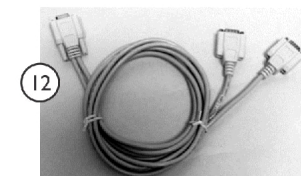

SICC_119A

Electronics & Pendants - BD0880/76

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	MTRM1633	Actuator, Head (Red Plug) Ref. #61831-0795280-01	1	1	
2		KIT80-NLA	NLA - Kit, Bracket Trunion for Head Motor	1	1	
			Pin, Cotter Bow-Tie (1/4 x 3/32 x 5/16")	1	1	
			Screw, Torx HD Taptite LG (8-32 x 1-1/4")	1	4	
			Bracket, Trunnion Plated	1	1	
			Clevis Pin (3/8 x 2-1/2" L)	1	2	
			NLA - Wrap, Tie (8")	1	8	
			Sheet, Instruction, Bracket Solo Head/HiLo Motors	1	1	
3			MTRM1634	Actuator, Foot (Blue Plug) Ref. #V07508AT760	1	1
4			MTRM17781	Actuator, Hilo (White Plug) Ref. #61831-1405220-01	1	1
5			MISC022	Cable, Extension - Head Motor	1	1
6			1186883-NLA	NLA - Wrap, Tie (8")	1	5
7			JNCTNBOXKIT2-NLA	NLA - Junction Box Ref. #NAX5105-3A29MPZ	1	1
8			1190228	Kit, Pendant, New Assist Rails, Linak # PNDNT-6FNC-1-2	1	1
9			1190233	Kit, Pendant, Full/Partial Rails, Linak # PNDNT-6FNC-1-2 (With DB9 Connector)	1	1
10	B	PNDNTKIT3-NLA A	NLA - Kit, Pendant, Assist Rails (Old Square), Linak # PNDNT-6FNC-1-2 (With DB9 Connector) (May 01 - July 02)	1	1	
11	C	1190225	Cable, Conversion	1	1	
12		PNDNTYCABLE	Cable, Y, Pendant	1	1	
13		OLD KEYPAD-NLA	NLA - Membrane Pad for 151 Only	1	1	
14		JNCTNDC20C	Membrane Pad for 151 Only	1	1	
NOTE: A - Trunnion KIT80C is NLA. Prior to Aug. 1999 HiLo Motor requires item #5 Motor Extension Cable and item #6 Wire Ties. B - Used from 5/01-7/02 C - Beds built Before 8/16/04 need to replace Head and Footboard with New Membrane Pad.						